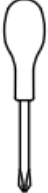
SPALLIERA DOPPIA – DOUBLE WALL BARS 04510

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The company produces in accordance with the quality standard UNI EN ISO 13485:2016

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INTENDED USE AND EQUIPMENT DESCRIPTION

Swedish beech-wood wall bars with double span, are natural varnished and supplied with anchoring white steel clamps galvanized.

The wall bars can be used to carry out some gymnastic and rehabilitation exercises: muscle and joint stretching, corrective exercises of various paramorphisms and they can also be used by children for exercises of joint looseness.

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SPECIAL WARNING AND USE RESTRICTIONS

These devices must not be used:

- without making all the settings required and recommended setups.
- when use is not advisable or causes the patient serious discomfort.
- if the original product has been tampered with and/or modified
- when set incorrectly or when there is any suspicion that it may harm the patient

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COLLATERAL EFFECTS AND CONTRAINDICATIONS

There are not known collateral effects or contraindications during proper use. Use the device always in the proper way. The knowledge of this instruction manual is recommended

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The specialist must assess the adequacy of the patient's condition before the use

The wall bars must be installed only on concrete walls or on perfored brick walls.

Check the suitability of the wall before installation. For further information about the installation eventually consult the manufacturer of the fixing dowels (HILTI item HRD-K)

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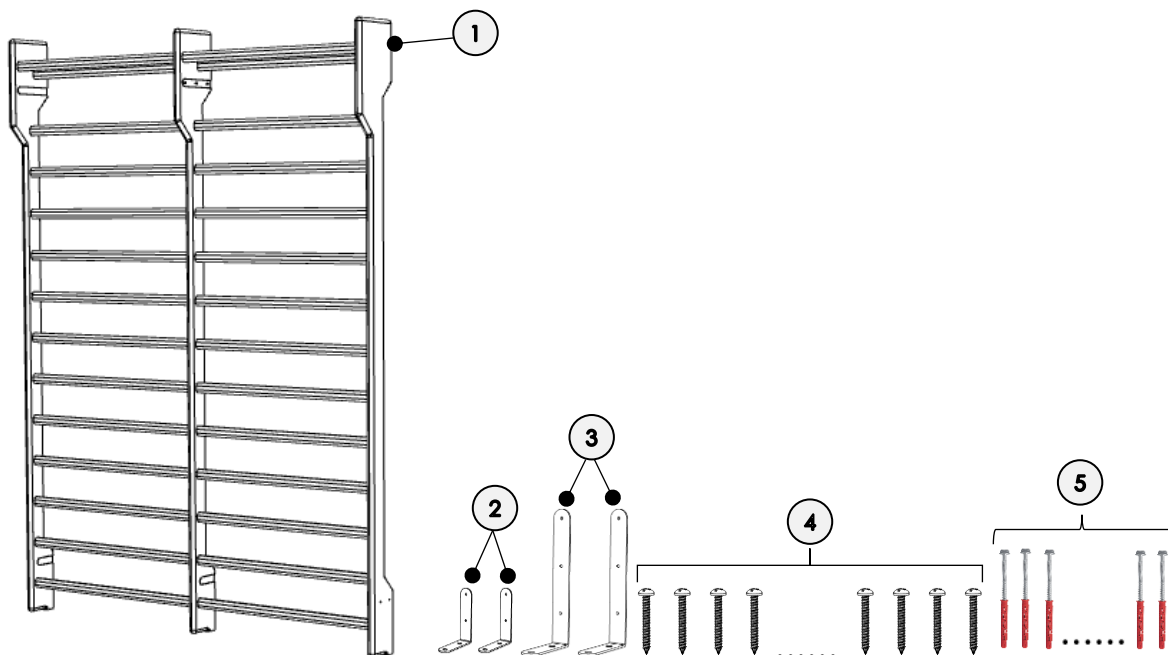
It is under the user's responsibility to seek the most suitable fixing methods in case of installations on walls other than those envisaged

The manufacturer shall not be liable, to the fullest extent permitted by applicable law, for any direct or indirect, special, incidental or consequential damages caused by:

- o Wrong use of the device
- o Wrong installation of the device
- o Improper use of the device and outside of its intended use
- o Using the device in environments not covered by this manual
- o Use with unsuitable patients
- o The distraction of operators or incorrect application of commands and adjustments
- o Use without checking the status of the device as described in the relevant paragraph
- o Incorrect maintenance or lack of maintenance
- o Use with parts or accessories that are not compatible or not approved by the manufacturer
- o Incorrect disposal or disposal is other than as described in this manual

UNPACKING

Simply remove the device from the package. **Never use cutters** to open box to prevent damages. Properly dispose of the packaging.

The carrier and the supplier must be immediately notified of any damage that may have occurred during transport.

Damaged equipment cannot be used

PACKAGE CONTENT

- (1) Wall bars – 1 pz
- (2) Lower brackets – 3 pz
- (3) Upper brackets – 3 pz
- (4) Bracket fixing screws – 15 pz
- (5) Dowels with screws – 12 pz

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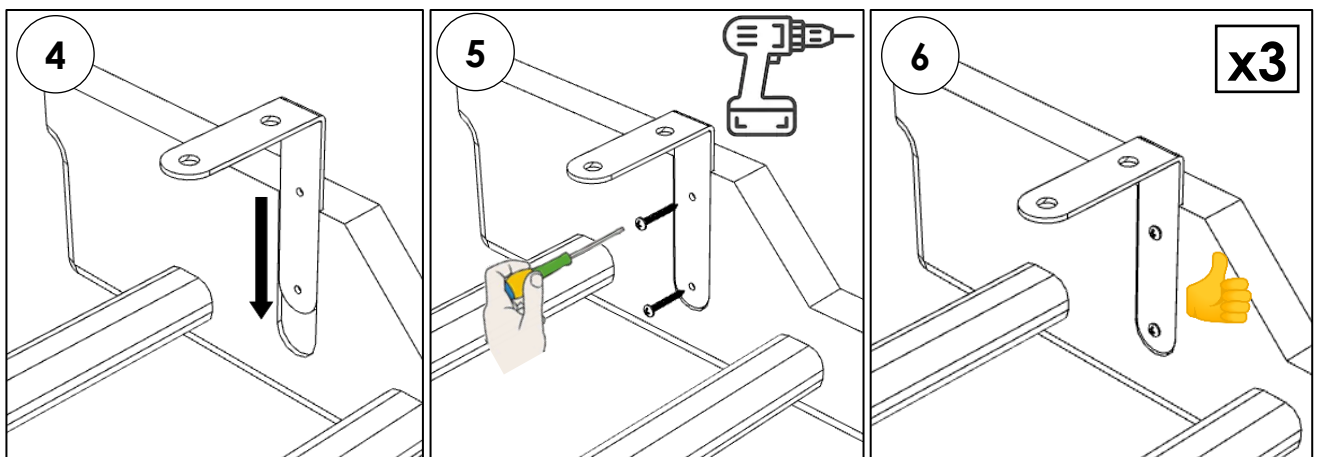
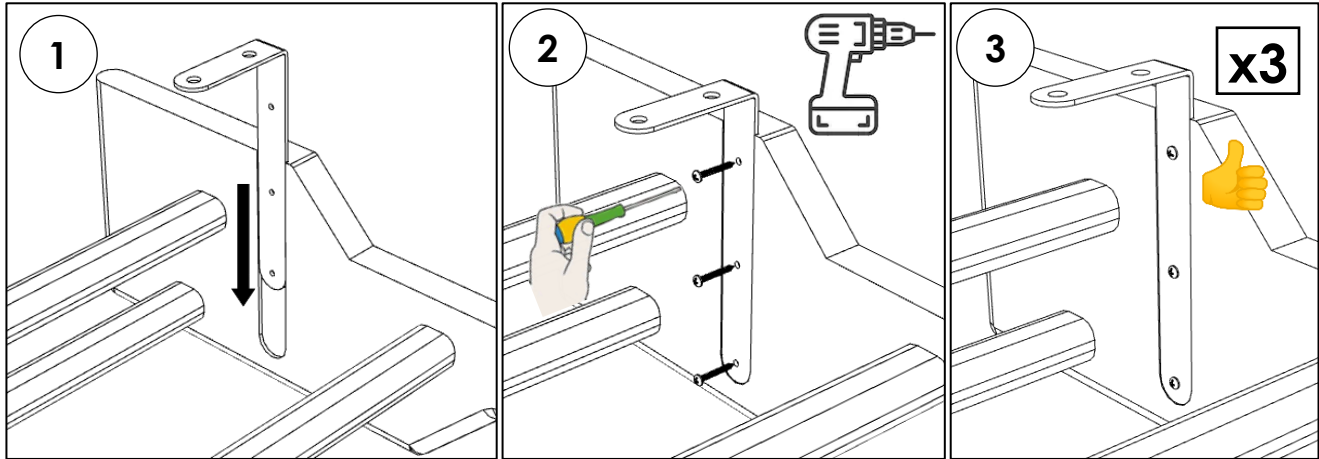
Check that the device does not show signs of damage and that all the contents of the package are present

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- Place the backrest on the floor and insert the upper brackets into the guides.
- Screw the supplied self-tapping screws (3 screws for each upper bracket) so that the bracket is secured to the backrest. Use a screwdriver or a drill driver.
- Check that the bracket is installed as shown in the picture
- Place the lower brackets in the guides
- Tighten the supplied self-tapping screws (2 screws for each lower bracket) so that the bracket is secured to the backrest. Use a screwdriver or a drill driver.
- Check that the bracket is installed as shown in the picture

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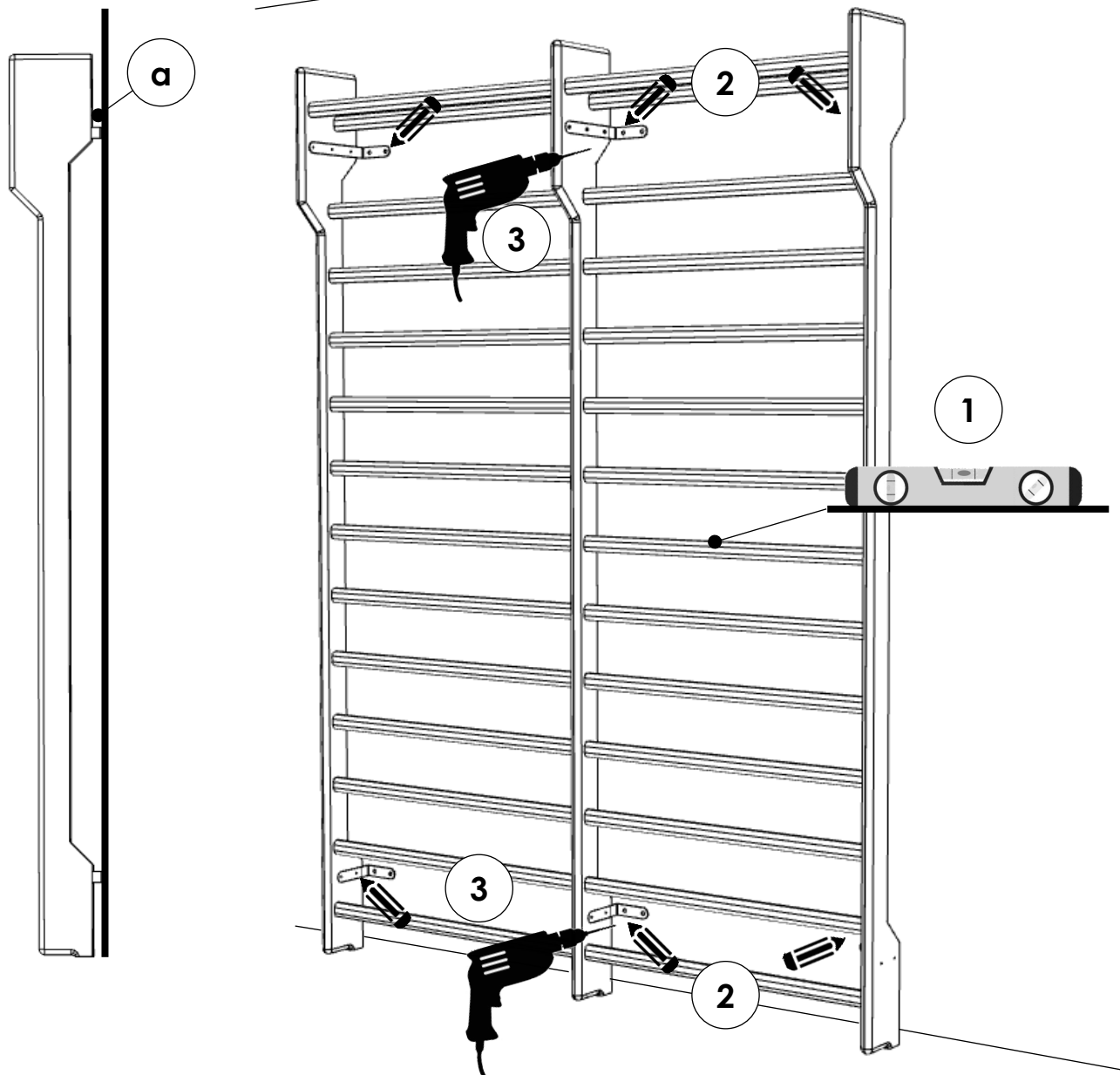
WALL FIXING

- Place the backrest close to the wall, check with a spirit level.
- The brackets are made in such a way that the backrest is always detached from the wall (a) to allow installation even in the presence of skirting boards.

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Mark the fixing holes with a pencil. Two holes are required for each bracket.

Remove the backrest and drill with a 10" drill bit. Insert the wall plugs into the wall, position the backrest and secure the brackets to the wall with the screws.

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MAINTENANCE

Check the condition of the safety of the device after each situation of "rough treatment" (ex.: fall, heavy impacts, liquid spills, etc.) and any time you are unsure about the state security apparatus.

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Routine maintenance tasks are to be carried out at least once every twelve months.
Keep constantly clean

FREQUENCY	TASK
12 months	General cleaning Visual and functional inspection of the equipment Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and bushing clearance; check for any damage or deformation

CLEANING

Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water. Dry avoiding direct exposure to the sunlight.

The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution.

DISINFECTION

For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1%. Rinse abundantly.

WARRENTY

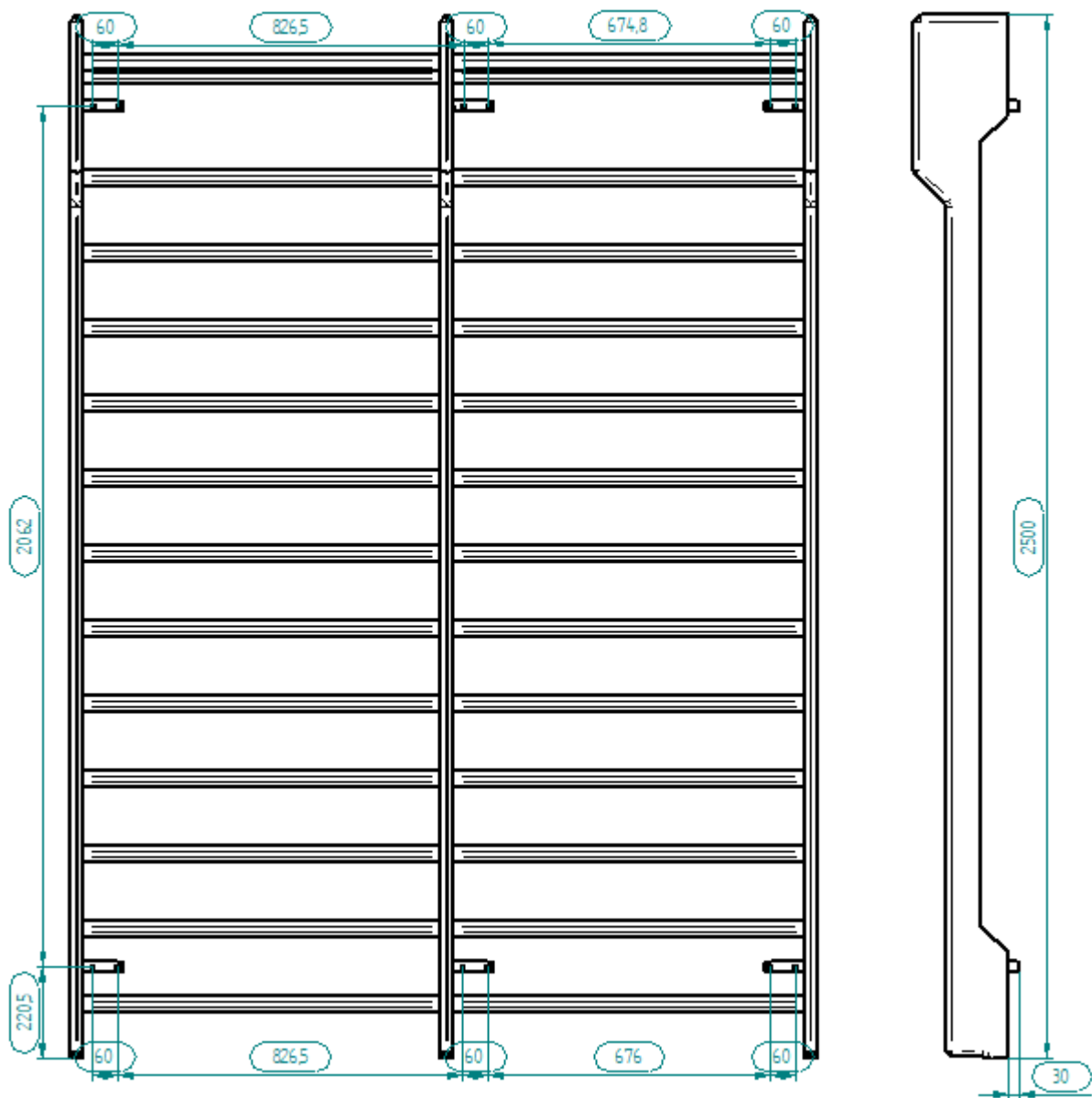
All Chinesport products are warranted against defects in materials or workmanship for a period of 24 months from the date of sale of the product, except for any exclusions, limitations or conditions defined at the time of delivery of the product.

The warranty does not cover consumables, rechargeable batteries, and in general, all material subject to wear, failures caused by impacts, falls, misuse or improper, accidental events, damage caused by transport. If the equipment is tampered with, the warranty expires automatically.

ADDITIONAL INFORMATION

www.chinesport.com

TECHNICAL DATA

 CHINESPORT SPA
 Via Croazia 2
 33100 UDINE
 Italy

MADE IN ITALY