EXAMINATION AND TREATMENT BASIC TABLES

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ENGLISH VERSION

1 GENERAL INFORMATIONS

1.1 Purpose, content and target audience

This manual is intended to provide the necessary information to permit a proper use of the equipment and to be able to use it independently and safely.

This manual contains information relating to the technical, operation, maintenance, spare parts and safety. This book is also intended to technicians in charge of maintenance equipment.

This Instruction Booklet should always follow the equipment and must be carefully preserved in order not to compromise its readability.

In case of transfer to third parties this manual has to be transmitted with other documents as required by European Directive 93/42 EEC.

1.2 Manufacturer

CHINESPORT S.P.A, Via Croazia, 2-33100 Udine - Italy

tel. +39 0432621621-fax+39 0432621620 - website: www.chinesport.com.

The company produces according a quality system by standard UNI EN ISO 13485:2012.

1.3 Intended use and equipment description

Metal Beds visit for physical therapy, manually adjustable in height for some versions. Equipped with head and / or sections can be manually adjusted.

1.4 Application Environment

Medical devices covered by this manual are in conformity according to attached VII of the directive 93/42/EEC amended by 2007/47/CE

In particular according to UNI CEI EN 60601-2-52:

ENVIRONMENT OF APPLICATION 5 - outpatient care administered in a hospital or other medical center under medical supervision, in which the EM devices are provided for the needs of sick people who have suffered injuries or are carriers of disability, for the treatment, diagnosis and monitoring.

1.5 Products insights

Further documentation can be found on our website www.chinesport.com.

Specially, you can reading the QR code on the cover of this manual to quickly connect to the website. Where provided QR code can also be found on the product.

You can download declaration of conformity and other product documents in their latest version and in color format directly from website.

1.6 Storage

The following environmental conditions must be ensured when the equipment is to be stored:

- 1. relative humidity 10% / 90 %;
- 2. temperature -10°C / +50°C.

1.7 Symbols

The symbols found in this manual and on the machine are used to draw the user's attention to important safety warnings and useful information. Observe all warning, caution and danger messages.

WARNING

This symbol identifies operating/maintenance instructions or procedures which, if not observed, could result in injury or danger to the user.

CAUTION

This symbol is used to warn the user that not to observe a certain operating or maintenance procedure, general rule or condition, could result in damage to the equipment.

2 SPECIAL WARNINGS AND USE RESTRICTIONS

01250. 01650. 01252. 01652. 01251. 01651.

	MAX PATIENT WEIGHT 180 kg
<u>^</u>	safe working load - swl 200 kg

01256, 01257, 01253, 01254, 01423,

	MAX PATIENT WEIGHT 220 kg
<u>^</u>	safe working load - swl 250 kg

This equipment must not be used:

- By inadequately trained operators.
- When having been tampered with and/or if the original product has undergone changes.
- On unstable and/or uneven surfaces (e.g.: stairway, steps, etc.).
- With loads greater than those declared to be safe by the manufacturer.
- In an improper configuration or when it is thought to be harmful for the patient.
- In association with other devices that may interfere with the equipment.
- In an explosive and/or corrosive atmosphere or exposed weather conditions.

In addition:

- Keep body parts away from moving parts and/or adjustments. These areas are marked by labels.
- Two or more people are required to carry the equipment.
- Set the bed in its lower and horizontal position when it is not supervised.

- Radiating therapy lamps can damage the upholstery.
- Corrosive and/or abrasive detergents can damage the upholstery.
- Periodically disinfect the table's leatherette.
- Record any significant safety-related intervention/defect/damage in the attached form.
- For operations not covered in the maintenance list, please contact Chinesport Service Centre.

2.1 Collateral effects and contraindications

There is no collateral effects or contraindications during proper use, is recommended knowledge of this instruction manual.

3 CONFIGURATED ITEM CODE

Product code		*	*	*	*	*	W	*
Variable code position		1	2	3	4	5	6	7
Position	Meaning variable code	Possible values						

I	1-2-3-4-5	Fixed	value reference to catalogue
	6-7	TYPE AND COLOR COATING	REFERENCE TO CATALOGUE

4 USE

4.1 Unpacking

Remove the equipment from the packaging and put the padded panels on a floor protected in order not to to damage the padding. Do not open the box with cutter to avoid damaging the padding.

After removing the bed of the box provide for the removal of the various protections used in the packaging.

Any transport damage must be immediately reported to the carrier and to the supplier: the damaged equipment must not be used.

4.2 Starting

Only models 200x100cm: Enter the crossbar between the side members.

Only models New Metal sections 3: Attach the bracket "U" on the side members with the screws.


Pull the legs to the side members, with the support for the rack on the side of the panel headrest.

Partially fix all the anchoring screws inserted into the barrels longoni (see diagram).

Screw the rack in support of the cross leg.

Put the bed in the position of use, check the alignments and finish tightening with the wrench provided.

Only for code 01423: to connect 2 beds, loosen the screws of the legs, apply blades and connect with supplied screws and tighten. Tighten screws (see picture).

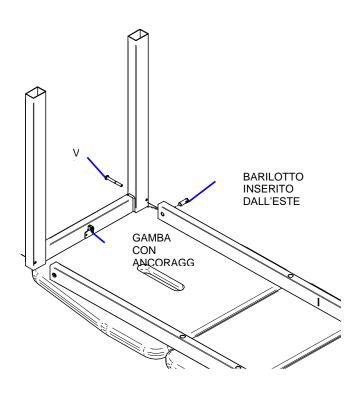01423 METAL L200

4.2.1 Adjusting the head section

Lift the platform up to the desired; return it to its basic position, you must lift it until it stops and then accompany him to the base.

4.2.2 4.2.2 Adjusting sections footboard

Lift the flatbed up to the desired, to bring it back to the home position, it is necessary to raise the floor reggicosce and accompany him to the position.

WARNINGS AFTER A FEW WEEKS OF USING THE COUCH, REPEAT THE TIGHTENING OF THE LEGS

4.2.3 Main adjustment configurations

MODEL	DIMENSIONS SINGLE PLATFORM AND POSTURE	MODEL	DIMENSIONS SINGLE PLATFORM AND POSTURE
01650 - METAL BASIC 1	+45° 68 124 195	01651 - METAL BASIC	445° 68 68 54 33 192
01652 - METAL BASIC L	1-45° 68 80 124 195	01251 - METAL 3	+45° +45° 68 68 54 33 192
01250 - METAL 1 01252 - METAL L	+45° 68/80 124 195	01257 - METAL HT100 01254 - METAL LT100	100 135 200
01256 - METAL H100 01253 - METAL L100	100 200	01423 - METAL L200	100 200


5 MAINTENANCE

Only qualified personnel may carry out maintenance on the equipment. Technicians must be familiar with the model, use and maintenance of the equipment.

Check the equipment whenever it has undergone a "rough treatment" (e.g.: fall, liquid spill, etc.) and whenever any doubts arise concerning its safety.


5.1 Routine maintenance

Routine maintenance must be carried out at least once every 12 months, following the table below: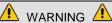

- WHEN SAFETY IS AT RISK, DO NOT USE IT UNTIL THE DANGER HAS BEEN ELIMINATED.
- ANY SIGNIFICANT WORK CARRIED OUT TO ENSURE THE COUCH IS SAFE FOR USE MUST BE RECORDED IN THE RELEVANT FORM (ATTACHED)

FREQUENCY	OPERATION
	General clean of the equipment
	Visual and functional inspection of the equipment
yearly	Frame and mechanical parts: check screws and bolts to ensure they are tight; pin and
	bushing clearance; check for any damage or deformation
	Check that the adjustable sections are stable and do not collapse
AFTER THE	ABOVE OPERATIONS, COMPLETE INSPECTION BY CARRYING OUT A FULL
LIFTING CYCL	LE USING THE MAXIMUM SAFETY WORKING LOAD.

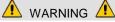FOR OPERATIONS NOT COVERED IN THE MAINTENANCE LIST, PLEASE CONTACT CHINESPORT SERVICE CENTRE

5.2 Non routine maintenance

Contact Chinesport service for all operations not mentioned in the maintenance list.

5.3 Spare parts

Only use original spare parts. Contact CHINESPORT S.p.a. for a spare parts list.


6 TROUBLESHOOTING

The equipment has been tested in very harsh conditions: during the test, there were no particular breakage and / or failure. In case of failure / breakage, you can open a support ticket through our website www.chinesport.it the "Support" section.

In case of troubleshooting table not solve your fail, you can open a service ticket means website www.chinesport.com customer service section.

ATTENTION:

Before calling our technical support, be sure to have your **SERIAL NUMBER** or the "Codice" and "Articolo" to hand: you can find it on the product label located on the equipment (as shown in the facsimile above); or the transport document / invoice number. This identification number helps reduce waiting times and improves service quality.

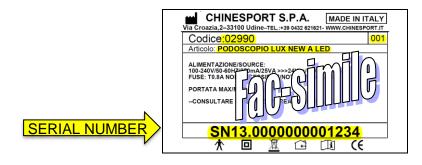NOTE:

The equipment in need of repair at Chinesport premises must carry the "AUTHORIZATION NUMBER" on the package. This identification number, to be required from the Technical Assistance Service, helps reduce waiting times and improves service quality.

Alternatively, you can contact the technical support by email (assistenza@chinesport.it), for users without internet connection it's possible to send a fax (0432 621 698) or phone (0432 621 699), but the timing of response could be longer.

7 CLEANING AND DISINFECTION

Sky is the flame retardant fabric made for residential and contract upholstery. Its resistant structure and the easy cleaning make sky particularly suitable for hospitals, nursing homes, spas, rehabilitation centers and gyms. Sky has a leather-effect surface with a brilliant finishing, in a wide range of colors all available on stock. Sky contains Vinyzene, an anti-microbial agent added in the first steps of production process. This creates a defensive barrier against mites, bacteria and fungus, avoiding smells and allergies.

7.1 Washing

Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water without squeezing. Let the fabric dry avoiding direct exposure to the sunlight. The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink,

coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.


7.2 Disinfection

7.2.1 Surface disinfection

For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1% at the following spectrum of action:

- bactericidal in the presence of interfering substances according to EN 1276:1997;
- fungicide in the presence of interfering substances according to EN 1650:1997;
- battericidal test according EN 13697:2001;
- fungicide test according EN 13697:2001;
- active on Salmonella tiphymurium according EN 13697:2001;
- active on HIV, HAV HBV, HCV;
- virucidal according EN14476:2005;
- virucidal influenza H1N1 according EN14476:2005.

Before use is recommended to read specific flyer.

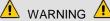- REGULARLY DISINFECT THE SYNTHETIC LEATHER AND USE DISPOSABLE PAPER ROLL TO KEEP THE COUCH CLEAN
- AGGRESSIVE PRODUCTS CAN DAMAGE SURFACES
- BEFORE USING THE EQUIPMENT CHECK THAT ALL COMPONENTS ARE COMPLETELY DRY
- TO AVOID PERMANENT STAINS, THE LIGHTER COLORS OF THE ARTICLE SHOULD NOT BE BROUGHT INTO CONTACT WITH TEXTILES AND CLOTHING COLORED WITH UNFIXED DYES. WE CANNOT GUARANTEE THE ABILITY TO REMOVE UNKNOWN DYES FROM LIGHT COLORED PRODUCT

8 REUSE OF THE PRODUCT

These products have been designed so as to be suitable for reuse by another user after execution of some works and the overcoming of controls for its release.

To prepare the products at re-use by another user must be done the following:

- cleaning and disinfection carefully with adequate means:
- maintenance according to the maintenance plan and control;
- granting permission for reuse, which must accompany the product.

The above transactions must be carried out by the manufacturer or authorized dealer.

9 TECHNICAL SHEET

		F	EATL	JRES							
CODE	Dimensions (cm)	Height (cm)	Weight (kg)	Maximum load to distributed load (kg)	Fireproof leatherette type W	Adjustable sections	Hole nose mouth	12163 Adjustable paper holder	12176 Plug hole nose mouth	12269 Headrest	12270 Armrest
01650	195x68	78	40	200	S	1	-	0	-	0	0
01651	192x68	78	41	200	S	3	-	0	-	0	0
01652	195x80	78	43	200	S	1	-	0	-	0	0
01250	195x68	78	40	200	S	1	S	0	0	0	0
01251	192x68	78	41	200	S	3	S	0	0	0	0
01252	195x80	78	43	200	S	1	S	0	0	0	0
01256	200X100	78	56	250	S	0	-	0	-	-	-
01257	200x100	78	58	250	S	1	1	0	1	-	-
01253	200x100	50	51	250	S	0	1	0	1	-	-
01254	100x200	50	53	250	S	1	1	0	1	-	-
01423	200x200	50	106	250	S	0	-	0	-	-	-

- = not available

S = series

O = optional

10 RELATED ITEMS AND ACCESSORIES

12269 – PILLOW FACE	12270 - ARMREST	AC0020 - TAPPO FORO NASO - BOCCA
Adjustable cervical pillow, only gray	Armrest. Indicated for the discharge of the arms, only gray color. To match the headrest must cod. 12269	Plug the hole for the nose / mouth series
AC0031 – SHEET HOLDER	AC003A - SHEET 2 VEILS	AC0290 - PROTECTION SHEET
	Lenzuolino Visita Januaria	
Sheet holder adaptable to different roll widths	Sheet set 2 ply, pack of 6	Protection sheet for cylinder heads with breathing hole
AC0025 – CUSCINO CON FORO	AC0033 – PORTA LENZUOLINO VERTICALE	AC0035 – LENZUOLO GRANDE
		00
Cuscino con foro, utilizzato sui lettini senza il foro naso/bocca. In caso di abbinamento ad un lettino da visita o terapia è possibile scegliere lo stesso colore di tappezzeria. La gamma si compone di 20 colori. Dimensioni 40x34x9h cm.	Porta lenzuolino verticale. Stativo per rotoli di grandi dimensioni.	Lenzuolo grande, rotolo da cm 90 x mt 100. Confezione da 2 pezzi.

11 WARRANTY

GENERAL AFTER-SALES SERVICE CONDITIONS FOR CHINESPORT ITEMS

The warranty shall be valid for a period of 36 months from the date of delivery.

The warranty does not cover consumable items, rechargeable batteries, and generally all the material subject to wear, breakdowns caused by knocks, falls, misuse or improper use, accidental events and transport damage. This warranty shall be automatically invalidated if the equipment is tampered with.

Repair of the equipment will be carried out in relation to the breakdown stated.

An estimate must be specifically requested.

Payment is cash on delivery, unless other agreements are made.

Any dispute will be dealt with solely by the Court of UDINE.

WARRANTY REPAIRS

Repairs under warranty must be specifically requested.

Repairs under warranty shall be carried out at our workshop and upon authorization, complete with the return authorization number.

The return shipment for products sent in their original package will be made free of carriage charges.

A fiscal document (bill of sale, purchase invoice or cash receipt) is required to make a warranty claim; the purchase date indicated must fall within the warranty time period.

If the product is found to be in good working order without defect, the fault-finding service shall be charged to the customer.

OUT-OF-WARRANTY REPAIRS

SERIAL NUMBER

For out-of-warranty repairs carried out at our workshops upon authorization, transportation charges shall be paid by the customer.

The warranty of the parts replaced during the repair operations is 12 months from the moment the equipment is collected.

If the product is found to be in good working order without defect, the fault-finding service shall be charged to the customer.

REPAIRS MADE AT HOME

EQUIPMENT

In the event repairs are made at the customer's home, a written request is required indicating the complete details of the person making the request, the type of machine and the breakdown.

The cost per kilometer for on-site technician visits will be agreed upon in relation to the urgency of the customer. In the event the machine is under warranty, the customer will have to pay only the expenses incurred for the technician to reach the customer's house.

Time is calculated from the moment the technician leaves our workshop until he/she returns. The time of his/her return will be estimated based on the time required to get there.

SALE OF SPARE PARTS AND CONSUMABLE ITEMS

The orders must be in writing and specify the customer, the item and the shipping method as well as the precise fiscal data of the customer.

12 MAINTENANCE RECORD SHEET

DATE of first installation

The order processing time can vary depending on the availability of the material.

Returned spare parts will not be accepted.

Payment will be cash on delivery, unless other agreements are made.

OPERATIONS CARRIED OUT TECHNICIAN SIGNATURE DATE OF **NEXT** SERVICE **VERIFY** DATE

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
Via Croazia, 2 33100 UDINE - ITALY Tel. +39 0432 621 621 Fax +39 0432 621 620

www.chinesport.it p.i. 00435080304

Some important notes to contact our after-sales customer service

ATTENTION:

Before calling our technical support, be sure to have your **SERIAL** NUMBER or the "Codice" and "Articolo" to hand: you can find it on the product label located on the equipment (as shown in the facsimile above); or the transport document / invoice number. This identification number helps reduce waiting times and improves service quality.

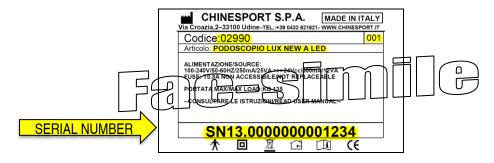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