**01325** rev.001  
PARALLEL BARS PLUS 2M  
*PARALLELA PLUS 2M*

**01326** rev.001  
PARALLEL BARS PLUS 3M  
*PARALLELA PLUS 3M*

**01327** rev.001  
PARALLEL BARS PLUS 3M TILT  
*PARALLELA PLUS 3M TILT*

---

**01391** rev.001  
CHILDREN'S PARALLEL  
BARS PLUS 2M  
*PARALLELA PLUS 2M BIMBO*

**02996** rev.001  
CHILDREN'S PARALLEL BARS  
PLUS 3M  
*PARALLELA PLUS 3M BIMBO*

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## 1 GENERAL INFORMATIONS

<b>PURPOSE AND CONTENT OF INSTRUCTION</b>	<p>The purpose of this manual is to provide users with the necessary information to allow proper use of the equipment and make it possible to use it independently and safely. This manual contains information on technical, operational and maintenance aspects, as well as spare parts and safety. It is also intended for technicians responsible for maintaining the equipment. This Instruction Manual must always be kept with the equipment and must be carefully preserved in order to avoid compromising its readability. If the equipment is sold to third parties this manual has to be handed over with the other documents as required by European Directive 93/42 EEC.</p> <p><u>This manual is not a medical handbook</u></p>
<b>MANUFACTURER</b>	<p><b>CHINESPORT S.P.A</b>, Via Croazia, 2-33100 Udine, Italia.  Phone. +39 0432 621 621 – Fax+39 0432 621 620 – web: <a href="http://www.chinesport.com">www.chinesport.com</a>  The company produces according to UNI EN ISO 13485:2012.</p>
<b>INTENDED USE AND EQUIPMENT DESCRIPTION</b>	<p>Parallel bars allow performing rehabilitation and ambulation exercises safely.. They are made of painted steel with a thermoplastic coated handrail, which can be adjusted in height. Item code 01329 can be adjusted in width as well. Rehabilitation can be diversified by using the available accessories to create different types of paths depending on the exercise : concave, convex, wedge, hemispheres, sensory, obstacle and divider modules. A glider can be supplied upon request to offer greater support to a patient during walking.</p>
<b>APPLICATION ENVIRONMENT</b>	<p>The Medical device described in this manual is in compliance with Directive 93/42 / EEC as amended by 2007/47 / EC in accordance with Annex VII</p>
<b>FURTHER INFORMATIONS AND DOCUMENTATION</b>	<p>Please visit <a href="http://www.chinesport.com">www.chinesport.com</a></p>
<b>STORAGE</b>	<p>In case of storage the following ambient conditions must be respected:  relative umidity 10% / 90 %;  temperature -10°C / +50°C.</p>
<b>SIDE EFFECTS AND CONTRAINDICATIONS</b>	<p>There is no collateral effects or contraindications during proper use. Use the device always in the proper way. The knowledge of this instruction manual is recommended.</p>
 <b>SPECIAL WARNINGS AND USE RESTRICTIONS</b>	<p>These devices must not be used:</p> <ul style="list-style-type: none"> <li>• without making all the settings required and recommended setups.</li> <li>• when use is not advisable or causes the patient serious discomfort.</li> <li>• if the original product has been tampered with and/or modified</li> <li>• when setted incorrectly or when there is any suspicion that it may harm the patient</li> </ul> <p>The plastic and wood parts can catch fire if brought into contact with naked flames.  Do not spill liquids on the metal parts.  The physical and psychological suitability of the user to make use of the aids must be approved by a specialist.  The adjustable mechanical parts can cause injury when handled</p>

## 1.1 SYMBOLS

The symbols found in this manual and on the machine are used to draw the user's attention to important safety warnings and useful information. Observe all warning, caution and danger messages.

	<b>WARNING</b> This warning symbol identifies operating/maintenance instructions or procedures which, if not observed, could result in injury or danger to the user.
	<b>ELECTRIC SHOCK HAZARD</b> This warning symbol identifies operating/maintenance instructions or procedures which, if not observed, could result in a risk of electric shock.
	<b>CAUTION</b> A Caution symbol is used to warn the user that failure to observe a certain operating or maintenance procedure, general rule or condition, could result in damage to the equipment.
	<b>NOTE</b> Notes draw the user's attention to key information on topics, applications or procedures.
	<b>TIPS</b> Tips draw the user's attention on the quickest or easiest way to do something.
	<b>WORKERS</b> This symbol indicates the minimum number of people recommended for installing the unit safely. More workers may be involved but we expressly advise against using fewer.

### NOTES

In case of doubts and/or any need for clarification during installation, contact the technical service department by e-mail, using the address: [assistenza@chinesport.it](mailto:assistenza@chinesport.it) and indicating the page and elements for which you need further details - if necessary attach a photo of the details.

**Chinesport is NOT responsible for anchoring to the walls, which must be done in a workmanlike manner, in compliance with current regulations, and entirely for the purchaser's account**

## 2 ASSEMBLY AND INSTALLATION

### **WARNING!**

Correct installation and compliance with the loadbearing capacities are fundamental for the safety of the patient and the professional.

### **WARNINGS!**

Before proceeding check the suitability of the environment and the structural adequacy of the available space

- The manufacturer cannot accept any responsibility for the installation done outside of their control.
- Technical skills and suitable equipment are required for a proper installation
- Unpack and handle the items with great care keeping them divided by type: if you notice any anomalies in the packaging or in the product do not continue the installation but promptly notify Chinesport SpA
- During the assembly use caution and always wear personal protective equipment
- Refer always to the installation schemes
- The manufacturer can not be responsible for the installation carried out of his control.
- Do not leave the device in use to minors.
- The equipment and accessories are intended only for the use established in the appropriate instruction manual supplied with the equipment

### **CAUTION**

The equipment must not be used:


- Without making the proper assembly and testing.
- Tampering with and / or changes on the original product.
- In improper configuration or when there is suspicion that may cause harm to the patient.
- By children and minors unattended.
- Under weathered.

In particular:


- The installation must be performed by a competent person who has consulted the manual and found the suitability of the premises.
- Assessing hazards resulting from the association of other equipment to the structure

## 2.1 LIST OF PARTS

<b>01325</b>	<b>[A]</b> Footboard [1 pc]
<b>01326</b>	<b>[B]</b> Gas springs [2 pcs]
<b>01391</b>	<b>[C]</b> Hardware bag [1 pc] – 85.00.00.0085
<b>02996</b>	<b>[D]</b> Handrail [2 pcs]
	<b>[E]</b> Ramps [2 pcs]
	<b>[H]</b> Spacers [4 pcs]
	<b>[N]</b> Spare parts bag [1 pc]

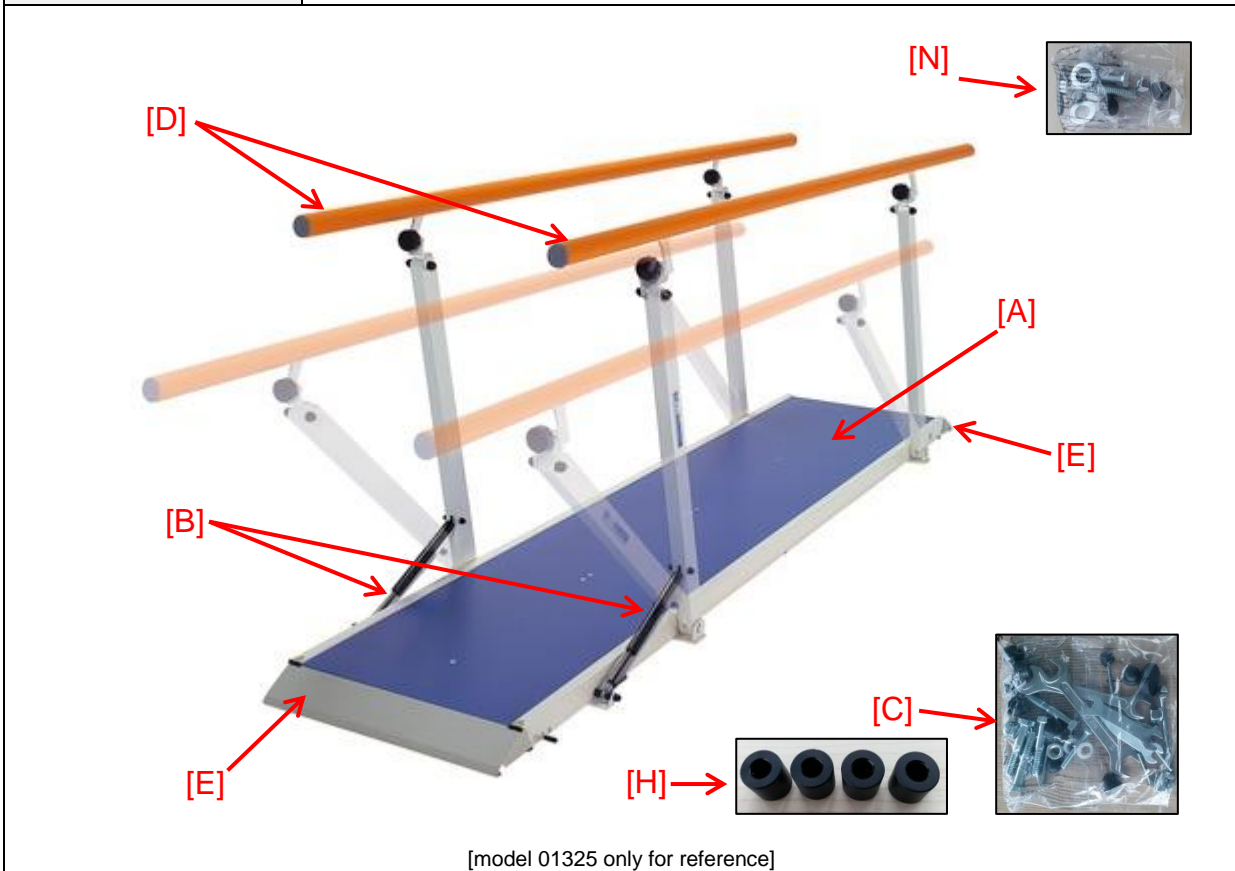<b>01327</b>	<ul style="list-style-type: none"> <li>[A] Footboard [1 pc]</li> <li>[B] Gas springs [2 pcs]</li> <li>[C] Hardware bag [1 pc] – 85.00.00.0085</li> <li>[D] Handrail [2 pcs]</li> <li>[E] Ramps [2 pcs]</li> <li>[F] Angle adjustment kit</li> <li>[G] Hardware bag for angle adjustment kit [1 pc]</li> <li>[H] Spacers [4 pcs]</li> <li>[I] Security sticks [7 pcs]</li> <li>[N] Spare parts bag [1 pcs]</li> <li>[O] Angle adjustment kit spare parts bag [1 pc]</li> </ul>
--------------	---
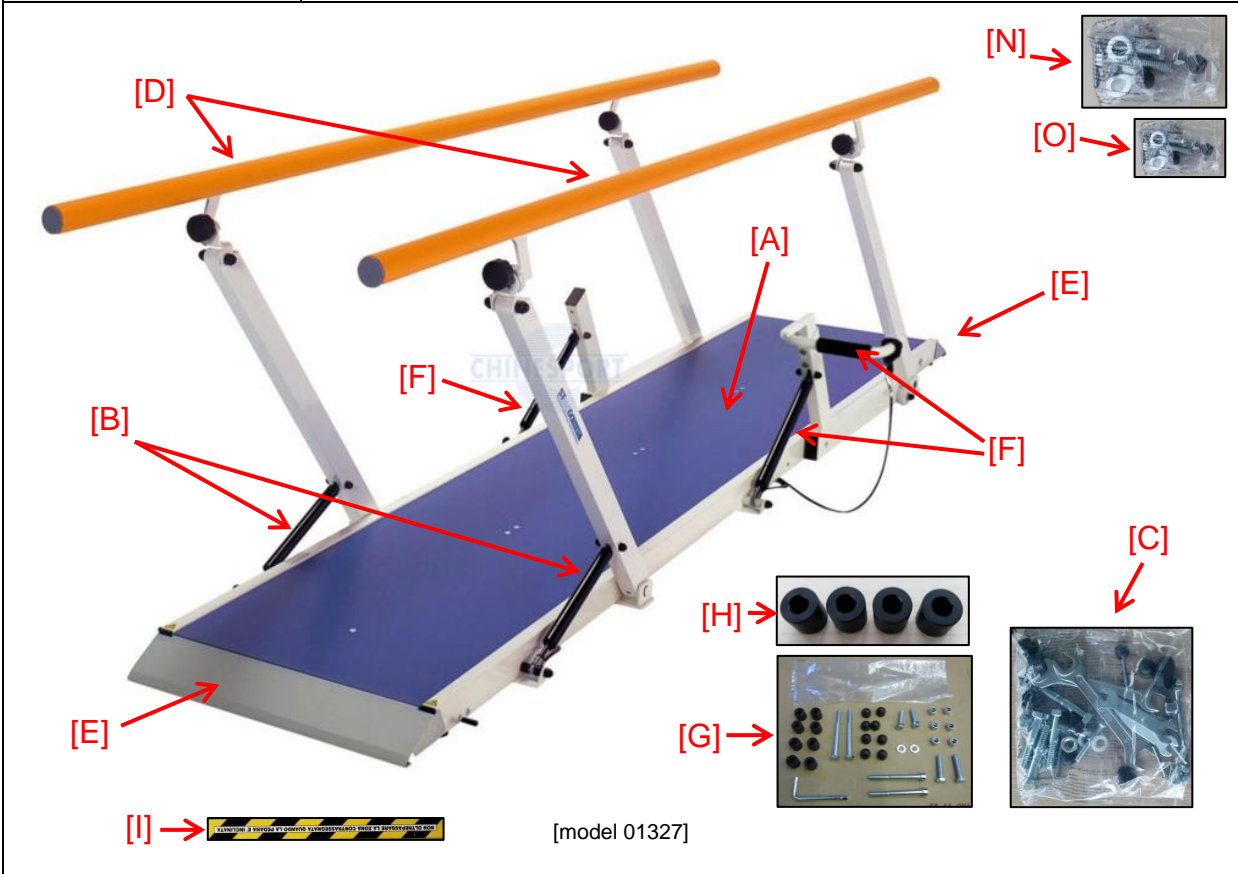
### 3 ASSEMBLY

#### 3.1 01325-01326-01391-02996

- 1 Remove the equipment, ramps and hardware bags from the packaging
- 2 Lay aside the spare parts bag [N] and storage it in a suitable place for possible future use
- 3 Open the hardware bag [C] and check that all the hardware match with the quantities and types as shown in the following picture:

	[C1]	17/19 wrench	2 pcs
	[C2]	13/10 wrench	2 pcs
	[C3]	Nut caps m8	6 pcs
	[C4]	Plastic washer	4 pcs
	[C5]	Nut caps m12	8 pcs
	[C6]	Screw M12x65	4 pcs
	[C7]	Screw M6x40	4 pcs
	[C8]	Self locking nut M12	4 pcs
	[C9]	Screw M8x50	2 pcs
	[C10]	Screw M8x90	2 pcs
	[C11]	Self locking nut M8	4 pcs
	[C12]	Nut M6	4 pcs

If the hardware doesn't match with the picture please contact our assistance service department

- 4 Put the walking platform on one side.
- 5 Install the ramps with the hardware provided (Fig. 1).

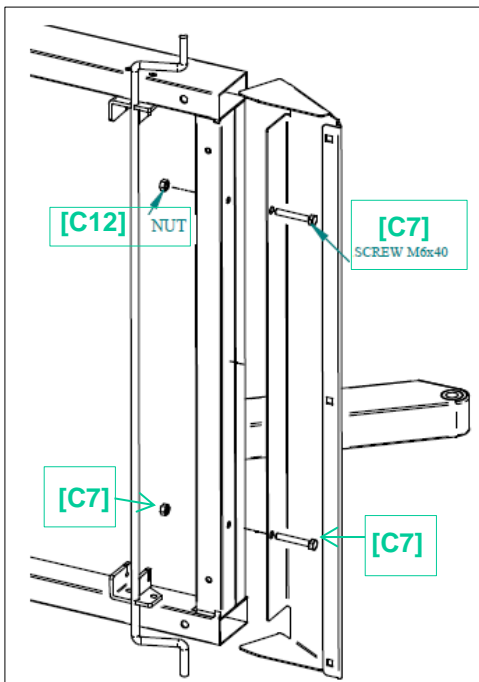
Fig. 1

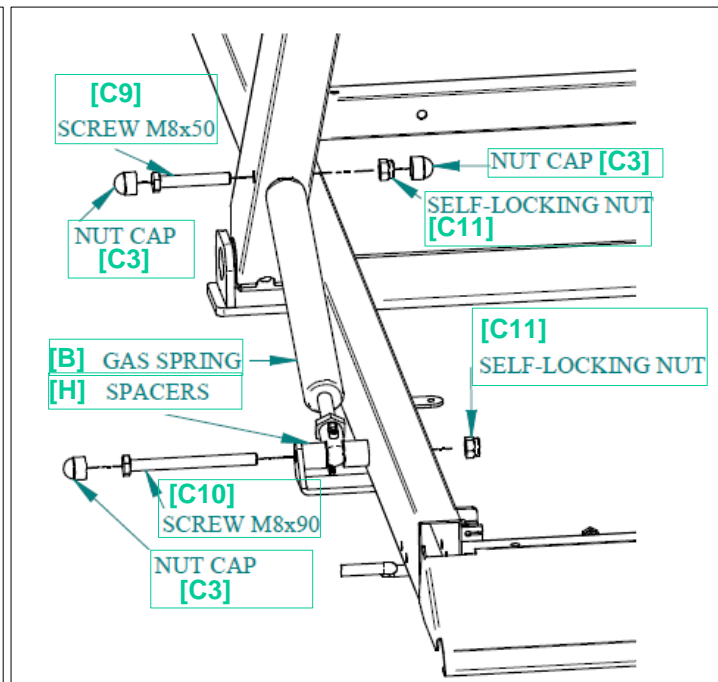
Fig. 2

- 6 Install the gas springs on the uprights from both sides: tighten the self-locking nut, leaving the spacers free to rotate (Fig. 2).
- 7 Place the platform in position for use.
- 8 Install the handrails with the hardware provided: tighten the self-locking nut thus allowing smooth height adjustment (Fig.3).

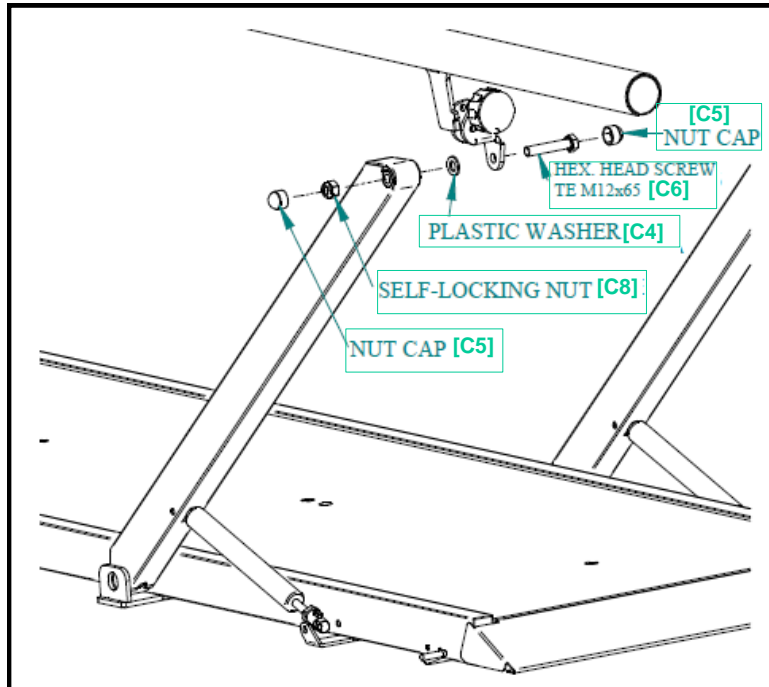
Fig. 3

### 3.2 01327

- 1 Remove the equipment, ramps and hardware bags from the packaging
- 2 Lay aside the spare parts bags [N] and [O] and storage them in a suitable place for possible future use
- 3 Open the hardware bag for the angle adjustment kit [G] and check that all the hardware match with the quantities and types as shown in the following picture

	[G1]	Screw M8x30	2 pcs
	[G2]	Screw M8x40	2 pcs
	[G3]	Nut caps m8	8 pcs
	[G4]	Screw M8x90	2 pcs
	[G5]	Screw M8x80	2 pcs
	[G6]	Nut M8	6 pcs
	[G7]	Dish washer	2 pcs
	[G8]	Hex wrench 6	1 pc
	[G9]	Spacer	8 pcs

If the hardware doesn't match with the picture please contact our assistance service department

- 4 Follow the steps from 3) to 8) of the previous chapter
- 5 Tilt kit mounting:
  - a. Install the gas springs and the tilting kit as shown in fig.4. Note that the screws must be free to rotate

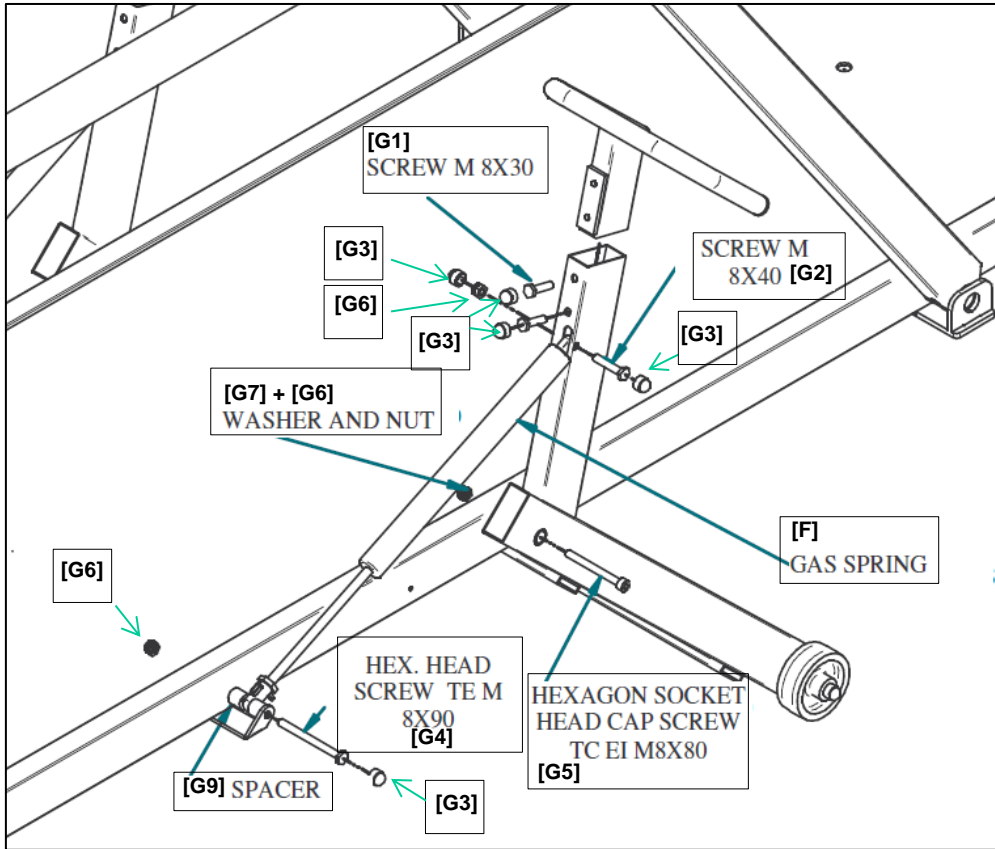
Fig.4

6 Fix the area boundary labels [ I ] onto the floor as shown in Fig. 5:

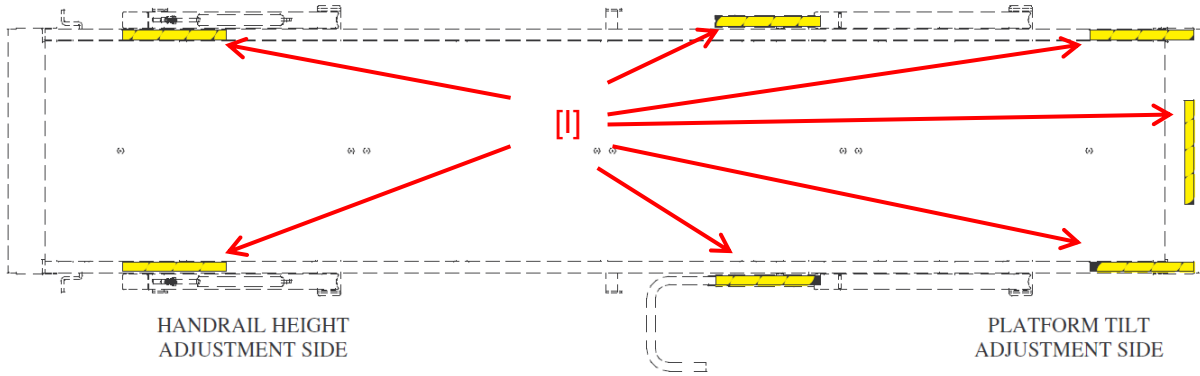
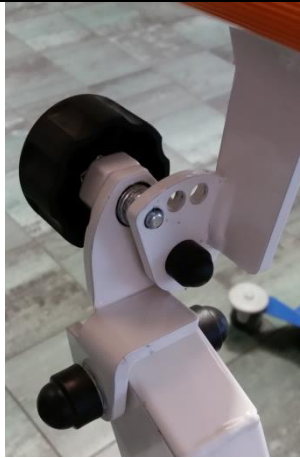
Fig. 5

### 3.3 WELL MOUNTED DETAILS

			
Handrail adjustment	Gas spring	Gas spring of tilting kit (only 01327)	Gas spring with spacers

If after mounted the parallel bar doesn't look like the details in the pictures please contact our assistance service department.

## 4 ADJUSTMENTS

### 4.1 HANDRAIL HEIGHT ADJUSTMENT

- 1 Grasp the handrails, release the position by pressing the pedal control (same effect achieved on right and left pedal)
- 2 Bring the handrails to the desired height and release the pedal to maintain the position.

**⚠️ WARNINGS! ⚠️**

During the handrail height adjustment, the springs alone will not ensure that the patient is secure on the platform

### 4.2 HANDRAIL WIDTH ADJUSTMENT

1. Unscrew one of the two knobs, on each handrail, until removing the pin from the three-hole blade.
2. Loosen the other knob (on the same handrail) and pull the handgrip to remove the pin from the three-hole blade; at this stage, while carrying out this operation with one hand, use the other hand to grab the handrail, which will now be free to move and can therefore be adjusted in width, by inserting the pin into the desired hole.
3. Tighten this knob and then screw back and tighten the other one.
4. The handrails, when tightened to the central hole, allow using the glider, code 01380

TIGHT KNOB

LOOSE KNOB

### 4.3 ANGLE ADJUSTMENT (ONLY 01327)

To adjust the angle of the platform use the lever as shown in the following figure

**⚠ATTENZIONE⚠**

Do not operate the adjustment handle with the patient load. The only gas springs do not guarantee the support of the structure beyond the limits set

## 5 MAINTENANCE

Only qualified personnel may carry out maintenance on the equipment. Technicians must be familiar with the model, use and maintenance of the equipment.  
Check the equipment whenever it has undergone a "rough treatment" (e.g.: fall, liquid spill, etc.) and whenever any doubts arise concerning its safety.

### 5.1 ROUTINE MAINTENANCE

Routine maintenance tasks are to be carried out at least once every 12 months.

FREQUENCY	TASK
Half-yearly	General cleaning of the equipment
	Visual and functional inspection of the equipment.
	Mechanical strength, tightening the screws, sliding and locking adjustments, degrade the decking

### 5.2 EMERGENCY MAINTENANCE

The emergency maintenance must be performed by specialized technicians or people with knowledge of the contents of this instruction booklet. For this type of interventions call for service of Chinesport SpA

### 5.3 PARTS LIST

Use only original spare parts. For the parts list, contact the offices CHINESPORT Spa.  
The parallel bars are provided with some spare parts bags as follows:

	[C3]	Nut caps m8	1 pc	
	[C4]	Plastic washer	2 pcs	
	[C5]	Nut caps m12	2 pcs	
	[C6]	Screw M12x65	1 pc	
	[C7]	Screw M6x40	1 pc	
	[C8]	Self locking nut M12	1 pc	
	[C9]	Screw M8x50	1 pc	
	[C10]	Screw M8x90	1 pc	
	[C11]	Self locking nut M8	1 pc	
	[C12]	Nut M6	2 pcs	
	Spare parts bag [N].			

	[G1]	Screw M8x30	1 pc
	[G2]	Screw M8x40	1 pc
	[G4]	Screw M8x90	1 pc
	[G5]	Screw M8x80	1 pc
	[G6]	Nut M8	1 pc
	[G7]	Dish washer	1 pc
	[G8]	Nut caps m8	1 pc
	[G9]	Spacer	1 pc
	[G10]	Washer	1 pc
	Spare parts bag [O]. Only for 01327		

## 6 CLEANING AND DISINFECTION

### 6.1 WASHING


Dust gently and in case of light stains pass the surface with a damp cloth or a neutral detergent, rinse with water without squeezing. Let the fabric dry avoiding direct exposure to the sunlight. The following instructions are recommended for small located and more obstinate stains (oil, fat, cosmetics, ink, coffee, liquor, chewing-gum etc.): dissolve immediately with clear ethylic alcohol diluted in water to 20%, dab and then treat with a neutral detergent solution. Rinse abundantly.

### 6.2 DISINFECTION

For the disinfection of surfaces such as chassis, sky, hand control, etc. we recommend the use of disinfectants such as AMUCHINA® 10% or equivalent disinfectant with sodium hypochlorite 0.1%

## 7 TECHNICAL FEATURES

FEATURE	01325	01326	01327	01391	02996
Weight [kg]	73	99	106	71	97
Overall dimensions [cm]	211x104x108	311x104x108	311x104x108	211x104x87	311x104x87
Handrail capacity (each) [kg]	135				
Inclined platform capacity [kg]	150				
Handrail height adjustment [cm]	74÷101			64÷80	
Handrail width adjustment [cm]	55÷71				

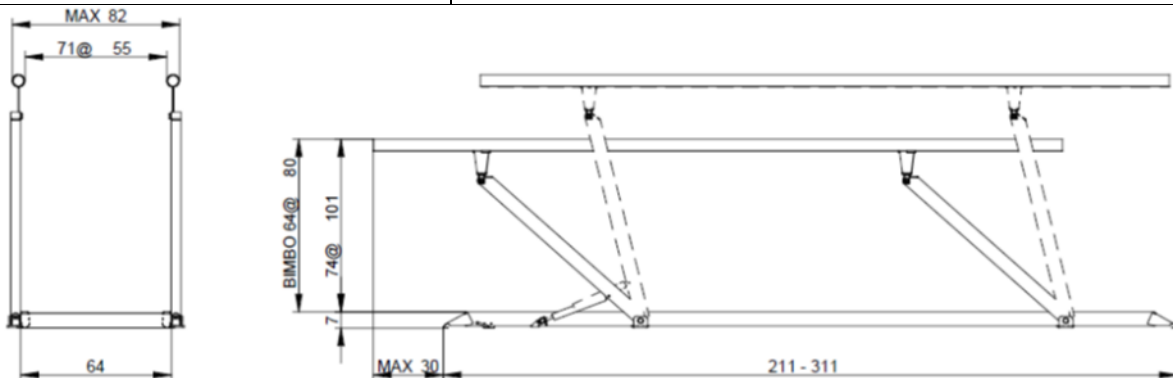## 8 PARALLEL BARS ACCESSORIES

A wide range of accessories can be provided with the parallels bars. Please visit [www.chinesport.com](http://www.chinesport.com) for the full list of accessories

## 9 WARRANTY

### GENERAL AFTER-SALES SERVICE CONDITIONS FOR CHINESPORT ITEMS

The warranty shall be valid for a period of 24 months from the date of delivery. The warranty does not cover consumable items, rechargeable batteries, and generally all the material subject to wear, breakdowns caused by knocks, falls, misuse or improper use, accidental events and transport damage. This warranty shall be automatically invalidated if the equipment is tampered with. Repair of the equipment will be carried out in relation to the breakdown stated. An estimate must be specifically requested. Payment is cash on delivery, unless other agreements are made. Any dispute will be dealt with solely by the Court of UDINE.

### WARRANTY REPAIRS

Repairs under warranty must be specifically requested. Repairs under warranty shall be carried out at our workshop and upon authorization, complete with the return authorization number. The return shipment for products sent in their original package will be made free of carriage charges. A fiscal document (bill of sale, purchase invoice or cash receipt) is required to make a warranty claim; the purchase date indicated must fall within the warranty time period. If the product is found to be in good working order without defect, the fault-finding service shall be charged to the customer.

### OUT-OF-WARRANTY REPAIRS

---

For out-of-warranty repairs carried out at our workshops upon authorization, transportation charges shall be paid by the customer. The warranty of the parts replaced during the repair operations is 12 months from the moment the equipment is collected. If the product is found to be in good working order without defect, the fault-finding service shall be charged to the customer.

#### REPAIRS MADE AT HOME

In the event repairs are made at the customer's home, a written request is required indicating the complete details of the person making the request, the type of machine and the breakdown. The cost per kilometer for on-site technician visits will be agreed upon in relation to the urgency of the customer. In the event the machine is under warranty, the customer will have to pay only the expenses incurred for the technician to reach the customer's house. Time is calculated from the moment the technician leaves our workshop until he/she returns. The time of his/her return will be estimated based on the time required to get there.

#### SALE OF SPARE PARTS AND CONSUMABLE ITEMS

The orders must be in writing and specify the customer, the item and the shipping method as well as the precise fiscal data of the customer. The order processing time can vary depending on the availability of the material.

Returned spare parts will not be accepted. Payment will be cash on delivery, unless other agreements are made.

### 10 MAINTENANCE RECORD SHEET

EQUIPMENT		DATE of first installation	SERIAL NUMBER		
DATE OF SERVICE	OPERATIONS CARRIED OUT	TECHNICIAN	SIGNATURE	NEXT VERIFY DATE	

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