# Invacare® H263 / H273 Lima

- EN** Shower and Toilet Commode  
User Manual
- DE** Dusch- und Toilettenrollstuhl  
Gebrauchsanweisung
- FR** Fauteuil/chaise de douche  
Mode d'emploi
- IT** Carrozzina da doccia e da toilette  
Istruzioni per l'uso
- ES** Silla de ducha y WC  
Manual de instrucciones
- NL** Douche- en toiletrolstoel  
Gebruiksaanwijzing
- DA** Bruse- og toiletkørestol  
Brugsanvisning
- FI** Suihku- ja wc-pyör Ätuoli  
Käyttöohje
- NO** Dusj- og toalettstol  
Brukerveiledning
- SV** Dusch- och toalettstol  
Bruksanvisning
- PT** Cadeira de banho  
Manual de instruções
- ET** Duši-/tualetitool  
Kasutusjuhend

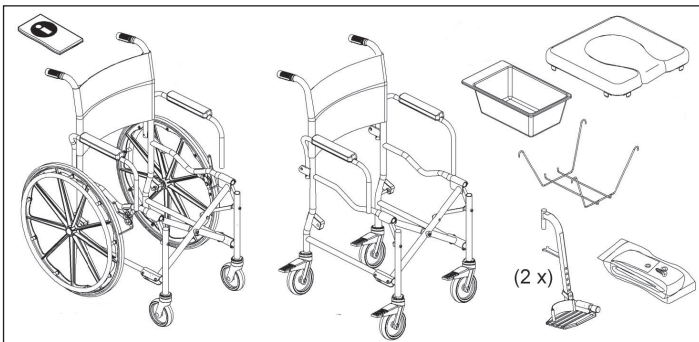
Fig. 1

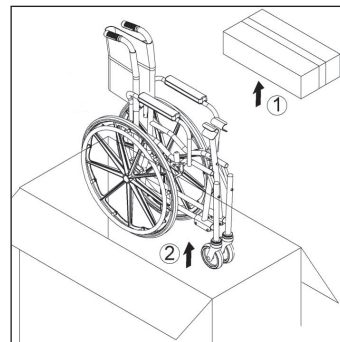
Fig. 2

Fig. 3

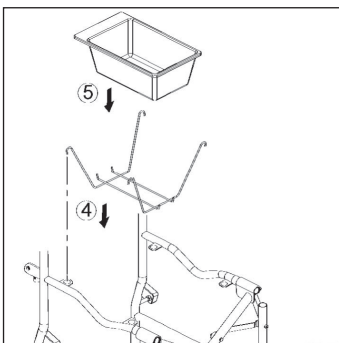
Fig. 4

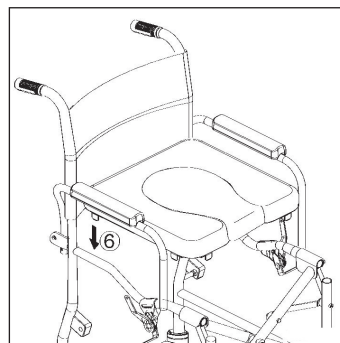
Fig. 5

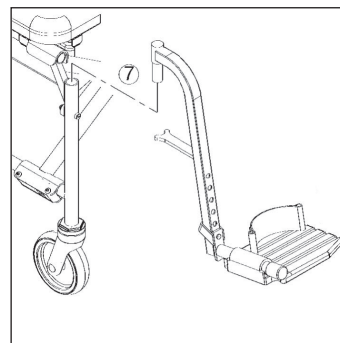
Fig. 6

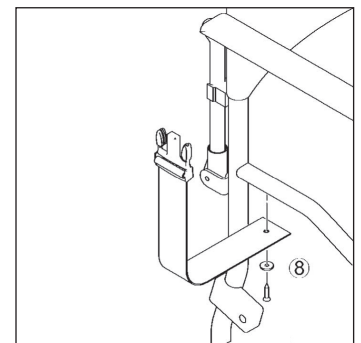
Fig. 7

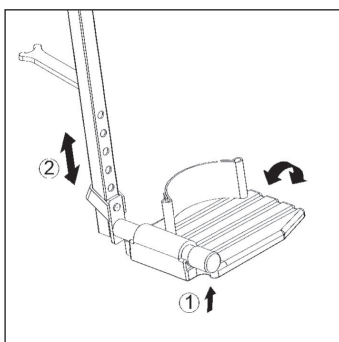
Fig. 8

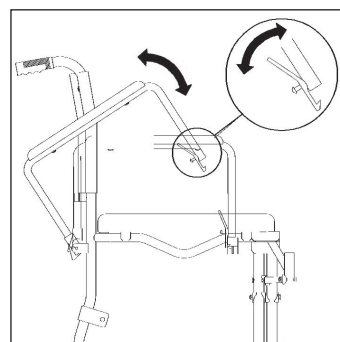
Fig. 9

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

Yes, you can.

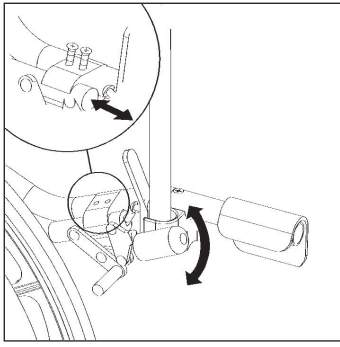
Fig. 10

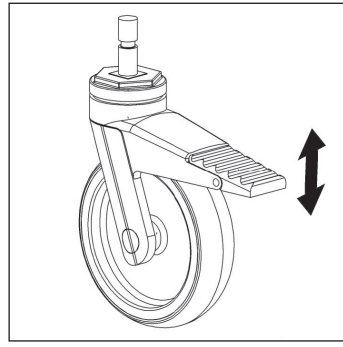
Fig. 11

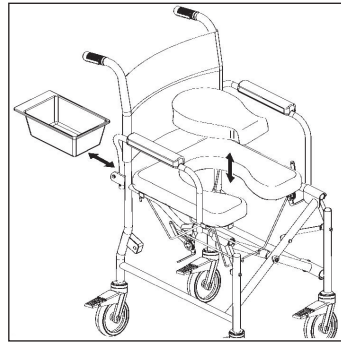
Fig. 12

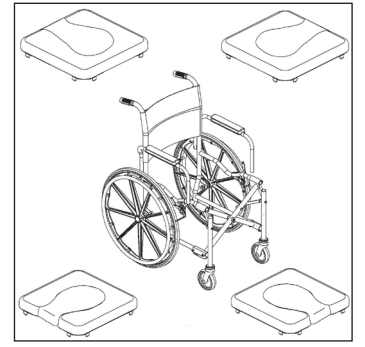
Fig. 13

## Intended Use

The commode is to be used solely as an aid for showering or bathing, going to the toilet or for transfer indoors on even surfaces. Any other use is prohibited.

### WARNING Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

Temperature for use: 10 - 40 °C

Max. user weight: 135 kg

## Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### WARNING Risk of injury!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.

## Scope of delivery

- Fig. 1.

## Assembly

### CAUTION Risk of pinching!

- Take care during assembly.

- Fig. 2 - 7.

## Usage

### WARNING Risk of falling!

- Apply the brakes in a parked position and for transfer.
- Do not stand on the footplates.

- Fig. 8 - 13.

## Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

- Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG002](http://www.nice.org.uk/CG002) and your local infection control policy.

- Clean by hand.
- Clean without scouring agent.
- Max. cleaning temperature 60 °C.

## Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

### WARNING

- Check product regularly for damage and to ensure that it is securely assembled.
- Regularly check firmly fitting of bolts and function of brakes.

## Re-Use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

## Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

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
## Technical Data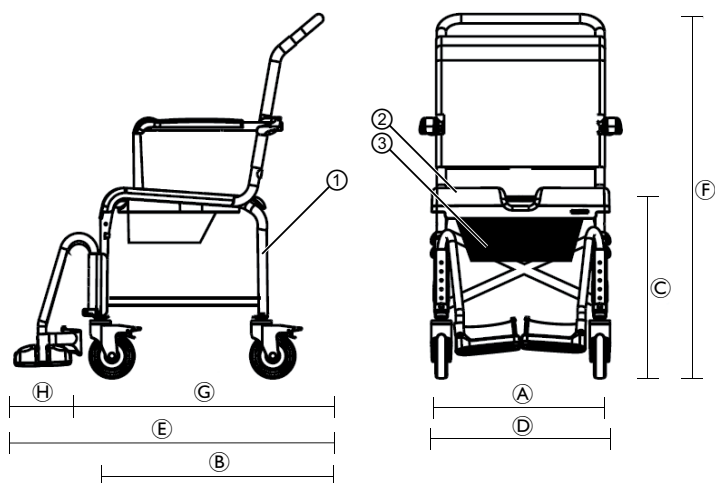

	H263	H273 (24" wheels)
<b>Order number</b>	1525732	1525733
<b>Dimensions</b>		
A	460 mm	460 mm
B	460 mm	460 mm
C	580 mm	580 mm
D	570 mm	680 mm
E	1050 mm	1120 mm
F	1000 mm	1000 mm
G	760 mm	760 mm
H	165 mm	165 mm
Total width folded	225 mm	335 mm
Total depth folded	1050 mm	1120 mm
Total height folded	1000 mm	1000 mm
Height to armrests	780 mm	780 mm
Height, footrests to seat	325 – 460 mm	325 – 460 mm
Height, ground to lower seat edge:		
- with holder for toilet pan	340 mm	340 mm
- without holder for toilet pan	535 mm	535 mm
Height, seat to armrest	210 mm	210 mm
Width between the armrests	475 mm	475 mm
Width, cut out in seat plate	190 mm	190 mm
Depth, cut out in seat plate	360 mm	360 mm
Ø Castors	5" (127 mm)	5" (127 mm)
Ø Drive wheels	-	24" (610 mm)
Width between the 24" wheels	-	550 mm
<b>More information</b>		
Weight	14.0 kg	18.5 kg
Maximum user weight	135 kg	135 kg
Colour	Dark blue - white	Dark blue - white
<b>Materials</b>		
①	Aluminium, corrosion-free	
②	PUR	
③	PP	

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##### Eastern Europe, Middle East, Russia & CIS:

European Distributor Organisation:  
Invacare, Kleiststraße 49  
D-32457 Porta Westfalica  
Tel. (49) (0)57 31 754 540  
Fax. (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

AQUATEC Operations GmbH  
Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0  
Fax: (49) (0)75 62 7 00 66  
kontakt@invacare.com / www.invacare.de

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Invacare AB, Fagerstagatan 9,
Algol-Trehab Oy