Aquatec® Sorrento

en	Shower stool / Shower chair User Manual	1
de	Duschhocker / Duschstuhl Gebrauchsanweisung	4
fr	Tabouret de douche / Chaise de douche Manuel d'utilisation	7
it	Sgabello per la doccia / Sedia da doccia Manuale d'uso	9
es	Taburete de ducha / Silla de ducha Manual del usuario	11
pt	Banco de duche / Cadeira para duche Manual de utilização	13
nl	Douchekruk / Douchestoel Gebruiksaanwijzing	15
da	Badetaburet / Brusestol Brugsanvisning	17
no	Dusjkrakk / Dusjstol Bruksanvisning	19
sv	Duschpall / Duschstol Bruksanvisning	21
fi	Suihkujakkara Käyttöohje	23

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

General

Images

Fig. 1

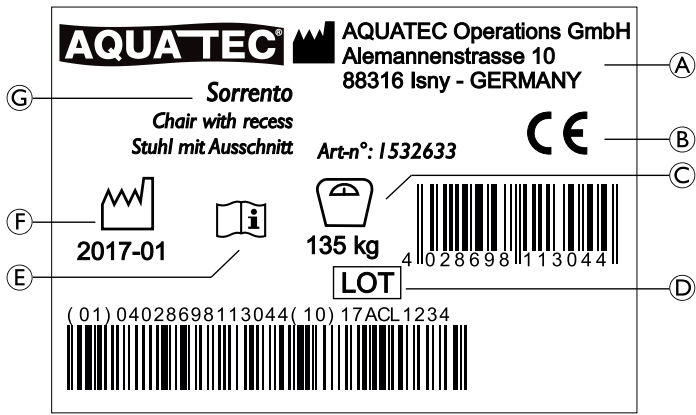
Fig. 2

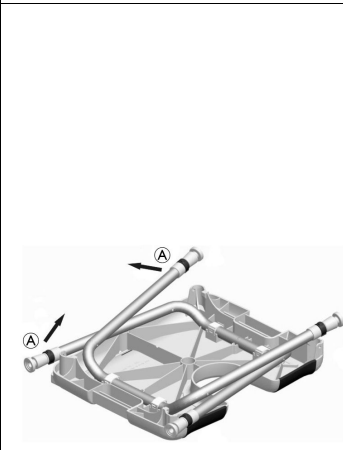
Fig. 3

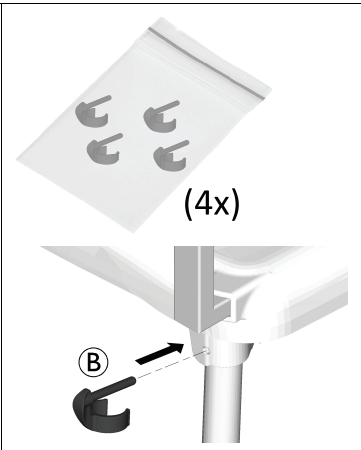
Fig. 4

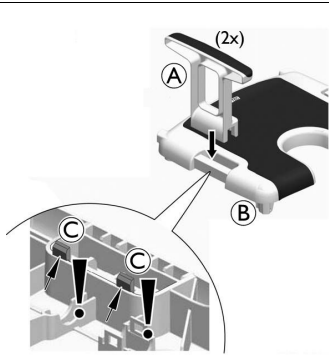
Fig. 5

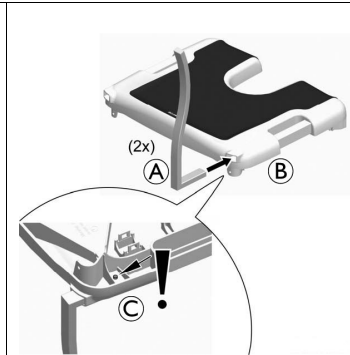
Fig. 6

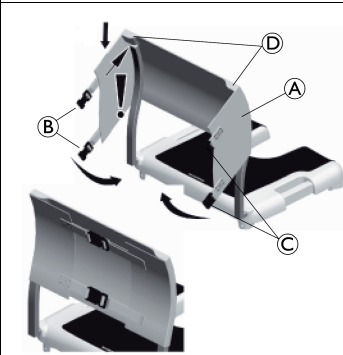
Fig. 7

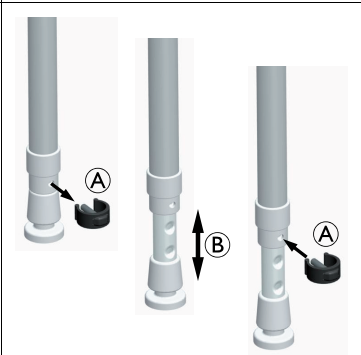
Fig. 8

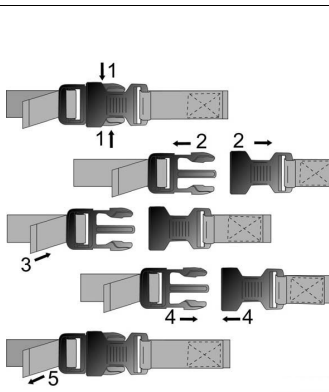
Fig. 9

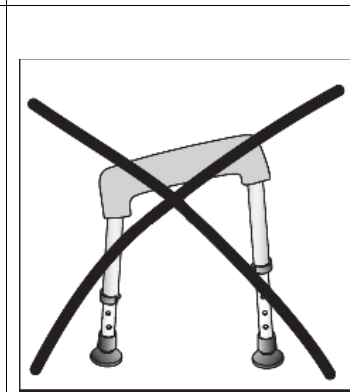
Fig. 10

Fig. 11

Important note:

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

Intended use

The product is intended exclusively as an aid for showering and for use in the bathroom by adult people who have:

- Difficulty standing or keeping their balance
- Weak legs or lower limb disability
- Medical conditions that impair their balance

Any other use is prohibited.

Operating temperature: 10 - 40 °C

Max. user weight: 135 kg

Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Scope of delivery

See Fig. 1

Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

Safety information

WARNING!

Risk of injury!

- Do not use if defective.
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.

WARNING!

Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use chair as a support for standing.
- Do not lift the product at the armrest for re-positioning or transfer while seated.
- Do not pull/push the chair to displace it under load.

WARNING!

Risk of burning your skin!

- The product components can heat up when exposed to external sources of heat.
- Do not expose the product to strong sunlight before use.

WARNING!

Risk of injury!

- Only use spare parts and accessories intended for this product.

CAUTION!

Risk of entrapment!

- Do not grab chair at the underside of the seat plate in the area of the armrest.

Symbols on the product

The identification label is located on the bottom of the seat. Refer to Fig. 2.

Ⓐ	Manufacturer address
Ⓑ	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.
Ⓒ	Maximum user weight
Ⓓ	Lot number
Ⓔ	Read user manual
Ⓕ	Date of manufacture
Ⓖ	Product designation

Assembly

CAUTION!

Risk of entrapment!

- The product must be assembled by a person capable to do so.

CAUTION!

Risk of entrapment!

- Be careful during assembly.

1. Fold out legs Ⓐ on both sides, see Fig 3.
2. Secure legs with safety clips Ⓑ on all four sides, see Fig 4.

CAUTION!

Risk of injury!

- Ensure that all safety clips are correctly assembled.

Adjusting the seat height

The height of the seat can be adjusted between 460 and 610 mm in six steps of 25 mm.

1. Remove safety clips **A** from all four legs, see Fig 8.
2. Adjust the height **B** of the legs, see Fig 8.
3. Re-insert safety clips **A** on all four legs see Fig 8.

WARNING!

Risk of falling!

- Ensure that all legs are adjusted to the same height, see Fig 10.

Fitting the armrests (option)

1. Drop armrests **A** into seat plate **B** on both sides, see Fig 5.

WARNING!

Risk of injury!

- Ensure that both catches **C** in each armrest audibly engage on the underside of the seat plate, see Fig 5.

CAUTION!

Risk of entrapment!

- Do not grab the chair at the underside of the seat at the armrest fixation.
- Do not put your fingers in open holes at the armrest and below the seat.

Fitting the backrest (option)

1. Push backrest frame **A** into the rear openings of seat plate **B** on both sides, see Fig 6.

WARNING!

Risk of injury!

- Ensure that both catches **C** of the backrest frame audibly engage on the underside of the seat plate, see Fig 6.

2. Pull down backrest cover **A** over both sides of the backrest frame, see Fig 7.

WARNING!

Risk of injury!

- Ensure that the ends of the backrest frame are properly located in the recesses **D** on the right and left of the backrest cover, see Fig 7.

3. Close the backrest bands by pushing latches **C** into buckles **B**, see Fig 7.

WARNING!

Risk of injury!

- Make sure that all buckles are securely closed.

4. Tighten the backrest bands, see Fig 9.

Tighten or slacken the bands to adjust the firmness of the backrest cover.

The chair is disassembled in reverse order to assembly.

Maintenance and periodic inspection

The product is maintenance-free, provided that the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled. Make sure that all screws are securely tightened.
- Regularly clean and disinfect the product.

Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

1. Clean the product regularly.
2. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 85 °C for 3 minutes in an autoclave.

Cleaning the backrest cover (option)

1. Remove the backrest cover.
2. Wash the cover in a washing machine at max. 60 °C using a mild detergent.

IMPORTANT!

Risk of damage!

- Washing at higher temperatures can cause shrinkage.
- Do not dry covers in a tumble dryer.

Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (→ Cleaning and disinfection, page 4), check the product for damages and ensure a proper mounting.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

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– Varmista, että selkänöjan rungon päädyt ovat asianmukaisesti aukoissa © selkänöjan suojuksen oikealla ja vasemmalla puolella, katso kuva 7.
Invacare Poirier SAS
Invacare Austria GmbH
Invacare AB