## Aquatec® Dot

en	Shower stool User Manual .....	3
de	Duschhocker Gebrauchsanweisung .....	4
fr	Tabouret de douche Manuel d'utilisation .....	6
it	Sgabello da doccia Manuale d'uso .....	8
es	Taburete de ducha Manual del usuario .....	9
pt	Banco de duche Manual de utilização .....	11
nl	Douchekrak Gebruiksaanwijzing .....	13
sv	Duschstol Bruksanvisning .....	14
no	Dusjkrakk Bruksanvisning .....	16
da	Badestol Brugsanvisning .....	18
fi	Suihkujakkara Käyttöohje .....	19
sk	Sprchový stolček Návod na obsluhu .....	21
lt	Dužo kėdutė Naudojimosi instrukcija .....	22

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

**AQUATEC®**

**Yes, you can.®**

Images

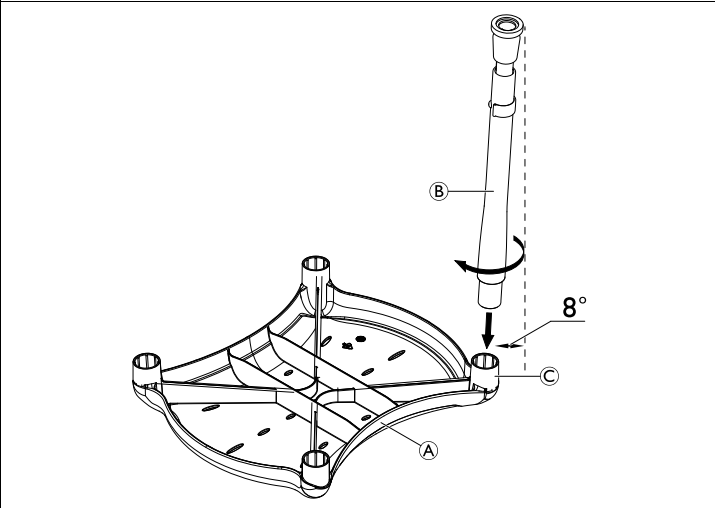
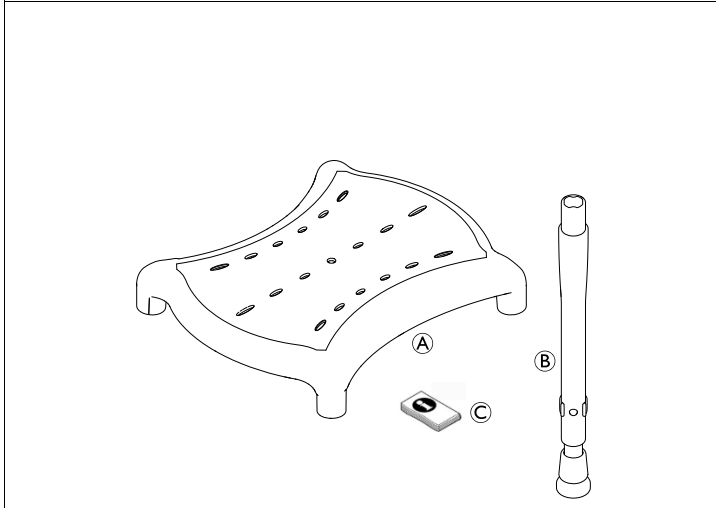
Fig. 1

Fig. 2

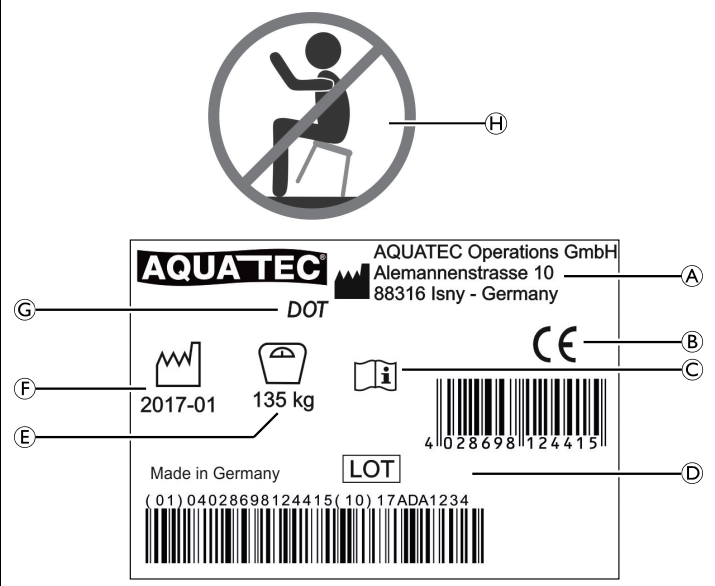
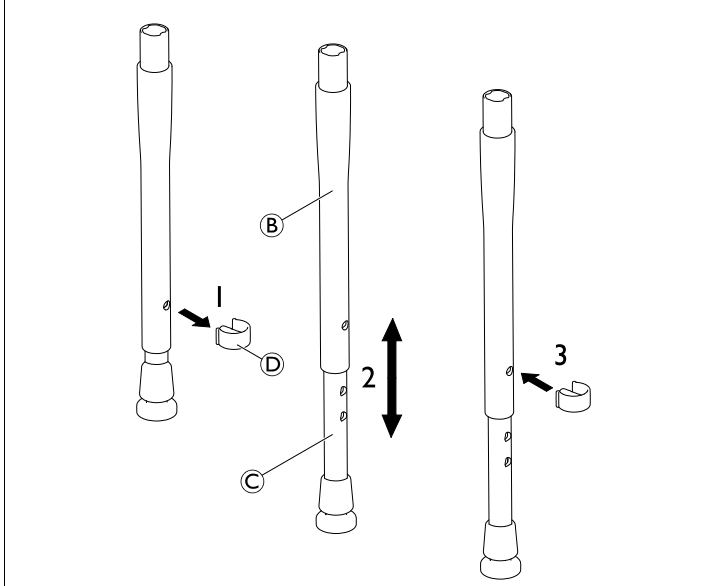
Fig. 3

Fig. 4

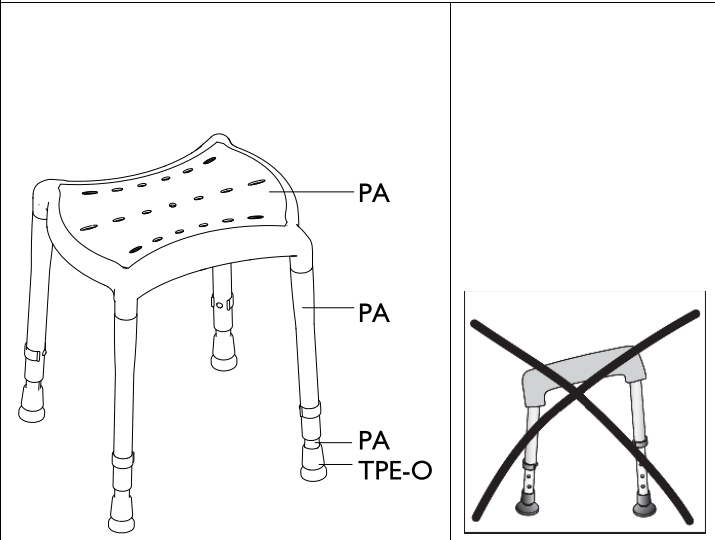
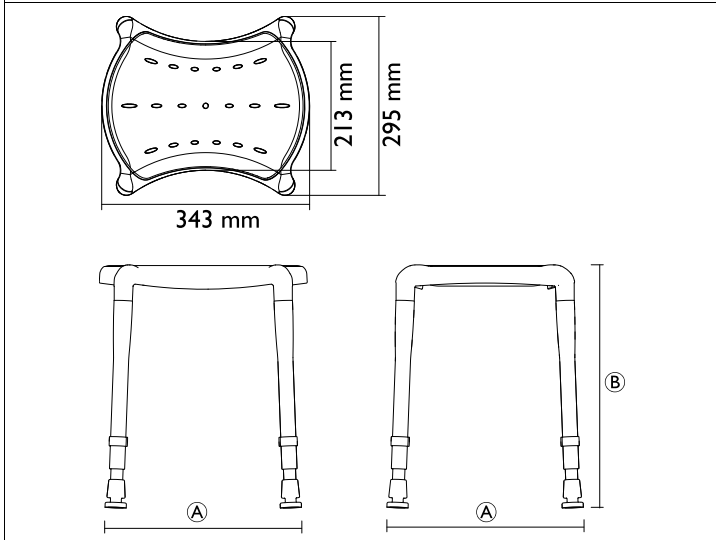
Fig. 5

Fig. 6

Fig. 7

## General

### Important note:

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

### Intended use

The shower stool is to be used solely as an aid for sitting in the bathroom or in the shower for people with sufficient torso stability. Any other use is prohibited. The user must be able to sit down and get up from the shower stool safely without help.

Operating temperature: 10 – 40 °C

### Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

### Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

### Scope of delivery

The following components are included in the scope of delivery (Fig. 1):

Ⓐ	Seat plate
Ⓑ	Height adjustable legs (4x) composed of two into each other moveable tubes, clips and rubber tips
Ⓒ	User manual

### Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### Safety information

#### **WARNING!** **Risk of falling!**

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use for transfer from the side or partial load.
- Do not sit on the product if not all four legs have full contact to the ground.
- Do not tilt.

#### **WARNING!** **Risk of injury!**

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.

### Labels and symbols on the product (Fig. 4)

Identification label	
Ⓐ	Manufacturer
Ⓑ	This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.
Ⓒ	Read user manual
Ⓓ	Lot number
Ⓔ	Maximum user weight
Ⓕ	Date of manufacture
Ⓖ	Product designation

#### Warning label

Ⓗ	Do not tilt the shower chair during usage
---	---

The labels are attached to the underside of the seat plate.

## Setup

### Assembling the product (Fig. 2)

1. Place the seat plate Ⓐ with the seat surface down on a flat table.

#### **IMPORTANT!** **Risk of damage to the product**

The leg supports Ⓒ are slightly angled outwards (8°) and could be damaged if pressing in the legs Ⓑ vertically from above.

- When attaching the legs to the supports, make sure to hold them slightly angled and simultaneously carry out a rotary movement.

2. Attach the leg Ⓑ into the seat plate by pushing it into support Ⓒ.
3. Repeat the same procedure for the other three legs.

#### **WARNING!** **Risk of injury**

- Make sure that all four legs are completely inserted in the supports.

## Adjusting the seat height (Fig. 3)

1. Remove the safety clip ① from the leg.
2. Adjust the desired height by moving the adjustment tube ② into or out of the leg ③.
3. Re-attach the safety clips through the desired holes of the leg and the adjustment tube.
4. Repeat the same procedure for the other three legs.

### WARNING!

#### Risk of falling!

- Make sure that all legs are adjusted to the same height (Fig. 7).
- Make sure that the safety clips are fully engaged in the holes of all legs.

## Maintenance

### Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

### Cleaning and disinfection

### IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 60 °C.

The product can be cleaned and disinfected using commercially available cleaning agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

### Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly.

### Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

## Technical data

### Dimensions and weight (Fig. 5)

Ⓐ	365 mm, in highest position
Ⓑ	400 – 550 mm, in steps of 25 mm
Product weight	1.65 kg
Max. user weight	135 kg

## Materials (Fig. 6)

All product components are corrosion free and do not contain natural rubber latex.

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1. Fjern sikkerhedsclipsen © fra benet.
Invacare Limited
Aquatec Operations GmbH