VERMEIREN

Bobby

INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI

EN

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Instructions to the specialist dealer

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: A, 2015-02

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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© N.V. Vermeiren N.V. 2015

Contents

Preface	2
1 Product description	3
1.1 Intended Use.....	3
1.2 Technical specifications	4
1.3 Components.....	5
1.4 Accessories.....	5
1.5 Explanation of symbols	5
1.6 Safety rules	6
2 Use	6
2.1 Carrying the transferchair.....	7
2.2 Unfolding the transferchair	7
2.3 Fold up the transferchair	7
2.4 Mounting or removing of the footrests	8
2.5 Folding / Unfolding the backrest.....	8
2.6 Operating the brakes.....	9
2.7 Transfer in and out the transferchair	9
2.8 Correct position in the transferchair.....	10
2.9 Riding the transferchair	10
2.10 Moving on slopes	10
2.11 Negotiating steps or curbs	10
2.12 Transport in the car.....	11
3 Installation and adjustment	12
3.1 Tools	12
3.2 Manner of delivery	12
3.3 Adjust the brakes.....	13
3.4 Adjusting of the footrest.....	13
4 Maintenance	14

Preface

First of all we want to thank you for putting your trust in us by selecting one of our transferchairs.

The Vermeiren transferchairs are the result of many years of research and experience. During the development, special attention was given to the ease of use and the serviceability of the transferchair.

The expected lifetime of your transferchair is strongly influenced by the care and maintenance of the transferchair.

This manual will help you get acquainted with the operation of your transferchair.

Following the user instructions and the maintenance instructions are an essential part of the warranty.

This manual reflects the latest product developments. Vermeiren has the right to introduce changes without the obligation to adapt or replace previously delivered models.

For any further questions, please consult your specialist dealer.

1 Product description

1.1 *Intended Use*

The transferchair is intended for people with walking difficulties.

The transferchair is designed to transport 1 person.

The transferchair is suited for indoor and outdoor use.

The transferchair shall be pushed by an attendant.

The different types of fittings and accessories, and the modular construction allow full use for aged persons.

When providing for individual requirements:

- body size and weight (max. 115 kg)
- physical and psychological condition
- residential circumstances
- environment

should be taken into consideration.

Your transferchair should only be used on surfaces where all four wheels are touching the ground and where there is sufficient contact to propel the wheels equally.

You should practice for use on uneven surfaces (cobblestones, etc.), slopes, curves and to get past obstacles (curbs, etc.).

The transferchair should not be used as a ladder, nor is it a transport for heavy or hot objects.

When used on mats, carpeted floors or loose floor coverings, the floor covering can get damaged.

Use only Vermeiren approved accessories.

The manufacturer is not liable for damage caused by the lack of or improper service or as a result of not following instructions from this manual.

Compliance with the user and maintenance instructions are an essential part of the guarantee conditions.

1.2 Technical specifications

Technical terms below are valid for the transferchair in standard settings.

If other footrests / armrests or other accessories are used, the tabulated values will change.

Make	Vermeiren	
Address	Vermeirenplein 1/15, B-2920 Kalmthout	
Type	Transferchair	
Model	Bobby	
Maximum occupant mass	115 kg	
Description	Dimensions	
Effective seat width	420 mm	480 mm
Overall width (depends on the seat width)	590 mm	640 mm
Folded width	270 mm	
Description	Dimensions Min.	Dimensions Max.
Overall length with footrest	950 mm	
Folded length without footrest	740 mm	
Total height	940 mm	
Folded height	750 mm	
Total mass	11,00 kg	11,55 kg
Mass of heaviest part	9,40 kg	9,95 kg
Masses of parts that can be dismantled or removed	Footrests: 1,60 kg	
Static stability downhill	10°	
Static stability uphill	10°	
Static stability sideways	10°	
Obstacle climbing	60 mm	
Seat plane angle	5°	
Effective seat depth	420 mm	
Seat surface height at front edge	500 mm	
Backrest angle	5°	
Backrest height	380 mm	
Distance between footplate and seat	370 mm	500 mm
Angle between seat and footplate	-1°	19°
Angle between seat and leg of footrest	114°	
Distance between armpad and seat	220 mm	
Front location of armrest structure	315 mm	
Handrim diameter	Not applicable	
Horizontal location of axle (deflection)	21 mm	
Minimum turning radius	1500 mm	
Diameter Rear wheels	12,5"	
Tyre pressure, rear (driving) wheels (for air tyres)	Not applicable	
Diameter steering wheels	150 mm	
Tyre pressure, steering wheels (for air tyres)	Not applicable	
Storage and use temperature	+ 5 °C	+ 41 °C
Storage and use humidity	30%	70%
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / °		


Table 1: Technical specifications Bobby

The transferchair complies to the requirements set up in:

ISO 7176-8: Requirements and test methods for static, impact and fatigue strengths.

ISO 7176-16: Resistance to ignition of upholstered parts

1.3 Components

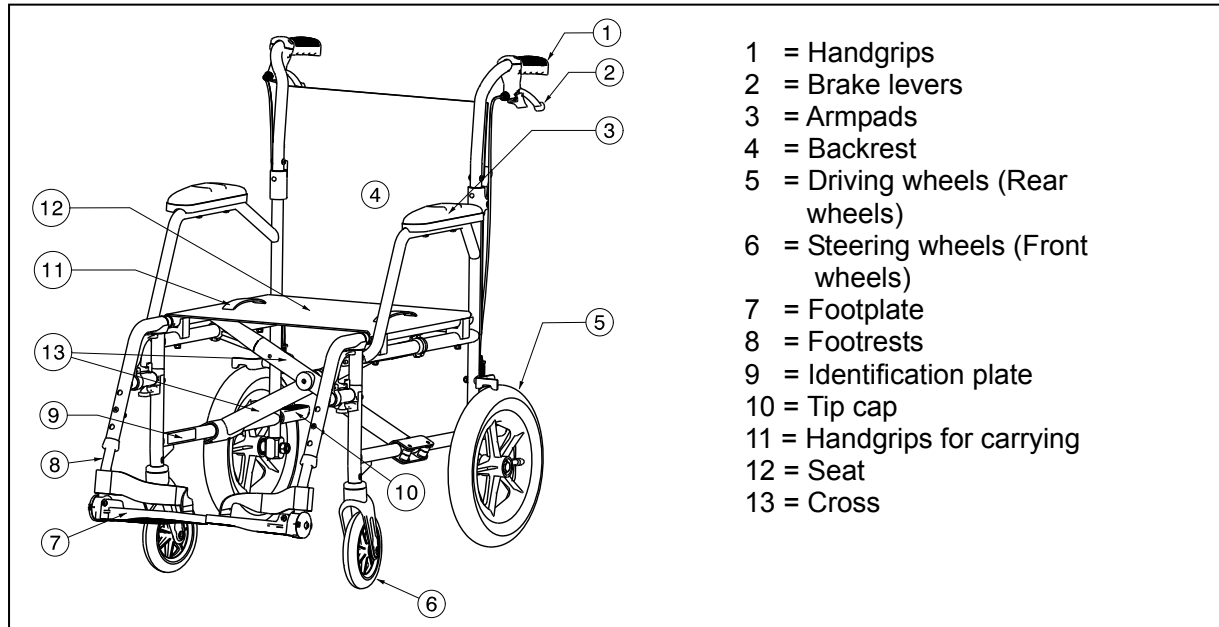1.4 Accessories

The following accessories are available for the Bobby:

- Anterior pelvic belt (B20) for mounting over the backrest tube

1.5 Explanation of symbols

Maximum mass

Indoor and outdoor use

Maximum Safe Slope

CE conformity

Type designation

Not intended to be used as a seat in a motor vehicle

1.6 Safety rules

- ⚠ To prevent injury and/or damage to your transferchair, make sure that no objects and/or body parts are caught in the driving wheels.
- ⚠ The parking brakes should be applied before getting into and out of the transferchair.
- ⚠ When getting into and out of the transferchair, do not stand on the footplates. These should be folded up beforehand, or the footrests should be swung completely outwards out of the way.
- ⚠ Investigate the effects of shifting the centre of gravity on the behavior of the transferchair, for example on up or down gradients, on laterally sloping ground, or when overcoming obstacles.
- ⚠ If you want to pick up something (lying in front of, on the side, or to the rear of the transferchair), you should not lean too far out to avoid tipping over.
- ⚠ When moving through doors, arches, etc. ensure that there is enough room at the sides so that you do not get your hands or arms caught or crushed and that there is no damage to the transferchair.
- ⚠ Only use your transferchair according to regulations. For example, avoid uncontrolled rolling against obstacles (steps, curbs, doorframes, etc) or dropping down from ledges. The manufacturer cannot assume liability for damage caused by overloading, collision or other improper use.
- ⚠ Stairs may only be negotiated when aided by another person. If furnishings like drive-on, ramps or lifts are available, use them.
- ⚠ When moving on the public highway, you are subject to the rules of the road.
- ⚠ When driving your transferchair, you should not be under the influence of alcohol or medicine as in the case of driving other vehicles. This also applies to indoor driving.
- ⚠ When travelling outdoors, adapt your driving to weather and traffic conditions.
- ⚠ When transporting the transferchair, never pick it up by grasping movable parts (footrests, etc.)
- ⚠ To be better visible when driving in the dark, wear the brightest possible clothing or clothes with reflectors, and check that the reflectors mounted on the sides and rear of the transferchair are clearly visible.
- ⚠ Never exceed the maximum load of 115 kg.

2 Use


This chapter describes the everyday use. **These instructions are for the user and the specialist dealer.**

The transferchair is delivered fully assembled by your specialist dealer. The instructions intended for the specialist dealer on how to set up the transferchair are given in § 3.


2.1 Carrying the transferchair

The best way to carry the transferchair is to make use of the wheels and roll the transferchair.

If this is not possible, firmly grasp the frame on the front and the grips. Do not use the foot- or armrests or the wheels to grasp the transferchair.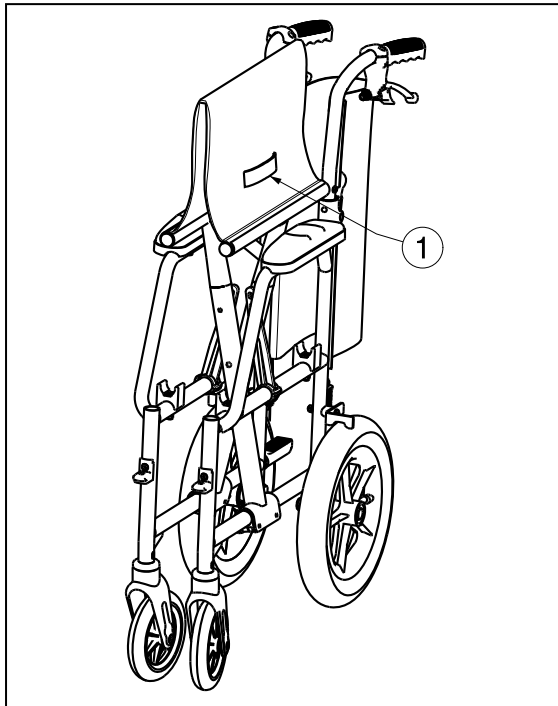

It is also possible to carrying the transferchair with the handgrips ①. First remove the footrests, fold up the transferchair. Now you can grasp the handgrips ① on both side and carry the transferchair to the desired place.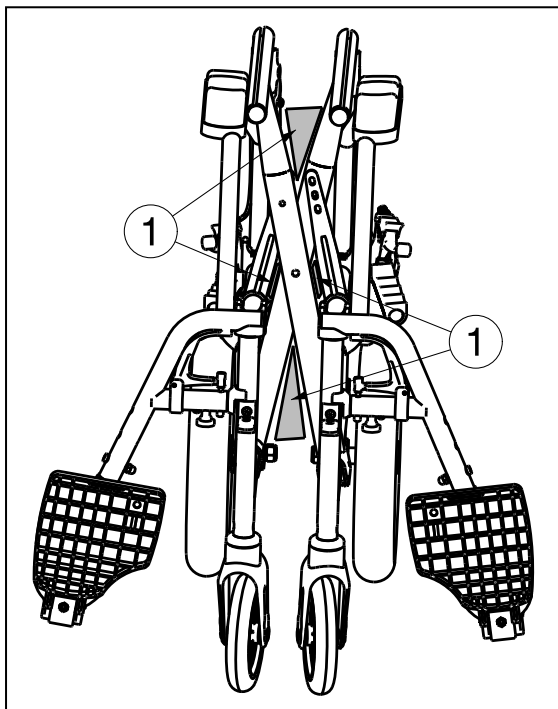

2.2 Unfolding the transferchair

⚠ CAUTION: Risk of clamping – Keep fingers away from moving parts of the transferchair.

1. Position yourself behind the transferchair.
2. Unfold the backrest (see § 2.5).
3. Use the handgrips to open the transferchair as much as possible.
4. Position yourself at the front of the transferchair.
5. Push both seating tubes down till they are fixed in their position.


2.3 Fold up the transferchair

⚠ CAUTION: Chance of pinching – Do not place fingers between the components (entrapment zone ①: cross) of the transferchair.

1. Fold the footplates or remove the footrests (see § 2.4).
2. Fold the backrest (see § 2.5).
3. Take the seat on the front side and backside and pull it up.

2.4 Mounting or removing of the footrests

- ⚠ **WARNING:** Risk of injury – Check that the footrests are securely locked in place before use.
- ⚠ **CAUTION:** Risk of clamping – Keep objects and people away from the footrests swing range.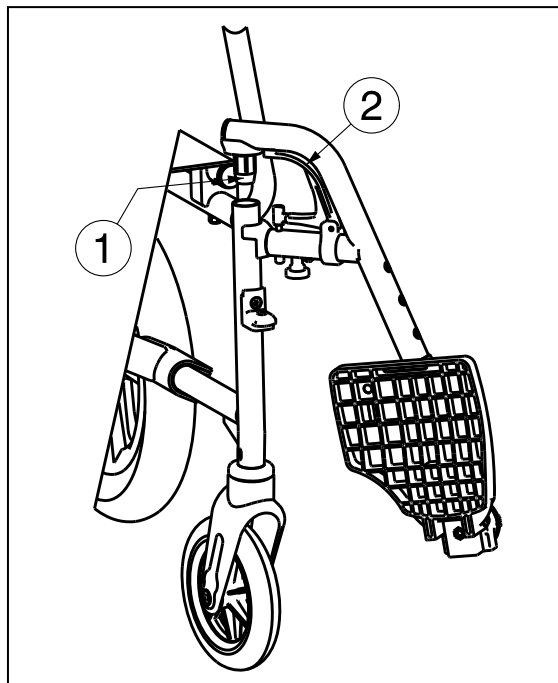

The mounting of the footrest is done as follows:


1. Hold the footrest sideways at the outside of the wheelchairs frame and mount the tube hood ① into the frame.
2. Swing the footrest inwards till it clicks in position.
3. Swing the foot plate downwards.

To take off the footrests:

1. Pull handle ②.
2. Swing the footrest to the outside of the wheelchair till it comes loose from the guidance.
3. Pull the footrest from tube hood ①.

2.5 Folding / Unfolding the backrest

- ⚠ **WARNING:** Risk of injury – Check that the backrest is securely locked in place before use.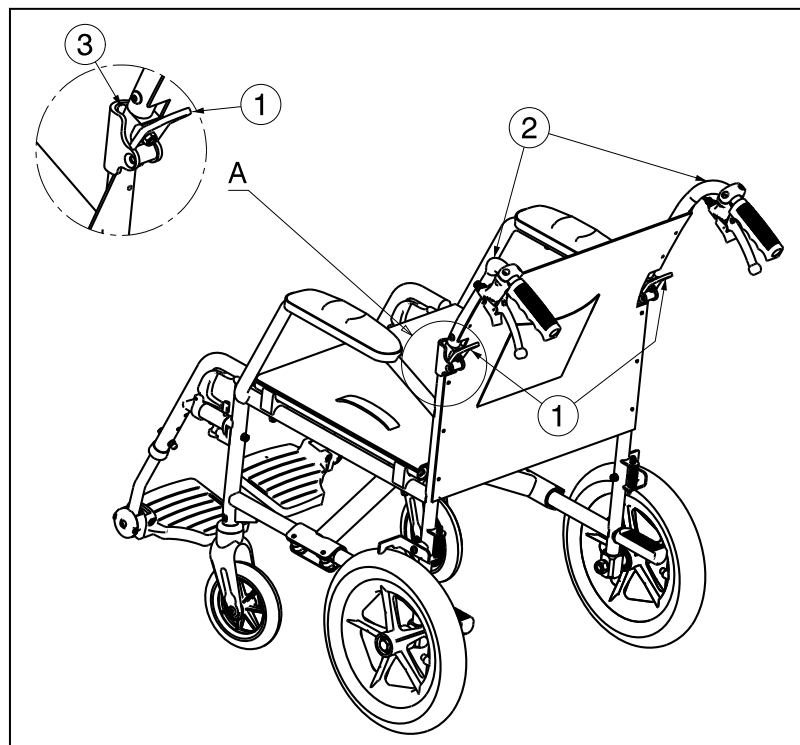

The folding of the backrest is done as follows:

1. Pull on the levers ①, until the pin ③ comes out the hole.
2. Pull the upper backrest tubes ② downwards until the backrest is fully folded.


To unfold the backrest:

1. Pull the backrest tubes ② upwards until the backrest is firmly secured.
2. Check that the backrest is locked in place.

2.6 Operating the brakes

- ⚠ **WARNING:** The brakes are not used to slow down the transferchair during movements – Use the brakes only to prevent the transferchair from unintended movements.
- ⚠ **WARNING:** Good operation of the brakes is influenced by wear and contamination of the tires (water, oil, mud, ...) – Check the condition of the tires before each use.
- ⚠ **WARNING:** The brakes are adjustable and can wear – Check the operation of the brakes before each use.

The brakes can only be activated by the attendant.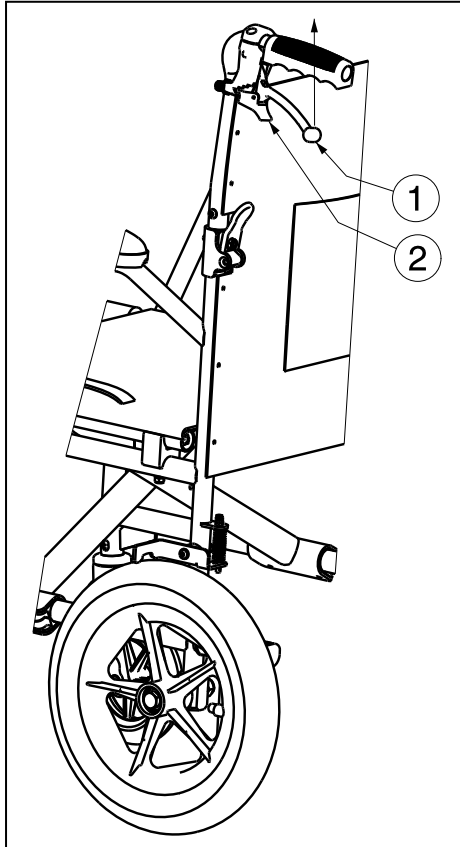

To engage the parking brakes:

1. Pull the brake levers ① upwards.
2. Push on part ② to block the brakes.


- ⚠ **CAUTION:** Risk of unintended movement – Make sure the transferchair is on a flat horizontal surface before releasing the brakes.

To disengage the parking brakes:

1. First release the part ②.
2. Release the brake levers ①.

2.7 Transfer in and out the transferchair

- ⚠ **CAUTION:** In case you cannot perform the transfer in a safe manner, ask someone to assist you.
- ⚠ **CAUTION:** Risk of tipping over of the transferchair – Do not stand on the footplates.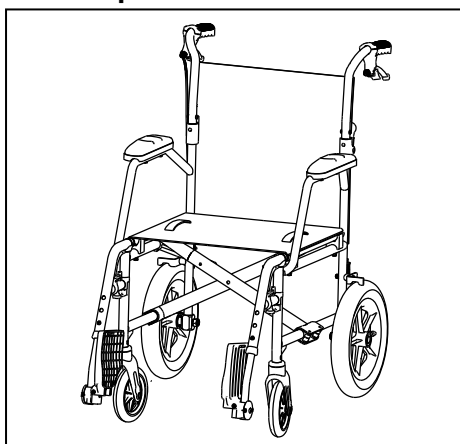

1. Position the transferchair as close as possible to the chair, couch or bed to/from you wish to transfer.
2. Check both brakes (parking brakes) from the transferchair are activated.
3. Fold the footplates upwards to prevent standing on them.
4. Transfer to/from the transferchair.

2.8 Correct position in the transferchair

Some recommendations for a comfortable use of the transferchair:


- Position your backside as close as possible to the backrest.
- Make sure your upper legs are horizontal – If needed adjust the length of the footrests (see § 3.4).

2.9 Riding the transferchair

- ⚠ **WARNING: Risk of clamping – Be careful passing through restricted passages (e.g. doors).**
- ⚠ **WARNING: Risk of burns – Be careful when driving in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.**
- ⚠ **WARNING: Risk of injury – Keep your arms always in the transferchair.**
 1. Release the brakes.
 2. The attendant grasp the handgrips.
 3. The attendant push the transferchair to the desired direction.

2.10 Moving on slopes

- ⚠ **WARNING: Control your speed – Moving on slopes as slow as possible.**
- ⚠ **WARNING: Consider the capacities of your attendant – If your attendant does not have enough force to control the transferchair, he must put on the brakes.**
- ⚠ **WARNING: Risk of tipping over – Lean forward to move your centre of gravity forward. To improve a better stability.**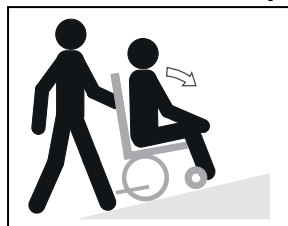


1. If available on the transferchair, wear the safety belt.
2. Do not attempt moving on too high slopes. The maximum slope angles (upwards and downwards) are mentioned in table 1.
3. The attendant shall push you on the slope.
4. Lean forward to move your centre of gravity forward.

2.11 Negotiating steps or curbs

2.11.1 Getting down steps or curbs

Riding down low curbs can be done moving forwards. Make sure that the footplates do not touch the ground.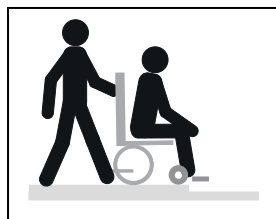

1. Ask the attendant to tip the transferchair slightly backwards.
2. Get past the curbs while moving on the rear wheels.
3. Put the transferchair back on the four wheels.


Higher curbs are best done backwards.

1. Turn the transferchair to have the rear wheels facing the curbs.
2. Lean forward to move your centre of gravity forward.
3. Move the transferchair close to the curbs.
4. The attendant pull you softly from the curbs in a controlled manner.

2.11.2 Moving up steps or curbs

Moving up steps or curbs with attendant as follows: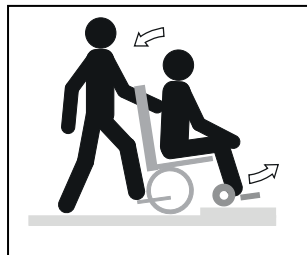

1. Prevent the footplates from touching the curbs.
2. Ask the attendant to tip the transferchair backwards, just enough to move the front wheels over the curbs.
3. Lean backwards to move your centre of gravity above the rear wheels.
4. Place the front wheels on the curbs.
5. Roll rear wheels of the transferchair over the curb.

Higher curbs are negotiated backwards:

1. Turn the transferchair to have the rear wheels facing the curbs.
2. Lean backwards and move your centre of gravity above the rear wheels.
3. Ask the attendant to pull the transferchair on the curbs.
4. Take back the normal position in the transferchair.

2.11.3 Taking of stairs

Never take a moving staircase.

Taking of stairs while you staying in the transferchair shall be according following rules:

⚠ WARNING: Risk of tipping over – Taking of stairs shall always with 2 attendants.

1. Remove the footrests.
2. One attendant tip the transferchair slightly backwards.
3. The second attendant take the front of the frame.
4. Stay calm, avoid sudden movements and keep your arms inside the transferchair.
5. Take the steps on the rear wheels of the transferchair.
6. Mounting the footrests back after taking the stair.

2.12 Transport in the car

⚠ DANGER: Risk of injury – The transferchair is not suited for use as a seat in a motor vehicle.

⚠ WARNING: Risk of injury – See that the transferchair is attached properly. So you can avoid injury from the passengers during collision or sudden braking.

⚠ WARNING: Risk of injury – Use for attaching the transferchair and passenger NEVER the same seatbelt.

Never use your transferchair as a seat in an automobile or other vehicle. The transferchair shall be marked with following symbol.

To transport the transferchair in the car use following steps:

1. Remove footrests and accessories.
2. Store footrests and accessories safely.
3. Fold the transferchair.
4. Place the transferchair in the luggage place.
5. If the transferchair and the passenger compartment is NOT separated, attach the frame of the transferchair securely to the vehicle. You can use the available safety belts in the vehicle.

3 Installation and adjustment

The instructions in this chapter are for the specialist dealer.

The Vermeiren transferchairs Bobby has been designed for short trip and easily transfer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.

⚠ WARNING: Variation of allowed adjustments can still change the stability of your transferchair (tilt back or sideways).

3.1 Tools

To set up the transferchair the following tools are needed.

- Wrench set n° 10
- Allen key set n° 4

3.2 Manner of delivery

The Vermeiren Bobby shall be delivered with:


- 1 frame with armrests, rear and front wheels
- 1 pair footrests
- Handgrips + brake levers
- Back and seat cushion
- Tools
- Manual
- Accessories (optional)

Before use check if everything is included and that no products are damaged (example by transport, ...).

3.3 Adjust the brakes

⚠ WARNING: Risk of injury – Brakes may only be adjusted by your specialist dealer.

Adjust the brakes according following rules: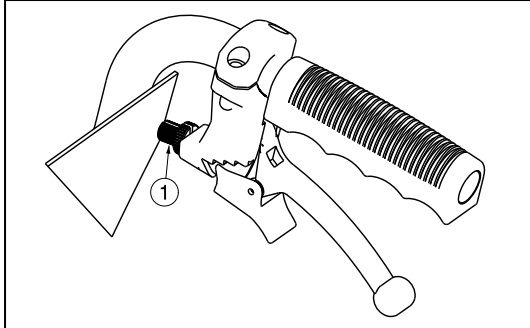


1. Turn on part ① to adjust the tension of the brakes.
2. Check working of the brakes.
3. If necessary repeat turning part ① until the brakes are adjusted well.

3.4 Adjusting of the footrest

3.4.1 Length of the footrest


⚠ CAUTION: Risk of damage – Avoid that the footplate make any contact with the ground. Keep a minimum distance from 60 mm above the ground.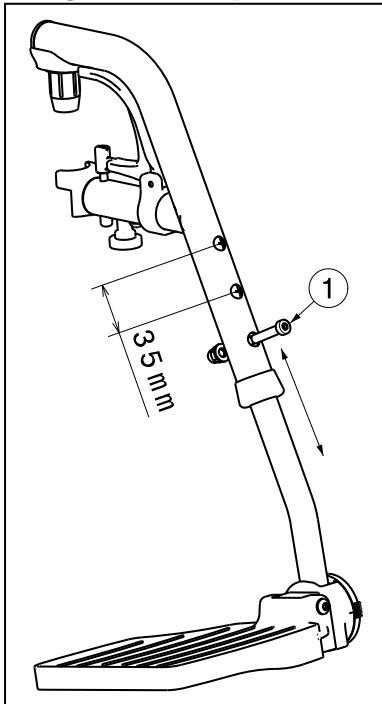

Adjust the length of the footrest as follow (3 positions possible: steps 35 mm → holes outside tube, 6 positions possible: steps 25 mm → inner tube):

1. Remove the screw ①.
2. Adjust the length of the footrest to a comfortable length.
3. Tighten the screw ① properly.

3.4.2 Adjusting angle footplates

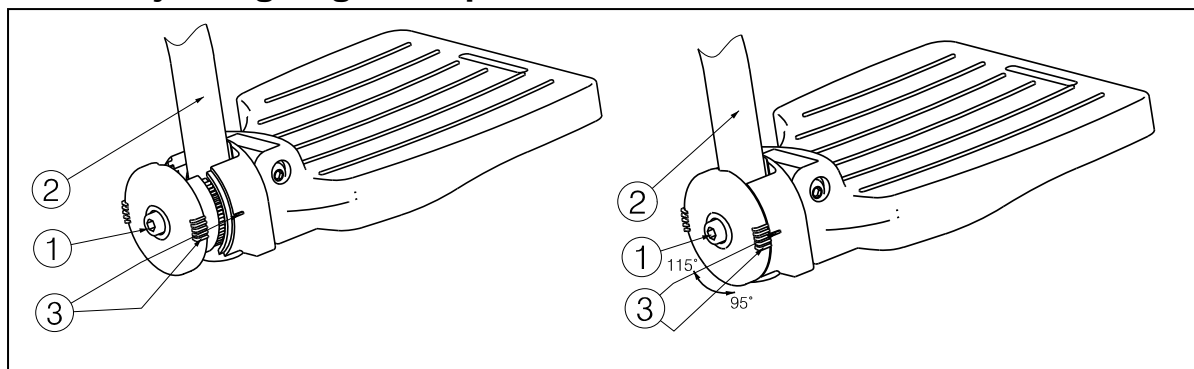Adjust the angle of the footplates as follow:

1. Loosen bolt ①.
2. Adjust tube ② in the desired angle. The angle indication (95°-100°-105°-110°-115°) is according the dashes ③.
3. Retighten the bolt ①.

4 Maintenance

For the maintenance manual of the transferchairs refer to the Vermeiren website:
www.vermeiren.be.

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CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles, beds and other products: 2 years (batteries 6 months) and multipoosition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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Vermeiren Suisse S.A.