ENGLISH

ITALIANO

**AP10003 - ELBA 3**  
**AP20002 - ISCHIA 2**

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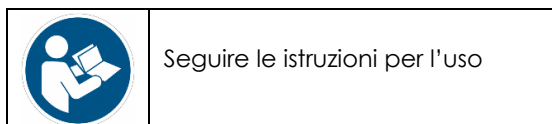
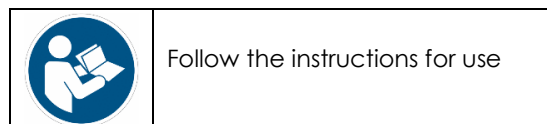
This manual contains information for practical, correct and safe use of the device and is intended for reading by specialised personnel and the user of the product. It is recommended that you read the entire manual carefully before using the product.

If you have any doubts or need clarification, please contact your dealer who will be able to advise you correctly.

The importance of reading and understanding the user manual is highlighted on the product by the following symbol:

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 <https://www.chinesport.com>

 [chinesport@chinesport.it](mailto:chinesport@chinesport.it)

The company produces in accordance with the quality standard UNI EN ISO 13485:2016

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Manual propulsion chair / chair intended for moving patients in hospital environments / nursing homes / retirement homes.

Steel structure painted with scratch-resistant epoxy powders and corrosion. Footrest adjustable up to floor level for safe and easy access.

Seat and backrest tilting system with gas spring to allow a safe rest position.

Fully foldable ergonomic armrests to facilitate lateral patient transfer.

Ergonomic seat and backrest upholstered in washable, antibacterial, fireproof class 1 IM leatherette.

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Chair / wheelchair designed for transferring patients in an hospital environment

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The use of the device for purposes other than those defined in this manual is prohibited. The manufacturer declines any responsibility for damage to persons or property resulting from improper use of the device or in any case other than that provided for in this manual.

The manufacturer reserves the right to make changes to the product and the manual without prior notice in order to improve its characteristics and performance.

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Medical staff – non-professional users

It is up to the specialist to judge the physical fitness of the patient for whom the product is intended to be used.  
Use under operator supervision is always recommended.

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Clinical/hospital environment. Use on flat, dry, stable and obstacle-free surfaces. Not for outdoor use (with the exception of model AP20002).

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Store in a dry place at room temperature. Avoid excessive pressure and contact with discolouring materials. Avoid excessive exposure to direct sunlight.

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The manufacturer declares that

- the device is not a measuring instrument
- the device is not intended for clinical investigation
- the device is not sterile and is not for single use
- for a correct functioning and for the safety of the user, it is necessary that the ordinary maintenance operations are carried out as described in the relevant paragraph.
- the device cannot be used for purposes other than those stated in this manual

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Always refer to this manual for proper use of the device.

- Store the device in an environment that complies with the labels on the packaging and the specifications in this manual.
- The useful life of the product is 10 years in accordance with the correct execution of the ordinary maintenance operations provided for in this manual. It is strictly forbidden to use the device beyond its stated useful life. At the end of its useful life, it is possible to proceed as described in the relevant paragraph
- The manufacturer shall not be liable, to the fullest extent permitted by applicable law, for any direct or indirect, special, incidental or consequential damages caused by:
  - o Wrong use of the device
  - o Improper use of the device and outside of its intended use
  - o Use of the device beyond the stated service life in this manual
  - o Using the device in environments not covered by this manual
  - o Use with unsuitable patients
  - o The distraction of operators or incorrect application of commands and adjustments
  - o Use without checking the status of the device as described in the relevant paragraph
  - o Incorrect maintenance or lack of maintenance

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- o Use with parts or accessories that are not compatible or not approved by the manufacturer
- o Incorrect disposal or disposal is other than as described in this manual.

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|  |                                 |
|--|---------------------------------|
|   | Follow the instructions for use |
|   | CE certified                    |
|   | Dispose of properly             |
|   | Manufacturer                    |
|   | Production date                 |
|  | Indoor use only                 |

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|   |                                 |
|---|---------------------------------|
|  | Fragile                         |
|  | Recyclable                      |
|  | Keep dry                        |
|  | Keep this side at the top       |
|  | Do not use sharp blades to open |

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The product is delivered in a suitable cardboard packaging so that it can be received intact and functional. To open the package and remove its contents, pay attention to the warnings and symbols on the package itself.

Dispose of the packaging and the resulting material in an appropriate manner and follow the instructions in the packaging and in this manual.

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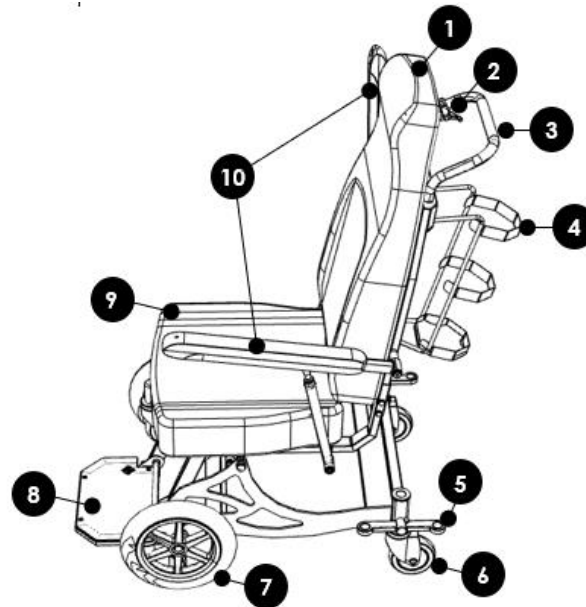
The package contains

- Instruction manual
- Product already assembled
- Packaging material to be disposed of

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- 1 Backrest
- 2 Seat inclination control
- 3 Push handle
- 4 Canister holder
- 5 Unidirectional and brake control
- 6 Rear wheel
- 7 Fixed front wheel (mod. AP20002)  
Front swivel wheel (mod. AP10003)
- 8 Retractable footrest
- 9 Seat
- 10 Reclining armrests

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The device is delivered assembled and ready to use.

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Before proceeding with the first use, it is necessary to read this instruction manual.

Make sure that the product corresponds to what is described in this manual and that the contents of the package coincide with what is described in the relevant paragraph.

Before using the product for the first time:

- Make sure there are no components of the device inside the packaging (screws, nuts, handwheels...)
- Make sure there are no obvious signs of damage or tampering
- Check that there are no obvious signs of damage or tampering
- Check the rotation and sliding of the wheels
- Check the correct operation and effectiveness of the parking brakes
- Check the operation of the inclination adjustment control
- Check that there are no obvious signs of loosening of the screws

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Before each use make sure that:

The product does not show any obvious signs of tampering or damage  
The product has been sanitised in the parts in contact with the user  
The physical and clinical condition of the user has been assessed and found to be appropriate for the use of the device.  
The operating environment is in accordance with the provisions of this instruction manual.  
There are no particular hazards in the areas around the device (shelves, obstacles, flammable materials, etc. ...)  
There are no oil stains on the floor caused by the gas spring

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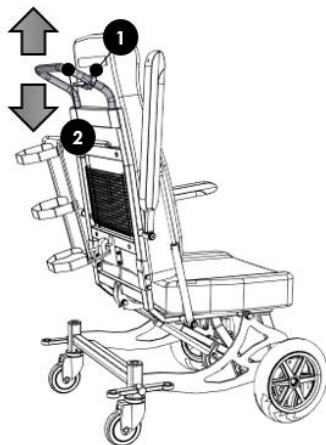
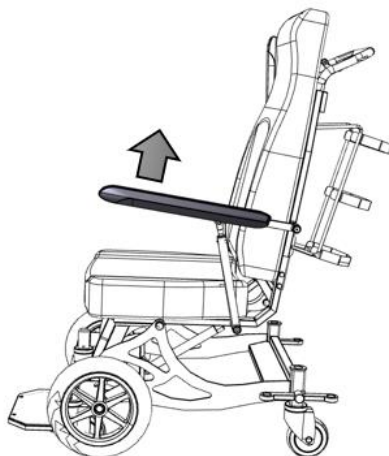
The device has a gas spring inclination adjustment system. To tilt the chair, operate the control (1) and, using the push handle (2), adjust the inclination by pushing upwards or downwards.

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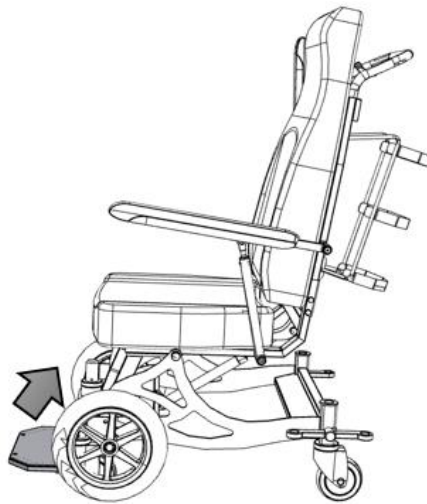
The armrests can be raised to facilitate side access to the chair. Simply lift the armrests with your hand, making sure they do not fall over the patient.

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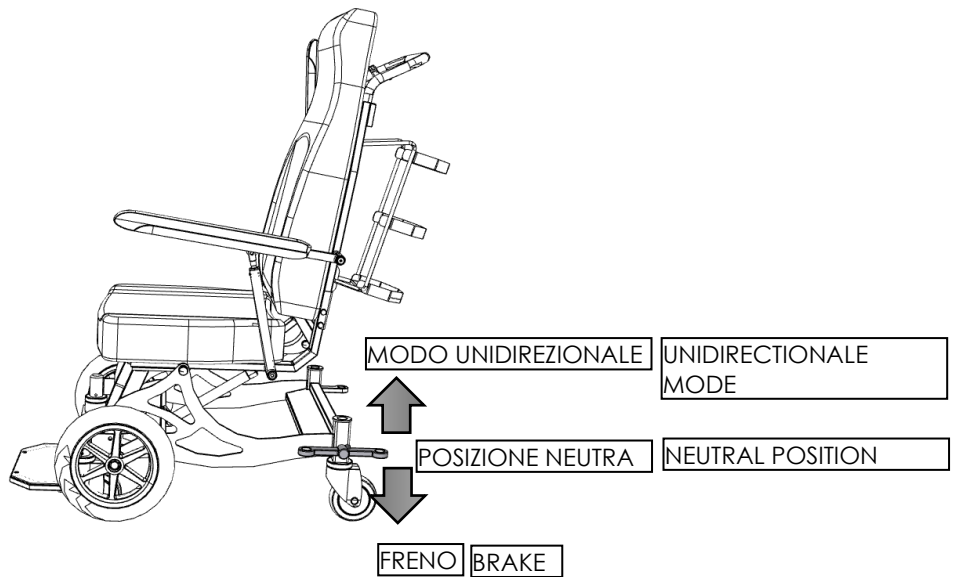
To retract the platform simply pull it upwards until it reaches its rest position.

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Scheduled maintenance is required for proper operation of the device, proper use, patient and operator safety, and performance assurance of the device.

Ordinary maintenance work may be carried out by specialised operators or authorised technical personnel.

| PERIODICITÀ | VERIFICA  |
|-------------|---|
| 3 months    | Check integrity of the back and seat                                |
|             | Verification of the operation of the inclination adjustment control |
|             | Integrity of the frame and absence of unsoldered elements           |
|             | Checking the sliding of the wheels and the swiveling                |
|             | Verification of the tip-up of the foot platform                     |
|             | Check of the folding of the armrests                                |
|             | Presence of bumpers on the base frame                               |
|             | Effectiveness of the brakes and the command of unidirectionality    |

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Extraordinary maintenance work may only be carried out by personnel authorised by the manufacturer. Otherwise any warranty conditions will be immediately terminated. The manufacturer is not responsible for any damage caused using this product.

Any malfunctions found by the user must be promptly reported to the distributor or directly to the manufacturer and inhibit the use of the device.

Repairs may only be carried out by technical personnel authorised by the manufacturer and may include the withdrawal of the device in order to carry out the necessary repairs.

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The service life of the device is defined at the beginning of this manual.

At the end of its useful life, you can proceed in the following ways:

- 1) Dispose of the device as described in the paragraph "Disposal".
- 2) Ask the manufacturer to recondition and recertify the device so that it can continue to be used

As stated in the paragraph "General warnings", the manufacturer declines all responsibility for the use of the device beyond the useful life established in this manual.

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It is necessary to clean the device at the end of each use if the device is intended for different users.

### 8.1 LAVAGGIO - WASHING

Remove dust from metal parts using mild detergents and drying immediately after washing.  
Clean the rubber parts and the antibrachial supports with a neutral product with a damp cloth and immediately dry the surfaces.

Remove dust from metal parts using mild detergents and drying immediately after washing.  
Clean the rubber parts and the antibrachial supports with a neutral product with a damp cloth and immediately dry the surfaces.

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For disinfection of the product use disinfection with a low chlorine content, such as AMUCHINA® 10% or equivalent solutions with a concentration of 0.1% sodium hypochlorite and the following spectrum of action:

- bactericide in the presence of interfering substances according to EN 1276:1997;
- a fungicide in the presence of interfering substances according to EN 1650:1997;
- bactericidal surface test according to EN 13697:2001;
- fungicide surface test according to EN 13697:2001;
- active on Salmonella Typhimurium according to EN 13697:2001;
- active on: HIV, HAV - HBV, HCV;
- virucide according to EN14476:2005;

active on H1N1 influenza virus according to EN14476:2005. For safe use, refer to the instruction leaflet included in the product.

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The symbol on the label of the equipment indicates that the waste must be subject to "separate collection".

Therefore, the user must either hand over the waste to the separate waste collection centres set up by the local authorities or hand it over to the retailer against the purchase of new equipment of an equivalent type. Separate waste collection and subsequent treatment, recovery and disposal operations promote the production of equipment with recycled materials and limit any negative effects on the environment and health caused by improper waste management. Illegal disposal of the product by the user entails the application of the administrative sanctions referred to in Article 50 et seq. of Legislative Decree no. 22/1997.

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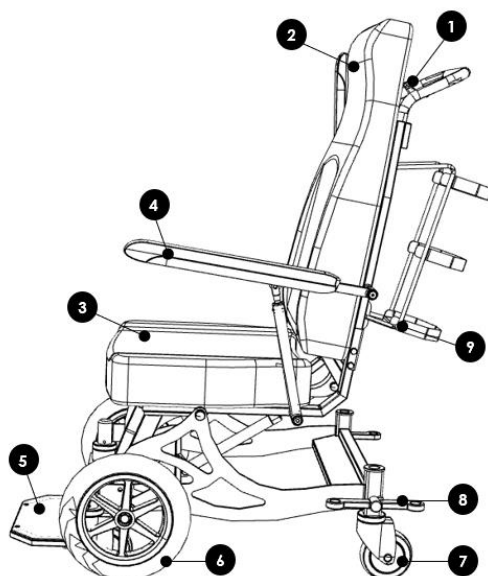
Contact your dealer or the manufacturer's technical support service for information on spare parts. Only use original spare parts supplied by the manufacturer or by authorised distributors.

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- 1 Tilt adjustment control
- 2 Padded backrest
- 3 Padded seat
- 4 Padded armrest
- 5 Footrest plate
- 6 Front wheels not swiveling
- 7 Swivel rear wheels
- 8 Brake control lever
- 9 Bottle holder (optional)

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### 12.2 RIPARAZIONI IN GARANZIA - REPAIRS UNDER WARRANTY

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All Chinesport products are warranted against defects in materials or workmanship for a period of 24 months from the date of sale of the product, except for any exclusions, limitations or conditions defined at the time of delivery of the product.

The warranty is not valid in case of improper use, tampering with the device, abuse or modification of the product or for any use or operation not explicitly mentioned in this manual.

The warranty is not valid if the device has not been correctly maintained and documented in accordance with this manual, or if the instructions regarding storage, cleaning and sanitation are not followed.

The manufacturer is not responsible for any damage or injury or any situation caused by incorrect installation or configuration of the device or using equipment that does not comply with the instructions in the installation, assembly and operating manuals.

The manufacturer does not guarantee its products against defects or damages in the presence of extraordinary conditions such as: natural disasters, unauthorised maintenance and repairs, improper power supply (where applicable), use of parts or components or accessories not original, shipping damage not directly managed by the manufacturer, lack of maintenance, obvious negligence on the part of the user or operator.

The warranty does not cover consumables, rechargeable batteries, and in general, all material subject to wear, failures caused by impacts, falls, misuse or improper, accidental events, damage caused by transport. If the equipment is tampered with, the warranty expires automatically.

In the case of a report of defects in materials or workmanship, the manufacturer assesses whether the defect is covered by warranty

Warranty repairs must be expressly requested and are to be understood in our laboratory, subject to authorisation and with the issue of the return number.

For products sent in their original packaging, the return shipment will be made freight free.

For warranty repairs, a fiscal document is required where the date of purchase is within the warranty period (sales note, purchase invoice, fiscal receipt).

Labour costs for warranty repair (when the warranty conditions are valid) are borne by the manufacturer.

Repairing or replacing a product does not renew or extend the terms and expiration dates of the warranty.

Non-warranty products can be repaired by the manufacturer by returning them after having been

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authorised by the technical assistance service. The costs of repair, including shipping, materials and labour, are to be understood as being borne by the customer or the retailer. The parts and components being repaired are considered to be covered by warranty for 24 months from the date of receipt of the repaired device.

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In the event that the manufacturer does not find any malfunction or defect in the returned products, it is concluded that the product is not to be considered as defective. Shipping and device management costs will be charged to the customer or distributor

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In case of repair at the customer's premises, a written request must be made indicating the complete details of the applicant, the type of machine and the fault. The kilometric cost for the technician's transfer is to be agreed in relation to the customer's urgency. In the event that the machine in question is under warranty, only the costs of the transfer will be charged. The time is counted from the departure of the technician from our laboratory until his return, the time of return will be estimated on the basis of the time spent on the outward journey.

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A detailed list of all spare parts can be obtained from the manufacturer. Spare parts are sold following a formal request for an offer to the technical assistance service. Processing times are related to the availability of the parts. Returns for spare parts are not accepted. The payment will be cash on delivery unless otherwise agreed.

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| TROUBLE  | SOLUTION  |
|--|---|
| The chair does not move                              | Check that the parking brake is not applied   |
|  | Check that the inclination of the chair is such that the footrest does not touch the ground           |
|  | Check that there are no elements obstructing the rotation of the wheels                               |
|  | If not resolved, contact the retailer or the technical assistance service                             |
| The chair does not rotate                            | Check that the parking brake has been released  |
|  | Check that the unidirectional command has been disarmed   |
|  | Check that there are no obstacles preventing the wheels from rotating                                 |
|  | If not resolved, contact the retailer or the technical assistance service                             |
| The chair is noisy while driving                     | Check that the inclination of the chair is such that the footrest does not touch the ground           |
|  | Check that the brakes are not inserted  |
|  | Check that there are no elements that block the rotation of the wheels                                |
|  | If not resolved, contact the retailer or the technical assistance service                             |
| The wheels are bitten and do not pirouette           | Check that there are no objects or dirt residues blocking the swiveling                               |
|  | If not resolved, contact the retailer or the technical assistance service                             |
| The wheels are no longer firmly secured to the frame | Check the tightening of the screw inside the wheel and tighten if the screw is not properly tightened |
|  | If not resolved, contact the retailer or the technical assistance service                             |
| The chair does not tilt                              | Check that you are not in the lowest or highest incline position                                      |
|  | Make sure to operate the control throughout its travel  |
|  | Check that the adjustment control cable is present  |
|  | Check that there are no oil residues on the floor   |

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|                          |   |
|--------------------------|---|
|                          | If not resolved, contact the retailer or the technical assistance service |
| The armrests do not tilt | Check that there are no elements that inhibit the inclination             |
|                          | If not resolved, contact the retailer or the technical assistance service |

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|           |                      |               |
|-----------|----------------------|---------------|
| APPARATUS | DATE OF INSTALLATION | SERIAL NUMBER |
|-----------|----------------------|---------------|

| DATE INTERVENTION | OPERATIONS PERFORMED | TECHNICIAN IN CHARGE | SIGNATURE | NEXT CHECK |
|-------------------|----------------------|----------------------|-----------|------------|
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |
|                   |                      |                      |           |            |

**MADE IN ITALY**