MultiEquipment. Ref F333

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

Radiofrecuencia
Radiofrequency
Radiofréquence
Radiofrequenzgerät
Radiofrequenza

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

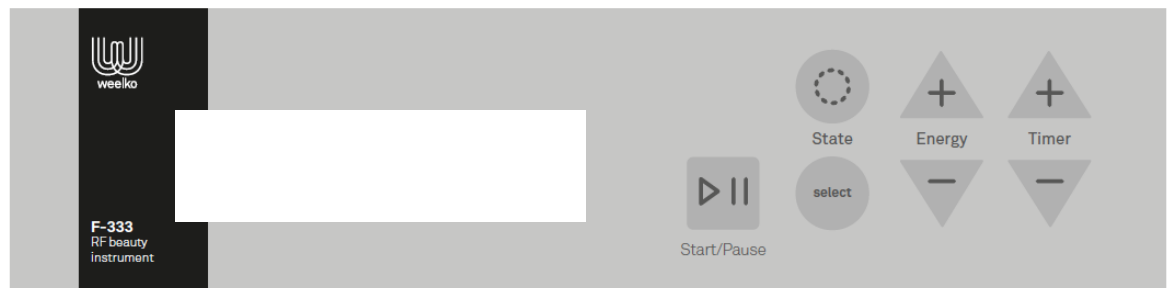
The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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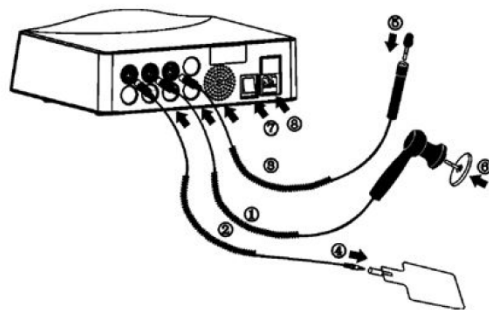
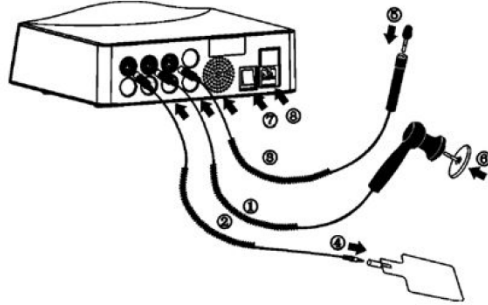
Instructions of use

1. Connect the power supply via the rear panel.
2. Connect the accessories properly to the front/rear panel as indicated in the sketch.

Number	Accessory/Connection
1	Body electrode holder wire
2	Negative plate wire
3	Face electrode holder wire
4	Negative plate
5	Face electrodes
6	Body electrodes
7	Power button
8	Power cord

3. Turn on the “POWER” button.
4. Press “SELECT” button to choose the desired electrode holder (face or body). The display will show the electrode holder that is working.
5. Put the negative plate in contact with a large surface of the skin. It must be close to the treatment zone.

Nota: to avoid burning to the customer, never forget to put this plate in contact with the skin.
Vegetable glycerin to be applied on the patient's skin is needed for a proper use of the radifrequency machine.

6. Choose the desired electrode according to the treatment. Depending the treatment zone is needed to choose an electrode of a proper size.
7. Apply enough glycerin in the treatment zone.
8. Press “START/PAUSE” button to start the treatment.
9. Choose the wave state according the customer need. Is recommended firstly choose discontinuous wave (for sensitive skins). Then, you can choose the other wave states according to the customer need/feelings and according to the discretion of the beautician.
10. Apply the electrodes on the customer’s skin making constantly circles, without stopping of making movements. Not keep the electrode without moving at the same zone during a long period of time, the customer could be burned.
11. Once the treatment has finished press “START/PAUSE” button.

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Cautions

- Radiofrequency treatments must not extend over a long period of time. The recommended duration is 10 minutes for facial treatments and 20 minutes for body treatments. In any case, treatments extend beyond 60 minutes.
- To avoid burning to the customer, never forget to put this plate in contact with the skin.
- Apply the electrodes on the customer's skin making constantly circles, without stopping of making movements. Not keep the electrode without moving at the same zone during a long period of time, the customer could be burned.
- Vegetable glycerin to be applied on the patient's skin is needed for a proper use of the radiofrequency machine.

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Secondary effects and contraindications

The secondary effects of the radiofrequency are:

- Redness and swelling after the session (are transient, but usually present).
- Some small ecchymosis (rare).

The radiofrequency treatment is contraindicated in:

- Pregnancy and lactation.
- Metal prostheses or pacemakers.
- Serious heart defects.
- Coagulation disorders.
- Connective tissue diseases and neuromuscular.
- Cancer.
- Patients with overweight.
- Recent collagen implants.

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