MultiEquipment. Ref F-350

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

Electroestimulación digital
Digital electroestimulation
Electroestimulation digital
Digitales Elektrostimulationsgerät
Elettrostimolazione digitale

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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EC-Declaration of compilance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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Introduction

The F-350 equipment works the muscle stimulation and contraction by using electric impulses to be applied on the Chinese acupuncture points of the body. The main aim of this system is to eliminate toxins and fat particles, to regenerate the body blood and lymphatic circulation and to improve the breast tissue. It is divided into three main programmes: body slimming, muscle strengthening and body massage and health care which consist of multiple subprogrammes to be used according to the patient needs. The programmes are described throughout the different touch screens of the unit.

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Product description

This digital system consists of a main unit with ten output channels. It includes three types of electrodes. The first type is a round-shaped electrode (with a radius of 2 cm) is designed for slimming and toning treatments, specially the facial one. The second type is eight-shaped (8,5 * 4,5 cm) is conceived for large areas, such as back, waist, hips, thighs, etc. Finally, the third one is oval-shaped (7,7 * 4,4 cm) ideal for reaching small areas. There are 10 pairs of each kind of electrode.

The equipment is placed on a wheel-base column which allows the professional to work with a great mobility and to have the set of electrodes organised into the housings. The unit has 10 output channels and 10 LED indicators. Select the channel according to the patient desire. Once the electrodes are correctly placed and the unit starts working, the corresponding LED indicators according to the output channels working will start to light.

The main treatments are body slimming, muscle strengthening and body massage and health care. The Body slimming treatment consists of nine programmes: breast enhancing, breast lifting, waist and belly reduction, hips reduction, thighs, calves and arms slimming, back and face toning. The Muscle strengthening treatment consists of seven programmes for chest, abdomen, shoulder and arms, forearms, back, thighs and calves. There are also six programmes for massage and health care treatment, including whole body lymph drainage, lymph drainage from the waist downwards, areolar tissue and abdomen improvement and toxins elimination. The different programmes are already set up according to the characteristics of each body part. In addition, it is equipped with a default time that can be adjusted according to the treatment.

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Operation instructions

- Instrument should be plugged on the power according to the requested technical parameters and connected to the earth leakage circuit breaker.
- In order to calibrate the screen, switch on the system and follow the instructions below:
 - a) Tap in any part of the screen except for the icons until you will hear a beep.
 - b) Press the small crosses that will appear on the screen.
 - c) Once the screen has been calibrated, get back to the main menu.

BEFORE THE TREATMENT:

- 1) Select the programme according to the area the patient wants to treat. Meanwhile, we recommend recording the current status of the patient and the proposed programme. This is to identify the difference before and after treatment and as guidance for next treatment.
- 2) Soften the surface skin to be treated with a steamer or hot pad for 10 min. And then massage the muscle for 5-10 min. with a scrub cream or oil. Meanwhile, it is recommended to create a relaxing and comfortable atmosphere.
- 3) Wash the excess cream or oil after the massage. Rub the skin surface with a damp sponge.
- 4) Apply some nutritive lotion or gel on the electrode surface. As shown in the diagrams, use elastic belts for a better fixation of the electrode on the treated skin.
- 5) Check that the electrodes are correctly placed. Also make sure that elastic belts are properly tightened and that the electrodes are in contact with the skin.

DURING THE TREATMENT:

1. Switch on the unit and select the language first. Then select the desired treatment.
2. Once the treatment is chosen, an interface will show you the details of the corresponding programme and where the electrodes should be applied. Once the electrodes are well-placed, press "OK" for next step.
3. Select PLAY to start working and time will begin to count down at the same time.
4. The corresponding channels should also be switched on. To start working, intensity should be adjusted at the minimum position. Later the intensity can be adjusted according to the reaction of the muscle.
5. Once the programme is finished, please remove the electrodes and clean the treated skin area with a wet

cloth or tissue.

6. Clean the unit and accessories after each treatment. The accessories should be sterilised as well.

The muscle strengthening programme allows selecting the different parameters for each programme:

- Interval: once the default time is finished, the unit will have a pause of one minute. If you adjust the interval parameter, you can modify the pause time.
- Group: Once the working and the pause time have finished, a cycle is completed. By adjusting the group parameter, you can select the number of cycles you desire.

Notes for the placement of the electrodes:

- One group of electrode consists of 2 pieces of electrodes with the same number and reverse polarity.
- Place the electrodes according to the polarity (+ or -). Otherwise the treatment effect will get reduced and the patient will feel uncomfortable.
- Each pair of electrodes should be symmetrically placed according to the schemes shown; the distance between both electrodes should not be less than 2 cm. entre ambos electrodos no debe ser menor de 2 cm.

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Electrodes maintenance

1. Electrodes should be cleaned after each use. Keep them dry and wrapped into their packing after clean and sterilisation.
2. Do not store them in high temperature and humid places. Avoid keeping them directly exposed to sunlight.
3. Electrodes should be replaced after 50 uses.
4. Do not keep the electrodes stuck with each other.
5. Do not remove or touch the electrodes from the body during operation. If it is necessary to do so, please turn off the machine firstly, and then remove the electrodes. The silicon part of the electrodes should not be touched with your fingers.

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Cautions

To make the most of this device, please observe the following advices.

- Do not tighten the elastic belts too strongly, because the patient could feel a stabbing feeling. On the other hand, do not leave it too loose to avoid reducing the treatment effect.
- Electrodes should be placed correctly regarding the instructions above. In some areas of the body it is not necessary to place the elastic belt to fix the electrodes. In this case it is enough to use the adhesive layer of the electrode. In cases which the elastic belt is not long enough, you can use two shorter belts attached by a Velcro.
- Adjust the output intensity based on the feeling of each area. If during the treatment any of the electrodes is released or moved from the patient's skin, press the "START/STOP" button to stop the machine. Place the electrode properly again before continuing the treatment.
- The duration of the treatment for one area should never exceed 60 minutes. If it is necessary to repeat the treatment in certain areas, you must wait for 1-2 hours.
- After the treatment the electrodes can get stained due to dirt or the use of different gels. In order not to reduce the properties of the equipment, cleaning the electrodes surface is recommended after each use.
- Do not place any electrode on any injured skin surface or the patient might feel some discomfort. To avoid this, it is necessary to place the electrode on a bandage.
- Patient may feel discomfort during the initial stage of treatment since muscles produce lactic acid which is a normal physiological phenomenon. To relieve the ache, the beautician can massage the muscle with some cream.
- Do not place the electrodes on areas close to the heart, on swollen or dilated veins or chafing skin.

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Contraindications

- You must not use this instrument on the following people:

 1. People with pacemakers
 2. People suffering from heart problems
 3. People suffering from arrhythmia
 4. People suffering from epilepsy
 5. Pregnant women
 6. People having ingested alcohol
 7. People suffering from stomach, liver, bladder or kidney problems

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TECHNICAL SHEET

Weights and sizes	Packing size (cm)	73x67x29
	Gross weight (kg)	22
	Net weight (kg)	18,5
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Working power (W)	60
	Treatment time (min.)	1 - 60

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