# Invacare® Propad®

Revolve<sup>V</sup> / Revolve<sup>Si</sup>

en	Cushion User Manual .....	1
es	Cojín Manual del usuario .....	5
it	Cuscino Manuale d'uso .....	9
pt	Almofada Manual de utilização .....	13

## General

### General information

Repositioning regularly is recommended to manage the risk for and aid in the prevention of skin breakdown. Essential nursing care and a holistic approach is pivotal when prescribing mobility and seating products. Education and a thorough clinical assessment, are also fundamental factors in the prevention of skin breakdown.

This user manual contains important information about the handling of the product. In order to ensure correct use of the product, read the user manual carefully and follow the safety instructions. For further information contact Invacare in your country (addresses are on the back page of this manual).

### Symbols in this user manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

#### WARNING

Indicates a potentially hazardous situation which if not avoided could result in death or serious injury.

#### CAUTION

Indicates a potentially hazardous situation which if not avoided could result in product damage, minor injury or both.

#### IMPORTANT

Indicates a hazardous situation which if not avoided could result in damage to the product.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

### Warranty information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

### Standards and regulations

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

The Invacare® Propad® Revolve<sup>V</sup>/Revolve<sup>Si</sup> cushion features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

The foam and cover used to manufacture the Invacare® Propad® Revolve<sup>V</sup>/Revolve<sup>Si</sup> cushion are fire safety tested and certified in accordance with EN1021 part 1 and 2.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (addresses are on back page of this manual).

### Intended use

This cushion is intended to be used in conjunction with an overall skin protection program of care. Designed to offer high level pressure redistribution, support, and comfort to static seating systems and/or monitored wheelchair seating,

This manual MUST be given to the user of the product. BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**

this cushion can be used for "High Risk" users when in normal use. Normal use is defined by Invacare Ltd as placement of the product, without additional covers or padding, between the user and the support surface. Replace the cover in case of damage. Do not use with a cover other than a Propad® Revolve cover.

## Product description

The Invacare® Propad® Revolve<sup>V</sup>/Revolve<sup>Si</sup> cushion is a multi-layered, planar cushion for static and wheelchair seating. Its outer layer consists of high resilient, moulded foam which provides immersion and pressure redistribution. Ultra-Fresh™ treatment of the foam ensures lasting freshness and longer life by providing enhanced antimicrobial protection and odour reduction. The Propad® Revolve<sup>V</sup> has an inner visco-elastic foam layer, allowing for targeted immersion and increased protection in areas with vulnerable bony prominences. The Propad® Revolve<sup>Si</sup> contains a welded fluid sac, divided into four sections, that provides immersion and envelops areas at highest risk for skin breakdown. The welded sections in the fluid sac prevent unwanted migration of the fluid. The water-resistant cover provides a vapour-permeable surface, to ensure patient comfort and maximise the effectiveness of the foam. The cover fabric provides sufficient grip to keep the cushion in place once positioned in the seating system. The Invacare® Propad® Revolve<sup>V</sup>/Revolve<sup>Si</sup> cushion provides high level skin protection, support, and comfort for "High Risk" users.

## Safety

### Safety information

#### WARNING!

Do not use this product or any available optional equipment without first completely reading and understanding the user manual supplied. Invacare product manuals are available on your local Invacare website or at your local dealer. If you are unable to understand the warnings, cautions or instructions please contact a health care professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

#### WARNING!

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

The introduction of certain third party products between the cushion surface and the user may reduce or impede the clinical effectiveness of this product.

'Third party products' may include, but are not limited to items including under blankets, plastic sheets and sheepskins, etc.

#### WARNING!

##### Risk of developing pressure ulcers

Any object between the user and the pressure reducing surface could increase risk for skin breakdown.

- Make sure that the support surface in contact with the user is kept free from crumbs and other food debris.
- Drip cables and other foreign objects must not become trapped between the user and the pressure reducing surface of the cushion.

#### IMPORTANT!

The information contained in this document is subject to change without notice.

- Check all parts for shipping damage and test before using.
- In case of damage, do not use.
- Contact Invacare for further guidance/information.

### Symbols on the product

	CE conform		Do not bleach
	Tumble dry low heat		Do not iron
	Do not dry clean		Maximum machine wash temperature 80 °C
	Do not pierce or cut		Do not put near flame
	Max user weight		Date of manufacture

## Usage

### Safety information

#### WARNING!

Invacare advise that the patient is assessed by a trained health care professional. This is to ensure the correct size and seating requirements are achieved for the best pressure and postural care and that the individuals needs are met.

#### WARNING!

It is very important for the patient to reposition themselves, or to be repositioned, on a regular basis. This relieves pressure which helps prevent both tissue compression and potential skin breakdown. This must be based on the clinical judgement of a qualified healthcare professional.

- Always consult a qualified health care professional before using the product.
- Monitor the patient frequently.

#### WARNING!

##### Risk of developing pressure ulcers

The zipper pull can cause pressure ulcers when permanently pressing against the underside of your thighs.

- Always place the cushion onto the seat with the zipper pull to the rear.

**IMPORTANT!**

In domestic settings common causes of damage include cigarette burns and the claws of pets that puncture covers, allowing fluid ingress and staining.

**IMPORTANT!**

- To prevent accidental cover damage, please ensure that sharp objects are not placed on the cover. When using patient transfer aids, care should be taken not to damage the cover. All transfer aids should be checked for sharp edges or burrs before use.
- Ensure that the cushion is not jammed or damaged by sharp edges if used on wheelchairs.

**IMPORTANT!**

The covers are breathable and water resistant, however with sustained periods of moisture the top layer of the cover may absorb fluids.

- To avoid this, wipe away any liquid as soon as it is identified.

**Using the cushion**

1. Remove all packaging before use.
2. Place the cushion on the seat surface with the zip preferably towards the rear.

**Maintenance****Inspection**

Check cushion (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the use of each patient and on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

**Check cushions**

1. Unzip the cover completely.
2. Check for any staining on the interior foam.
3. Replace any damaged cover or stained foam and dispose of as per local authority procedure.

**Cleaning and care****IMPORTANT!**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.

- For further information on decontamination in health care environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Healthcare-associated infections: prevention and control in primary and community care [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local country infection control policy.

**Cleaning covers**

(Removal of contaminants such as dust and organic matter)

1. Remove cover for laundering.
2. Launder the cover with the maximum temperature of 80 °C using a diluted detergent solution (Instructions on label).

**IMPORTANT!**

Washing at higher temperatures will cause shrinkage.

**Drying covers**

1. Hang cover from a line or bar and drip dry in a clean indoor environment.  
or  
Tumble dry on a low heat setting.

**IMPORTANT!**

- Tumble dry setting must not exceed 40 °C.
- Do not tumble dry for longer than 10 minutes.
- Dry thoroughly before use.

**Disinfecting covers**

(Reducing the number of microorganisms)

Please contact your hygiene specialist in the event of contamination.

**IMPORTANT!**

- Ensure that any residual detergent has been removed with clean water prior to disinfection.

**Light soilage**

1. Wipe down the cover with a 0.1% Chlorine Solution (1,000 ppm).
2. Rinse the cover with clean water using a single use non-abrasive cloth.
3. Dry the cover thoroughly.

**Heavy soilage**

Where the cushion is badly soiled, we recommend cleaning with a dilute cleaning solution at 80 °C in the washing machine.

Large spillages of blood should be absorbed and removed, followed by as above.

1. Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a 1% Chlorine Solution (10,000 ppm).
2. Rinse with clean water using a single use non-abrasive cloth.
3. Dry the cover thoroughly.

**IMPORTANT!**

1% Chlorine Solution used on a regular basis can diminish the life of the cover if not rinsed and dried properly.

- Do not use granules.

**WARNING!**

- Remove contaminated foams from use.

**CAUTION!**

- Keep clear of open heat sources.

**IMPORTANT!**

- Do not use phenols, alcohols, bleaches, or other abrasive materials.

**Replacing covers**

1. Unzip the cover and remove it carefully from the foam core.
2. Place new cover onto the foam core.
3. Close the zipper.

**IMPORTANT!**

- Ensure that the corners of the foam core are positioned correctly into the corners of the cover.

## After Use

### Storage

#### IMPORTANT!

- Store cushions in a dry environment.
- Store cushions within a protective cover.
- Store items on clean, dry surface free from sharp edges to avoid any possible damage.
- Never store other items on top of a cushion.
- Do not store cushions next to radiators or other heating devices.
- Protect cushions from direct sunlight.

1. Refer to the storage and shipping conditions in section *Environmental Parameters, page 4*.

### General data

Product	Total Height [mm]	Total Width [mm]	Total Depth [mm]	Nominal density range [kg/m <sup>2</sup> ]	Nominal hardness range [N]	Minimum / maximum user weight [kg] <sup>2)</sup>	Weight of product [kg] <sup>1)</sup>
Propad® Revolve <sup>v</sup>	75	355 - 610	355 - 508	HR Foam: 70	160 – 200	40 / 190	1.25
Propad® Revolve <sup>Si</sup>	75	355 - 610	355 - 508	70	160 – 200	40 / 190	2.5

<sup>1)</sup> Weights can vary depending on size of cushion ordered, average weights used as an indication.

<sup>2)</sup> Maximum user weight limits are set provided that the cushion selected is of appropriate width and depth for the intended user(s). The maximum user weight limit applies to a 610 mm x 508 mm cushions.

Local requirements / certification available upon request from Invacare.

### Materials

Foam	Polyurethane foam components	Inner liner	Polyurethane ether film
Cover	Polyurethane	Fluid	Silicone
Fluid sac	Polyether based thermoplastic polyurethane film		
All cushion components are Latex-Free.			

### Environmental Parameters

	Operating conditions	Storage and shipping conditions
Ambient temperature	10 - 35 °C	-40 - 70 °C
Relative humidity	30% - 75% Non-Condensing	10% - 100%
Atmospheric pressure	70 - 106 kPa	50 - 106 kPa

### Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used.

1. Before reuse, clean the product thoroughly, see section *Cleaning and care, page 3*.

### Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

1. Ensure that the cushion is cleaned prior to disposal to avoid any risk of contamination.

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Invacare Limited
Invacare Ireland Ltd,
Invacare UK Operations Limited