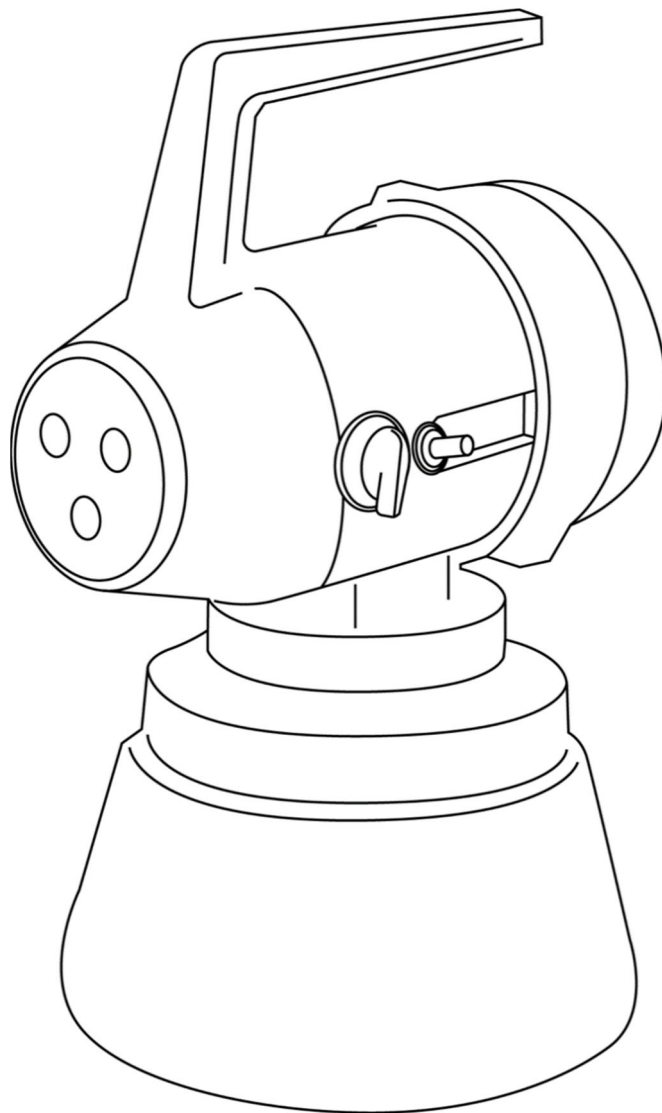
# SafetySpot

<b>ITALIANO</b>	<b><i>Istruzioni Per L'uso</i></b>
<b>ENGLISH</b>	<b><i>Instructions For Use</i></b>
<b>FRANÇAIS</b>	<b><i>Manuel De L'utilisateur</i></b>
<b>DEUTSCH</b>	<b><i>Anwenderhandbuch</i></b>
<b>ESPAÑOL</b>	<b><i>Manual Del Usuario</i></b>

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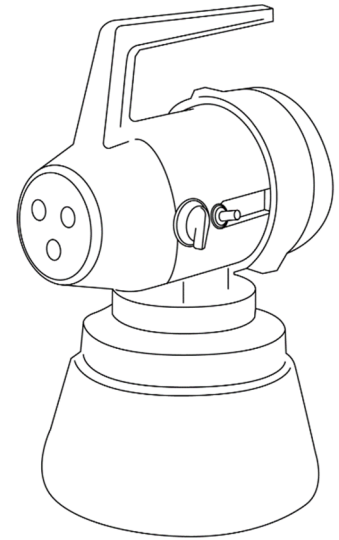
# Use and maintenance

## Technical features

- Vortexes and delivery regulator in hostafon.
- Tank in corrosion-proof polythene.
- Silicone type delivery and suction lift pipes.
- Structure with double insulation against short circuits.
- Switch directly mounted on the molar casing.
- 5 m cable with current tap supplied.
- Delivery regulation valve.
- "CE" symbol.

## Technical data

- Motor: universal 220 V, 50 Hz. Others are available on request voltages and frequencies (100 V, 50/60 Hz; 110 V, 50 Hz; 120 V, 60 Hz).
- Power: 1.000 W.
- Tank capacity: 5 l.
- Empty weight: 3,5 Kg.
- Dimensions: 29 (l) x 43 (h) x 26 (p) cm.
- Delivery: 15 l / hour max. (with product density 1).
- Nebulization density: 5 l. enough to saturate a volume of 4.500 m3.
- Acoustic intensity: 69 dB.

## Warnings

- Install a circuit breaker switch in the current socket used with the sprayer.
- Check that the cable is not put under tension during work. The electrical connection must be made in such a way as to prevent people and machinery from damaging the cable, putting themselves and the operator in danger.
- Avoid using the nebulizer with a wet floor.
- Do not use the nebulizer simultaneously with other electrically powered tools in the same room.
- Always keep the engine ventilation clean and clear.
- Disconnect the power during maintenance operations.
- Carefully follow the instructions for use and the warnings specified for each liquid used in the nebulizer, paying particular attention to irritating and, in any case, dangerous and harmful to health liquids.
- Do not lie down or turn the nebulizer upside down as liquid may leak or short circuit.
- Do not use against wind or forced ventilation.
- Keep away from heat sources.
- Keep out of the reach of children.

**Note:** the drops of liquid that form on the equipment after use are due to the normal bleeding of the machine itself.

## Conditions and limits in use

- 1) This machine is a nebuliser that must be used according to the prescriptions that correspond to the descriptions and warnings concerning safety, mentioned in this manual.
- 2) Any other sort of use is not part of the prescriptions. The manufacturer shall not be held responsible for any damage owing to other uses. The risk is entirely at User's charge.
- 3) Part of the use according to the prescriptions, is also that to observe the rules regarding use, maintenance and repairs given by the Manufacturer.
- 4) Only those persons, who have good knowledge of the nebuliser and of its dangers, may use, repair, and carry out maintenance on it.
- 5) Haphazard modifications of the nebuliser exempt the manufacturer from any responsibility for damage owing from it.
- 6) Apart from the warnings given in this instruction manual, also follow the general safety and accident preservation C.E.E. rules and those of the nations in which this nebuliser being used.
- 7) In addition to the warnings contained in the instruction manual, also observe the general rules relating to the safety and accident prevention of the State where the nebulizer operates.
- 8) Who uses the nebulizer must use the PPE indicated on the safety sheet by the manufacturer of the liquid to be nebulized.

## Preparation of the appliance

The device is supplied ready for use, tested and tested in the factory with plain water, to avoid dirtying it with unwanted liquids. It is sufficient to unscrew the body of the nebulizer from the tank and fill the latter, even partially, with the liquid to be used already diluted. Screw the body back on, avoiding excessive strain on the handle.

Check that the voltage and frequency indicated on the appliance rating plate and in the technical data below correspond to those of the network; then insert the plug correctly into the socket.

Start spraying by holding the nebulizer by the handle and placing the ignition lever in the "ON" position. The dispenser allows an adjustment between a minimum and a maximum without however completely closing the jet.

The direction of rotation does not matter, and half a turn is sufficient for adjustment. See the orientation of the index with respect to the word MIN and MAX.

## Use

This use is intended for bactericidal treatments. The mist formed by the nebulizer is so fine and widespread that it reaches every place, even where it has not blown directly, without dripping along the walls or soaking paper and fabrics.

However, it is necessary not to get too close to the walls or other objects with the appliance because the spraying is perfect beyond 2 meters. The useful distance is 6-7 meters, depending on the type of liquid used, and is able to meet the needs of any environment. The nebulizer can also be useful as a humidifier, simply nebulizing water, or as a distributor of deodorant liquids.

## Maintenance

Drain off the liquid from the tank after use, wash the tank and empty the internal pipes by allowing the sprayer to operate empty for a few seconds. Store the machine in a sheltered place.

## Warranty

If we have doubts about the causes and solutions of an inconvenient datum, contact our Authorized dealer. This is essential during the warranty period, as any repairs made by unauthorized workshops will void the warranty. The materials of construction are guaranteed for 12 (twelve) months and the Company undertakes to replace free of charge the damaged parts. The labor required for the repair and any transport costs will be borne by the customer.

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**TECNO-GAZ S.p.A.**