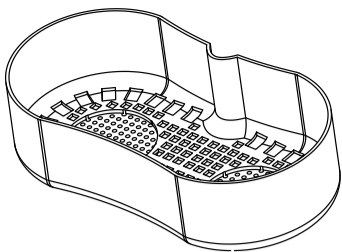
ES - EN

UC-001
Manual de instrucciones
Instruction manual
WE-2018-UC-001

ESTERILIZADOR DE ULTRASONIDOS ULTRASOUND STERILIZER

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ENGLISH

WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU



2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	UC-001	-	1
2	TRAY	-	1

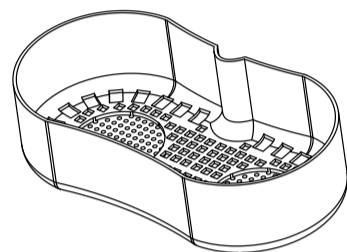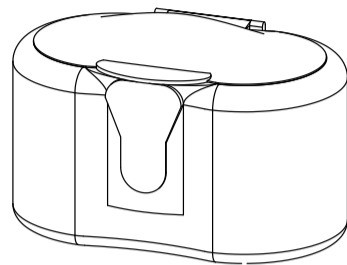CAUTIONS

• During long periods of continuous operation, the water temperature will rise a little. This is a natural phenomenon.

• Please do not place objects directly in the cleaning basin.

• Do not clean any substances with high levels of acidity and alkalinity.

• If water gets into the bottom of the cleaner or the machine starts to work strangely, please disconnect the power immediately and contact the dealer.

MAINTENANCE

• After cleaning the objects, drain out the water and use a towel to wipe the cleaning basin and the net.

• After cleaning, drain out the in order to prevent the formation of bilges caused by dirt sediment remaining in the water.

• During operation, the temperature of the piezoelectric magnetic parts rises. In order to protect the cleaning machine and ensure it works properly for a long time, avoid continuously starting the machine.

INSTRUCTIONS OF USE

1. Place the objects in the isolation net.

2. Pour water or purified water into the cleaner. The water should not exceed the maximum level. Add a little cleaning solution to the water.

3. Place the isolation net in its correct position in the cleaner.

4. Close the top lid of the cleaner and turn the unit on by pressing the "POWER" button on the rear panel.

5. Adjust the time using the "TIMER" button.

6. Press "START/PAUSE" button to begin operation. Pressing the "START/PAUSE" button again makes the cleaner stop working.

7. When the cleaning process ends the buzzer sounds three times. Then, you can open the top lid and take out all the objects.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	30x19,5x21,5
	Gross weight (kg)	2,1
	Product size (cm)	26,2x16,8x15,6
Working characteristics	Capacity	1,2 l
	Sterilization net size (cm)	22x10,5x6,5
	Recommended operating time	1 - 3 min
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	60 W

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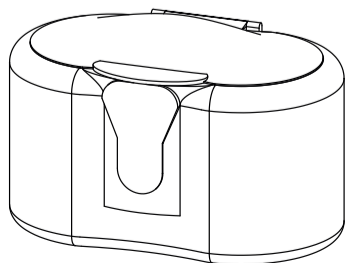
1

2

© 2018 Weelko
Visit our website at www.weelko.com for more information
Importer: Weelko Barcelona, S.L.
ESB65397556

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