

## **WALKERS**


VARIANT
WITH SUPPORTS
AXILLARY-BRACHIAL
UPHOLSTERED TABLES
PECTORAL-PELVIC



SERVICE MANUAL GUIDELINES MAINTENANCE TECHNICAL SHEETS





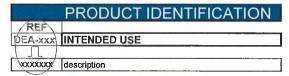



	SPECIFIC INSTRUCTIONS FOR USE	WARNINGS AND RISKS
	proceed with four-wheeled model	Grip the walker pay attention to have previously braked the wheels, to prevent the walker from escaping. Check that the weels are locked in advance of the handle
A	Clamping device on folding models 75CH model	Bend the sides on themselves. When reopening, insert the handlebars and tighten the front knobs.
В	Clamping device on removable models 75C E 73PR models	Loosing the bolting and or knobs only for transport needs. Chech the complete tightening before the next use.
1	integrated brake on wheels	Brake the wheels before the handle, to prevent the walker from escaping
2	Brakes for slowing down and parking by lever.	Parking block by hooking red lever. Before holding the walker lock the brakes
3	frame height adjustment	Unscrew the knob and remove the extendable at the holes, then screw back. Follow medical-orthopedic indication to maintain proper walking and posture.
4	independent adjustment of handle height, or padded table, pectoral support, axillary or antibrachial	Unlock the top clamp, unscrew the latch knob and remove the extendable at the holes, and lock the clamp and knob again. Follow medical-orthopedic indication to maintain proper walking and posture. Unless otherwise prescribed by medical-orthopedic keep the handles at the same height
5 6 7 8 9	Axillary support  Antibrachial support  Padded table support  Bib support  Pelvic support	Place at height according to medical-orthopedic prescription, acting on adjustments. Do not exceed in height to avoid overturning. Use only after appropriate training and on safe routes.
10	Seat insertion on forward-made models	Act by inserting a curved locking bracket on a horizontal lateral tubular.  Brake the walker before using the seat



### PRODUCT IDENTIFICATION



### SPECIFIC INSTRUCTIONS FOR USE

WARNINGS AND RISKS

















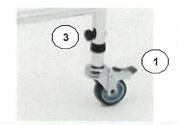

### DETACHABLE

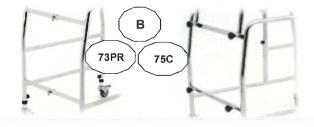















### TRANSPORT, STORE AND DISPOSAL

Check proper packing to be strong and closed

During storage and transportation check room and vehicle to be clean, possibly protect with anti dust plastic bag

The product do not contain components classified as hazardous at disposal Check and apply local disposal rules

### MANTEINANCE AND CLEANING

Check every 3 month wheels and tyres, at least pressure and distorsion

Check every 6 month tightning of screws and bolts Check every 6 month and possibly apply lubricant on moving parts

Subsitute immediately defective parts

Regular cleaning to be done with water and soft detergent Special cleaning and sanification with low pressaure steam Wash and dry immediately after contact with sand and salt water

For special maintenance appy to the authorized dealer



## INSTRUCTION/WARNING GENERAL RISKS

The selection of the device is due to the user with the endorsement of medical-ortopaedhic staff Contents of this manual need to be appliedin order to avoid injury or damages. The producer can modify datas and accessories listed in this manual

Pay attention to parts exposed to sun light, they could warm up excessively Use only on flat surface, stable and uniform
Do not use on staircase, mobile stair or steps
Be careful reagrding carpets, eelctric cable or other object on the floor
During transfer to a wheelchair, lock brakes of both walker and wheelchair
Do not exceed Max load
Avoid long distance from the walker, it can run instable
Do not use the walkers with seat like a wheelchair
Before sitting lock the brakes



PUBBLICATION 18.05.2021 REV 06/2021 INSTRUCTION MANUAL DEA-75-73PR

DLG DIRECTIVE 37/10 2007/47/CE

# CE

### **UE CONFORMITY DECLARATION**

with relation to the products identified as:

FOLDABLE-DEMOUNTABLE WALKER ON WHEELS PREPARED FOR USE OF SUPPORT AS

UNDERPIT, FOREARM, WALKING TABLE, BREAST, PELVIC AND SEAT

REF	
DEA-xxx	UDI 08058480835089
75CH	FOLDABLE WALKER ON WHEELS PREPARED FOR SUPPORT AS
	UNDERPIT, FOREARM, WALKING TABLE, BREAST, PELVIC AND SEAT
75C	DEMOUNTABLE WALKER ON WHEELS PREPARED FOR SUPPORT AS
	UNDERPIT, FOREARM, WALKING TABLE, BREAST, PELVIC AND SEAT
73PR	DEMOUNTABLE WALKER ON WHEELS PREPARED FOR SUPPORT AS
	UNDERPIT, FOREARM, WALKING TABLE, BREAST, PELVIC AND SEAT
	and the related accessories to compose kit system
+SED	SEAT FOR WALKER
+SPE	PELVIC SUPPORT FOR WALKER
+ASC	UNDERPIT SUPPORT FOR WALKER
+ABR	FOREARM SUPPORT FOR WALKER
+TAV	WALKING TABLE FOR WALKER
+SOP	BREAST SUPPORT FOR WALKER
+ACC	ACCESSORIES FOR FRAME AND WHEELS
MAXI	MAXIMUM CAPACITY 250 KG

#### Classified as:

### CLASS I MEDICAL DEVICE - Annex VIII (non-invasive)

The manufacturer Piai Ortotech srl, with registered office in Vittorio Veneto via Antonio Meucci 32, C.F. e P.I. 04193400266, Registration register of companies TV330742, declares under its sole responsability that the devices described comply with the requirements of regulation (UE) 2017/745 repealing Council Directives 90/385/EEC and 43/42/EEC.

Technical reference norm applied: UNI EN ISO 11199-2:2005 UNI EN ISO 11199-3:2005

Vittorio Veneto, (\*) see the original Legal representative Piai Fabio

Responsable for the respect of the normaty Campodell'Orto Mauro



### MANUFACTURER

PIAI ORTOTECH srl

Via Meucci 32 - 31029 Vittorio Veneto (TV) ITALY tel +390438500884 fax +390438902058

### **GUARANTEE CONDITIONS**

The following guarantee cover for the period of 24 months from the purchase date all defects related to design, materials, fabbrication and assembly, excluding from the guarantee damages due to wear, modification or improper use.

The defective product will need to be retuned together with a copy of present Guarantee Conditions, with a Purchase Proof and a filled up DEFECT REVIEW as below.

The Guarantee become void if Purchase proof will be missing, unauthorized repair attempt, damages from accidental event, non observance of instruction and warning reported at the user manual.

DEFECT REVIEW FOR INFORMA	TION:		-		
ITEM REF:			]		
ACCESSORIES					
DEFECT					
COMPANY REFERENCE					
PURCHASE REFERENCE					

with reference to Italian Privacy law regulation 196/2003, the information will be used only for Tecnical Assistance purposes





