


**KM62**

## Perforated polyurethane foam bariatric mattress



### Intended Use:

Prevention of pressure ulcers

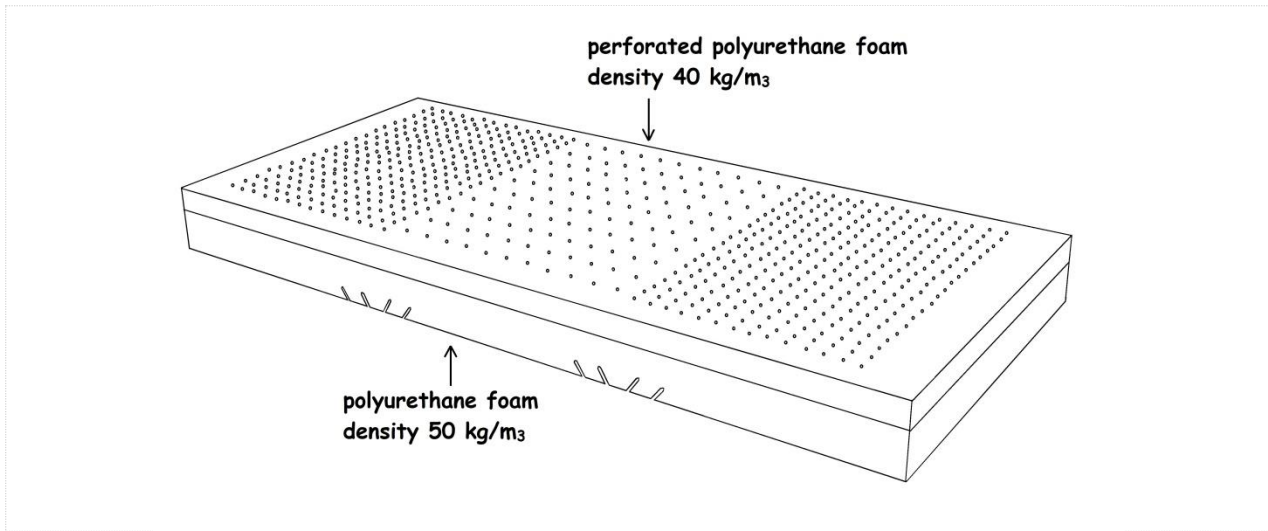

### Instruction for use

Place the device upon the hospital bed. An arrow placed on one of the two major sides of the mattress allows you to identify the support side.

A cover between the user and the device is recommended.

### Maintenance and cleaning instructions

Wrap tightly, tie and washing machine using a standard detergent. Rinse using a mangle or a low-speed spin-dryer. Dry away from direct sunlight or in a warm air stream at max 70° C. Sanitize by saturated steam at max 106° C for ten minutes.

### Warning and precautions

- Single patient use.
- Use and store in a horizontal position, away from free flames or heat sources.
- The KM62 mattress is extremely sensitive to sunlight and air. An integral cover is recommended. A possible colour fading is not an indication of a failure in the expected performance.
- Sanitize before each reuse.
- No side effects are known.

### Disposal

The product must be disposed of properly according to local laws and regulations. A correct disposal of the product prevents negative environmental and health effects and makes the reuse of the material easier, saving energy and resources.

**KM62**

**Perforated polyurethane foam  
 bariatric mattress**



**Product code and size**

	length	width	height	weight	nominal load
<b>KM6220012017</b>	200 cm	120 cm	17 cm	19 kg	300 kg