# HERCOLINO

15,5 cm	84 - 108 cm	205 g	120 kg

Fabbricante  
Manufacturer  
Fabricant  
Hersteller

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**MATERIALE:** Poliammide, PVC, polietilene e polipropilene.

## MANUAL FOR THE HERCOLINO

Transfer belt class 1 medical device complying with directive 93/42 EEC

The **HERCOLINO** transfer belt is suitable for transferring and moving the patient from a sitting to a standing position, for transferring from one seat to another and for short movements.

**HERCOLINO** allows the assistant to make the transfer with a good working posture avoiding excessive loads on joints and back, optimising control by the operator, while distributing the effort and generating trust and comfort of the patient. It is equipped with padded grips and an adjustable strap and buckle with double safety hook.

**WARNING:** The medical device must be used by qualified personnel or professionals.

If in doubt as to how to apply the device, consult an orthopaedic technician or qualified personnel.

The device has been designed to be well tolerated by the patient.

**MAINTENANCE:** Do not wash in a washing machine or at high temperatures (over 30°C) as this may damage the materials. Dry far from sources of heat. Store and transport in a dry place at room temperature.

**MATERIAL:** Polyamide, PVC, polyethylene and polypropylene.

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