
# **Stepless™**






*by Guldmann*

CE

F .....	Rampes transportables .....	3
I .....	Rampe .....	11
N .....	Skinner/ramper .....	19
FIN .....	Luiskat .....	27
US .....	Ramps .....	34

Vers. 1.00

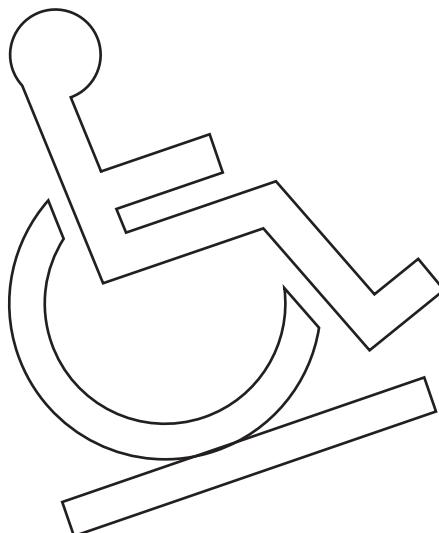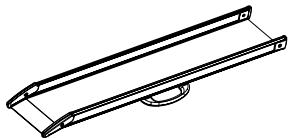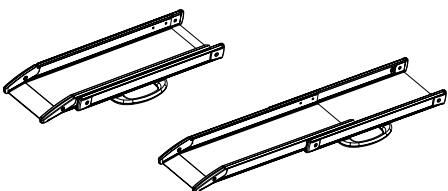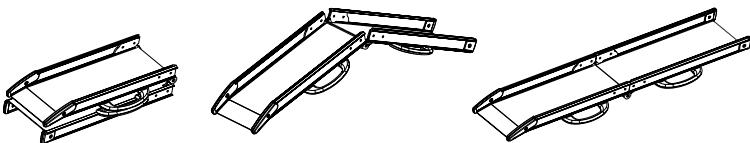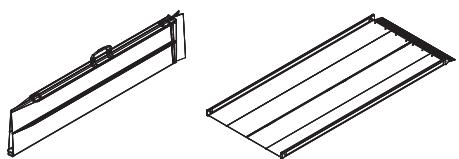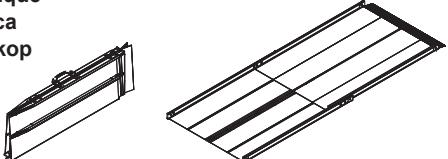I	Rampe étroite fixe et rampe large fixe Rampa fissa e rampa larga fissa Skinne fast og bred fast rampe Kiinteä tasoluiska ja leveä tasoluiska Plain ramp and wide plain ramp	
II	Rampe télescopique Rampa telescopica Skinne teleskop Teleskooppiluiska Telescopic ramp	
III	Rampe étroite pliante et rampe large pliante Rampa pieghevole e rampa larga pieghevole Skinne foldbar og rampe bred foldbar Taittoluiska ja leveä taittoluiska Folding ramp and wide folding ramps	
IV	Rampe portefeuille simple Rampa a valigetta Rampe koffert foldbar Salkkuluiska EasyFold ramp	
V	Rampe portefeuille télescopique Rampa a valigetta telescopica Rampe koffert foldbar, teleskop Salkkuluiska teleskoolla EasyFold Telescopic ramp	

# F.....Rampes transportables

## Références:

30000 - xxx  
30010 - xxx  
30015 - xxx  
30020 - xxx  
30030 - xxx  
30040 - xxx  
30060 - xxx  
30070 - xxx

---

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Representative      Company \_\_\_\_\_  
Address \_\_\_\_\_  
Country \_\_\_\_\_  
Phone \_\_\_\_\_

Hereby declare that      Product      **Stepless ramps**

Type No.      **XXXXX-YYYY-ZZZZ** x)

x) XXXXX: Reference No. of the product, YYYY: Serial No. of the product in the production year, ZZZZ: Production year

was manufactured in conformity with the provisions in:

- EN 12182:      Technical aids for disabled persons -  
General requirements and test methods

according to Council Directive 93/42/EEC of 14 June 1993.

Skejby 15.06.2002

Place and day of issue

Product responsible

Technical Manager

# US ..... Ramps

## Product Numbers

30000 - xxx  
30010 - xxx  
30015 - xxx  
30020 - xxx  
30030 - xxx  
30040 - xxx  
30060 - xxx  
30070 - xxx

---

<b>1.00</b>	<b>Purpose and application</b>	<b>35</b>
1.01	Manufacturer	35
1.02	Purpose and use of the ramp	35
1.03	Important/warnings	37
1.04	Unpacking and preparation of the ramp	37
<b>2.00</b>	<b>Description of functions</b>	<b>37</b>
2.01	Pictograms/labels	37
<b>3.00</b>	<b>Transportation</b>	<b>37</b>
<b>4.00</b>	<b>Maintenance and storage</b>	<b>37</b>
4.01	Cleaning the ramp	37
4.02	How to prevent/avoid corrosion	37
4.03	What maintenance is required by the owner?	38
<b>5.00</b>	<b>Service and useful life</b>	<b>38</b>
5.01	Useful life and servicing of the ramp	38
5.02	Inspection of Stepless	38
<b>6.00</b>	<b>Technical specifications</b>	<b>39</b>
<b>7.00</b>	<b>EC-Declaration of conformity</b>	<b>40</b>
<b>A.</b>	<b>Users guide</b>	<b>41</b>
<b>B.</b>	<b>WARRANTIES</b>	<b>41</b>

---

---

## **1.00 Purpose and application**

---

### **1.01 Manufacturer**

V. Guldmann A/S  
Graham Bells Vej 21-23A  
DK-8200 Arhus N  
Denmark  
Tel. +45 8741 3100  
Fax +45 8741 3131

---

### **1.02 Purpose and use of the ramp**

---

#### **Purpose**

Stepless ramps are a carefully designed and fully developed series of trans- portable driving ramps which can be used where wheelchair users need to overcome height differences of up to 24".

Stepless ramps are available in two basic versions:

Wide ramps, available in both fixed and retractable models.

Narrow ramps, available in fixed, extendible and retractable models. All ramps are fitted with a handle and corner covering in a non-skid rubber material.

#### **Use**

##### **Plain ramp, wide plain ramp and EasyFold ramp**

The ramp should be placed on a firm underlay. When setting up narrow ramps, the two parts of the ramp should be placed parallel to one another at a distance corresponding to the distance between the wheels of the wheelchair. The ramp should be positioned such that the entire bent contact surface of the ramp is supported. When setting up EasyFold ramps, the ramp should be positioned such that the bent contact surface is supported. Then unfold the upper ramp. The ramp can be set up at a maximum angle of 1:5 when used for electric wheelchairs, or for wheelchairs pushed by a helper. The ramp can be set up at a maximum angle of 1:7 for independent users of manual wheelchairs. To apply with the ADA codes please use 1:12.

##### **Telescopic ramp**

The ramp should be placed on a firm underlay. When setting up narrow ramps, the two parts of the ramp should be placed parallel to one another at a distance corresponding to the distance between the wheels of the wheelchair. The ramp should be positioned such that the entire bent contact surface of the ramp is supported. The ramp can be set up at a maximum angle of 1:5 when used for electric wheelchairs, or for wheelchairs pushed by a helper. The ramp can be set up at a maximum angle of 1:7 for independent users of manual wheelchairs. To apply with the ADA codes please use 1:12.

##### **To extend the ramp**

Hold the ramp by the handle in your right hand, and pull the ramp out to its full length by gripping the contact surface of the inner part of the ramp. Position as required. If the catch device should be moving too tightly, it can be lubricated with oil or another lubricant.

## **EasyFold telescopic ramp**


The ramp should be placed on a firm underlay.

The ramp should be positioned such that the entire bent contact surface of the ramp is supported. The ramp can be set up at a maximum angle of 1:5 when used for electric wheelchairs, or for wheelchairs pushed by a helper. The ramp can be set up at a maximum of 1:7 for independent users of manual wheelchairs. To comply with the ADA codes please use 1:12.

The ramp should be carried by the handles and the contact surface positioned on the upper level. Then unfold the upper ramp. The upper ramp closest to the body is extended to the desired length and placed on the upper level. Afterwards the other half of the upper ramp is extended.

Always check that the ramp is placed in a stable and safe way.

To prevent the EasyFold ramps from sliding on underlays with low friction, the safety pin should always be mounted before using the ramps.

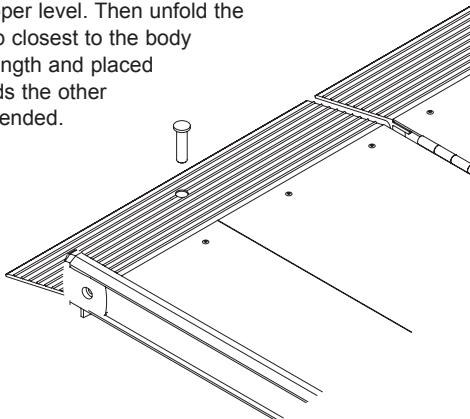### ***Important!***

Do not start up electric wheelchairs which have stopped on the ramp, as the various parts of the ramp may extend unintentionally by so doing. In the event of a wheelchair stopping in the middle of the ramp, reverse down and start from the bottom again.

## **Folding ramp, wide folding ramps**

When setting up narrow ramps, the two parts of the ramp should be placed parallel to one another at a distance corresponding to the distance between the wheels of the wheelchair.

The ramp should be positioned such that the entire bent contact surface of the ramp is supported.

The ramp can be set up at a maximum angle of 1:5 when used for electric wheelchairs, or for wheelchairs pushed by a helper.

The ramp can be set up at a maximum angle of 1:7 for independent users of manual wheelchairs. To apply with the ADA codes please use 1:12.

The ramp should be placed on a firm underlay.

### ***Important!***

Unfold the ramp by gripping both handles.

To avoid anything getting caught up in the hinge, the ramp should be held with the hinge opening away from the body while it is being opened and closed.

1.03	<b>Important/warnings</b> <ul style="list-style-type: none"><li>• Read all the operating instructions before using the ramp.</li><li>• Do not exceed the load marked on the ramp.</li><li>• The ramp should only be used for driving wheelchairs up and down.</li><li>• Wheelchairs should be driven up and down at low speed - without a run-up.</li><li>• The ramp should be positioned such that the entire contact surface is supported.</li><li>• The ramp should be used on a firm underlay.</li><li>• The ramp should not be used for gradients greater than 1:5 – for electric wheelchairs or wheelchairs pushed by a helper.</li><li>• The ramp should not be used for gradients greater than 1:7 – for manual wheelchairs. To apply with the ADA codes please use 1:12.</li><li>• Collapsible ramps should be held away from the body whilst being unfolded.</li></ul>
1.04	<b>Unpacking and preparation of the ramp</b>  <b>Visual check of the ramp</b> If the packaging is damaged on receipt, the ramp should be inspected carefully for visible deficiencies, defects or faults. If damage is suspected, the ramp must not be used before qualified service personnel or the Guldmann Service Team have authorised it.
2.00	<b>Description of functions</b>
2.01	<b>Pictograms/labels</b> <ul style="list-style-type: none"><li>• Stepless label by the handle on the side of the ramp.</li><li>• Label stating max. load.</li></ul>
3.00	<b>Transportation</b> The ramp should be carried by the handle(s) mounted on the left side of the ramp.
4.00	<b>Maintenance and storage</b>
4.01	<b>Cleaning the ramp</b> Clean the ramp with a damp cloth, sponge or soft brush applied with a normal cleaning agent. The lifting platform may also be rinsed under running water from garden hoses or the like. Chemicals and autoclaving may not be used to clean the ramp.
4.02	<b>How to prevent/avoid corrosion</b> The ramp is made from corrosion-resistant materials, and does not require further corrosion prevention.

---

4.03	<p><b>What maintenance is required by the owner?</b></p> <p>The hinges on the Stepless folding ramp should be lubricated with a light lubricant every six months, or as required.</p> <p>The sliding rails on the Stepless telescopic ramp should be wiped with a dry cloth as required.</p>
------	--

If the ramps are slippery (covered with ice), use salt on the ramp – NEVER gravel.

**The telescope system must not be taken apart by the user**

If the telescope system is taken apart, it should be taken apart and put together again by qualified personnel.

*Tightening moments:* Hinge = 10 Nm. Other = 5 Nm.

*Safety:* All screws and nuts should be fastened with Locktite no. 270

---

## 5.00 Service and useful life

---

5.01	<p><b>Useful life and servicing of the ramp</b></p> <p>Given the maintenance prescribed above, Stepless ramps will have a useful life of 15 years.</p> <p>Spare part lists and drawings can be ordered from the manufacturer or the distributor.</p>
------	--

---

5.02	<p><b>Inspection of Stepless</b></p> <p>Parts which are worn or defective should be replaced by new spare parts from Guldmann Inc.</p>
------	--

**1. Visual check of the product**

- Check the product for wear and tear.
- Check the product for any faults.
- Check that there is no other form of damage to the product.

**2. Testing of the product, as for normal use**

- Check all the product's functions, loaded and unloaded.

**3. Control of the mechanical state of the product**

- Clean dirt and other impurities from the product.
- Inspect and assess vital parts of the product.
- Replace any parts of the product which are defective or worn.

**4. Go through point 2 again to recheck**

**5. Did any new problems arise/were new problems noted in point 4?**

- If new problems arose, go back to point 3.
- If no new problems were noted, conclude the inspection.

**Plain, telescopic and folding ramps**

Lengths .....	20" - 122"
Widths	
outer, max .....	10 1/4" and 11"
inner .....	6 1/4" and 7 1/4"
Max. load under 7 feet / over 7 feet each pair .....	.770 lbs./440 lbs.

**Wide plain and wide folding ramps**

Lengths .....	39" - 118"
Widths	
outer, max .....	34 3/4"
inner .....	30"
Max. load under 7 feet / over 7 feet .....	.770 lbs./440 lbs.

**EasyFold ramp**

Lengths .....	3' - 6'
Width, inner .....	30"
Max. width, folded .....	16"
Max. width, unfolded .....	31 1/2"
Max. load under 9 feet .....	.700 lbs./440 lbs.

**EasyFold Telescopic ramp**

Lengths .....	6' - 10'
Width, inner .....	30"
Max. width, folded .....	16"
Max. width, unfolded .....	32 3/4"
Max. load under 9 feet / over 9 feet .....	.700 lbs./440 lbs.

**All types**

Height of edge .....	1 3/4"
Angle of contact .....	max. 12 degrees

**Weight, min. and max. per ramp, depending on length**

Plain ramp .....	3.3 lbs. - 13.2 lbs.
Telescopic ramp .....	7.7 lbs. - 22.3 lbs.
Folding ramp .....	8.8 lbs. - 22.2 lbs.
Wide plain ramp .....	21.3 lbs. - 62.9 lbs.
Wide folding ramp .....	38.5 lbs. - 69.7 lbs.
EasyFold ramp .....	16.0 lbs. - 32.0 lbs.
EasyFold Telescopic ramp .....	36.0 lbs. - 58.0 lbs.

**Materials**

Covering and handle .....	Impact-proof and non-skid plastic
Ramp slopes .....	Anodized aluminium, with skid resistant surface

Guldmann™

## EC-DECLARATION OF CONFORMITY

Manufacturer      V. Guldmann A/S  
Graham Bells Vej 21-23A  
DK-8200 Aarhus N  
EAN country code: 57  
EAN distribution No.: 07287  
Phone +45 8741 3151  
Fax    +45 8741 3131

Representative      Company \_\_\_\_\_  
Address \_\_\_\_\_  
Country \_\_\_\_\_  
Phone \_\_\_\_\_

Hereby declare that      Product      Stepless ramps

Type No.      XXXXX-YYYY-ZZZZ <sup>x)</sup>

x) XXXXX: Reference No. of the product, YYYY: Serial No. of the product in the production year, ZZZZ: Production year

was manufactured in conformity with the provisions in:

- EN 12182:      Technical aids for disabled persons -  
General requirements and test methods

according to Council Directive 93/42/EEC of 14 June 1993.

Skejby 15.06.2002

Place and day of issue

Product responsible

Technical Manager

## **U.S.A. and countries outside the EU**

---

### **A. Users guide**

**IMPORTANT!** Please read the entire warranties and the entire chapter before the use of the product.

---

### **B. WARRANTIES**

This Stepless product made by Guldmann is designed to be used for the accessibility of persons with a physical handicap who are not able to physically negotiate height differences.

If the product is used in an irresponsible manner or for any use other than that described above or in the enclosed written instructions, the manufacturer's warranty will be nullified and rendered void.

There are no warranties which extend beyond the description in the enclosed written instructions.

Guldmann warrants that this product is free from defects in materials and workmanship under normal use. Guldmann warrants that this product will perform substantially in accordance with the specifications set forth in the documentation provided with the equipment.

The above express warranties are made for a period of 365 days from the date the product is delivered to you as the first user.

Your distributor will replace any product that proves defective in materials or workmanship, without additional charge, on an exchange basis.

Your distributor will either replace or repair without additional charge any Guldmann product that does not perform in substantial accordance with the specifications of the documentation.

Guldmann does not warrant that the functions of this product will meet your requirements. The warranty does not cover any of the part of the product that has been subject to damage or abuse by you. The warranty does not cover any part of the product that has been altered or changed in any way by you or others. Any implied warranties including any warranties of merchant ability or fitness for a particular purpose are limited to the term of the express warranties.

Guldmann shall not in any case be liable for special, incidental, consequential, indirect or other similar damages arising from any breach of these warranties even if Guldmann or its agent has been advised of the possibility of such damages.

You must call Guldmann or your distributor for an authorization to return any defective item during the warranty period. If your distributor is unable to correct your problem by telephone, you will be provided with a return authorization number and address for returning the defective item for warranty service or replacement. You must insure any defective item being returned because Guldmann does not assume the risk of loss or damage while in transit. Do not return items or warranty service to Guldmann.

The warranties set forth above are in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann and should not be relied upon as such. The warranty gives you specific legal rights, and you may also have the other rights which vary from state/country to state/country.

# Stepless™

by Guldmann

#### **Guldmann Sarl**

66 rue du Moulin de la Pointe  
F-75013 Paris  
Tél. +33 145 54 78 36  
No Vert 0800 914 179  
Fax +33 145 57 10 52  
[france@guldmann.com](mailto:france@guldmann.com)  
[www.guldmann.fr](http://www.guldmann.fr)

#### **Guldmann Srl**

Via Focherini 7  
I-43058 Sorbolo (PR)  
Nº Verde 800 781 604  
Tel. +39 0521 660132  
Fax +39 0521 691399  
Cell. +39 340 9087107  
[italia@guldmann.com](mailto:italia@guldmann.com)  
[www.guldmann.it](http://www.guldmann.it)

#### **V. Guldmann A/S**

Head Office:  
Graham Bells Vej 21-23A  
DK-8200 Århus N  
Tel. +45 8741 3100  
Fax +45 8741 3131  
E-mail [info@guldmann.com](mailto:info@guldmann.com)  
[www.guldmann.com](http://www.guldmann.com)

#### **Guldmann Inc.**

5525 Johns Road  
Suite 905  
Tampa, FL 33624  
Tel. 800 664 8834  
Tel. 813 880 0619  
Fax 813 880 9558  
E-mail [info@guldmann.net](mailto:info@guldmann.net)  
[www.guldmann.net](http://www.guldmann.net)