Mobilex "DYNAWALK"
Light weight - Verticalizer/Walking frame
Item No. 313001 & 313002

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

Model 313002
Brake lever on handle

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1. General information

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The Mobilex Product is medical device which is intended to be used only as a standing & walking assistant during walk re-education by patients that because of invalidity are unable to stand & walk independently and who require additional stable support during walking. It gives for such persons possibility to practice their walking skills during rehabilitation process with or without the assistance of a helper. Any other use is prohibited.

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product labels.
- Only use the product for the described purpose.
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- During different adjustments which are possible on the device the user must take care not to squeeze the fingers or other body parts between two moving elements of the device
- The device must only be used on flat, smooth & stable surfaces.
- All wheels **MUST** be in contact with the floor at ALL TIMES during use. This will ensure the product is properly balanced and should avoid accidents.
- When using the device in a stationary position, the hand brakes **MUST** be locked.
- The device must not be loaded with more than 135 kg.
- The device's is dedicated to be indoor used only.
- The device is not intended to be self-propelled while seated
- The seating belt has supportive function only and can't carry the maximum user's weight for extended period of time.
- The device isn't flame retardant. Don't leave it near to sources of fire, ignition or extreme heat.


3. Description

Product description

It offers great manoeuvrability & stability. It is height and width adjustable for a user fit. It is also possible to adjust handles' length and angle. It is equipped with comfortable, elevable & lockable safety back bars and soft, adjustable supporting seat belt which allows user to rest if needed, DynaWalks are supplied in cartons, detached in to few elements for easier transportation:

The contents are:

- 1 Main Frame
- 2 detachable side & arm supports with adjustable handles and safety back-bars
- 1 User manual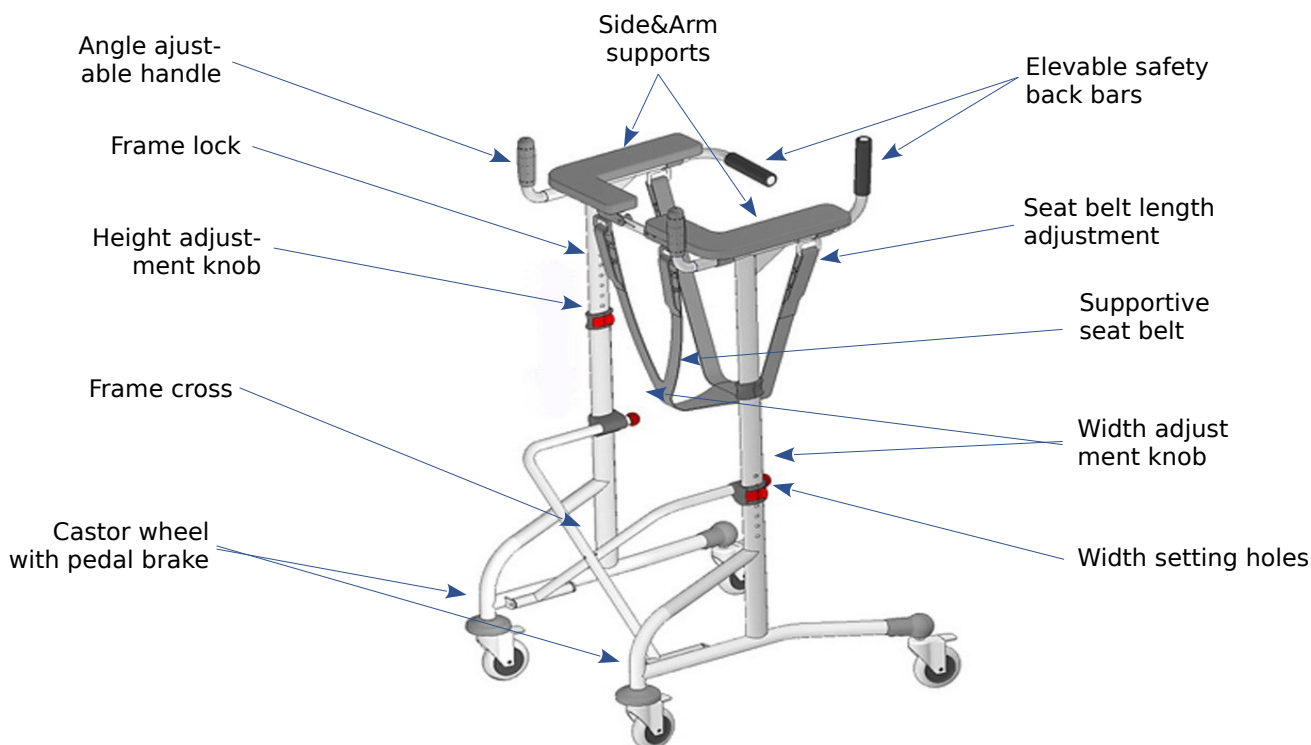


Assembly description & Function description

- **Initial check.** Check with the list from above if nothing is missing. If you'd discover at that stage that some part is missing or damaged, do not continue assembly process. Contact your supplier of the product right away.
- **Frame opening/closing and width adjustment.** Press the front parts of both crossbars downwards until the width adjustment knob will reach the level of one of the width setting holes. Then turn the both knobs clockwise to fix the desired frame width. The lower hole you'll choose, the wider frame width will be set. To close the frame you have to loosen two width adjustment knobs and pull the both crossbars up.
- **Attaching the side&arm supports and height adjustment.** Slide the side and arm supports pole in to the dedicated oval hole on the top of main frame. Choose the desired height of the support and tie the height adjustment knob, so the support is fixed and stable. Repeat above steps to fix the support on the second side of the device. If you want to change adjusted height of side supports, simply loosen the knobs set the needed level and tie the knobs back.

WARNING! Both side&arm supports must be always adjusted to the same height!

- **Locking the arm&side supports.** Pull the locking pin out of its spot. Insert the tube end with smaller diameter, located on one of side&arm supports in to the tube end with larger diameter on the other side&arm support. Align the holes located on both tubes. Then reinsert the locking pin in to its place.

NOTE! To readjust the frame width or to fold the frame, you must unlock the frame lock in the first place!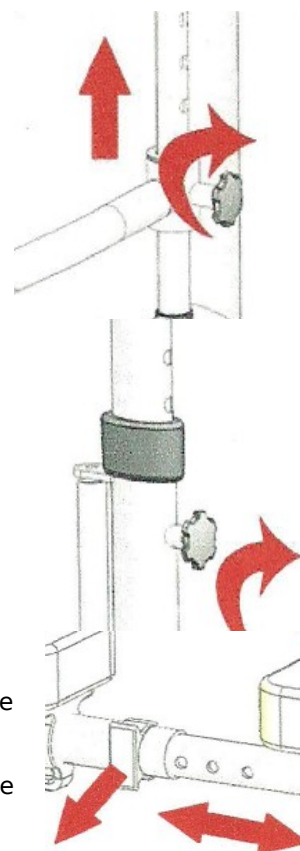

Adjusting the handles position.

- **To adjust the angle** of handle loosen the knob located at the pivoting point, set the desired angle and tie the knob. Repeat above steps for the second handle.
- **To adjust the length** of the handle press the pin button located under the arm support and slide the handle to the desired position. Let the pin button to “catch” the hole located the nearest to desired position. That will lock handle in place.

NOTE! We recommend that both handles should be adjusted to the same length and angle, unless there are important therapeutic reasons to do differently.

- **Using of the elevable safety back bars.** Safety back bars are designed to protect user against tipping to the back and falling of the device. For the user to get in to the device, back bars should be lifted up by pressing the push buttons lock located under the arm supports and pulling them up. When user is placed in the device the back bars should be lowered to horizontal position and locked.

WARNING! It is essential for user safety, that safety back bars are always lowered and locked in flat position always when there is user in the device. The safety back bars should be unlocked and lifted only when user enters or leaves the device.


- **Using of the supporting seat belt.** Seat belt is adjustable in length and attached to the frame with four quick release buckles. When user suppose to enter in to the device release 2 rear buckles of the belt, so the belt would hang on the 2 front buckles. Lead two loose rear ends of the belt between user's legs and lock the 2 released buckle when user is already in the device and safety back-bars are lowered and locked. If it is necessary adjust the length of the belt by shortening or extending 4 tails of the belt.

NOTE! All belt's tails should be adjusted to approximate same length.

NOTE! Correct seating belt length is achieved when the middle (sitting) part of the belt is right between user's legs, about 3-4 cm below user's crotch. User should not seat on the belt at all times during walking. The belt is a safety feature to be used only in the situation of user's exhaustion.

WARNING! Do not remove the belt when user is in the device. The belt must always be in closed position when the device is in use! Otherwise the user may fall down and get injured.


- **Using of the brakes.** The brakes are easily engaged by depressing with a foot the pedal located on top of each castor wheel. To unlock the brake pedal should be lifted with your toes.

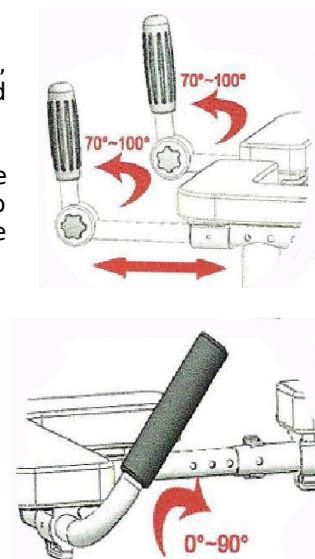Brake lever on model 313002

Rear brake
on model 313002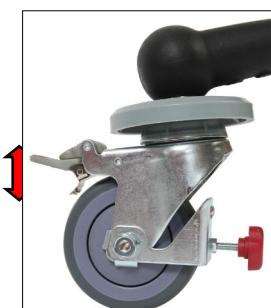

Rear brake with pedal
and sliding brake
on model 312001

WARNING! Always engage brakes when the device is used in stationary position! Never leave device without breaks engaged when there is someone sitting in it!

Failure to follow WARNINGS! & NOTES! can result in personal injury or property damage.
Transport recommendations

The device is a foldable product convenient for transportation. If it is necessary to transport it, we recommend to fold it as it is written in description above. Device should be transported in the way that it is stable and there is no risk of sudden and dangerous movements of the transported device.

Accessories

There are no accessories provided for this device

4. Technical data

Walking frame Mobilex Bison	Item nr. 313001 Standard	Item nr. 313002 Brake
Max. load	135 kg	135 kg
Weight	12,5 kg	12,5 kg
Length	82 cm	82 cm
Width (adjustable)	67,5 - 75 cm	67,5 - 75 cm
Height of armrests (adjustable)	100 - 123 cm	100 - 123 cm
Max. height of the handles	136 cm	136 cm
Front brakes	2 lockable pedal brakes	2 lockable pedal brakes
Rear brakes	2 lockable pedal brakes, with friction	2 lockable brakes on the handles

Product label

The product label is located on the frame tube.

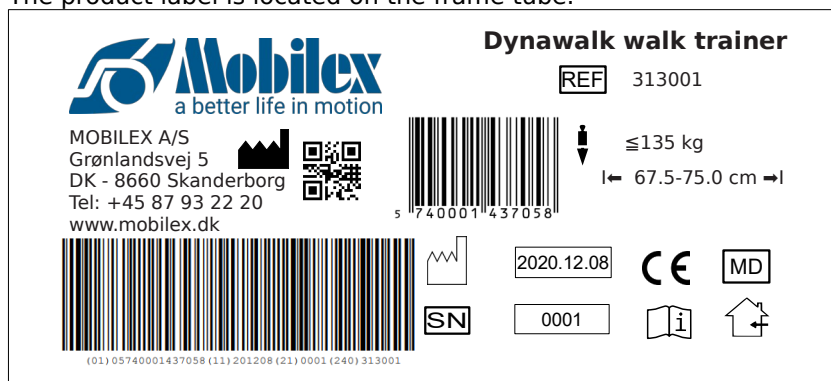
Image of the product label for Dynawalk 313001

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

5. Maintenance & Cleaning

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only in one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
The seat does not fit in when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices. The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

DynaWalk

Verticalizer/Walking Frame 313001, 313002

Harmonized norms used during conformity estimation:

PN-EN ISO 11199:2005, EN ISO 14971:2012, EN 1041:2009

EU2017/745 for Medical Equipment

(über Medizinprodukte / for medicinalprodukter / per prodotti medicinali / sur les équipements médicaux)

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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