Mobilex rollator "LION"

Art. 312371, 312372, 312373 & 312374

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

Metallic brown
Indoor & Outdoor

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1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a description of the medical device and important guidance to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these. We continuously develop our products and we reserve the right to change the specifications and functions of products without further notice.

Intended use of the device

The Mobilex rollator "Lion" is a medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily chores. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

Indications

The device is indicated for use as walking, standing and seating support by persons who suffer different diseases of human motion system. The variety of such diseases is very wide but the symptoms are similar: people who because of their health status need stable walking support and are indicated to use rollators. The final decision to choose the correct type of device depends on condition of the individual patient and should always be taken by professional physician or physical therapist

NB The rollator should not be used by persons with balance problems whose muscles are severely weakened, thus unable to stand or walk on their own. The rollator should be recommended by a physiotherapist or a doctor.

If you have any questions, please contact your dealer or Mobilex A/S

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

2. Security instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes have been firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 200 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturer's written permission
- The rollator must always be braked when not in use

3. Description

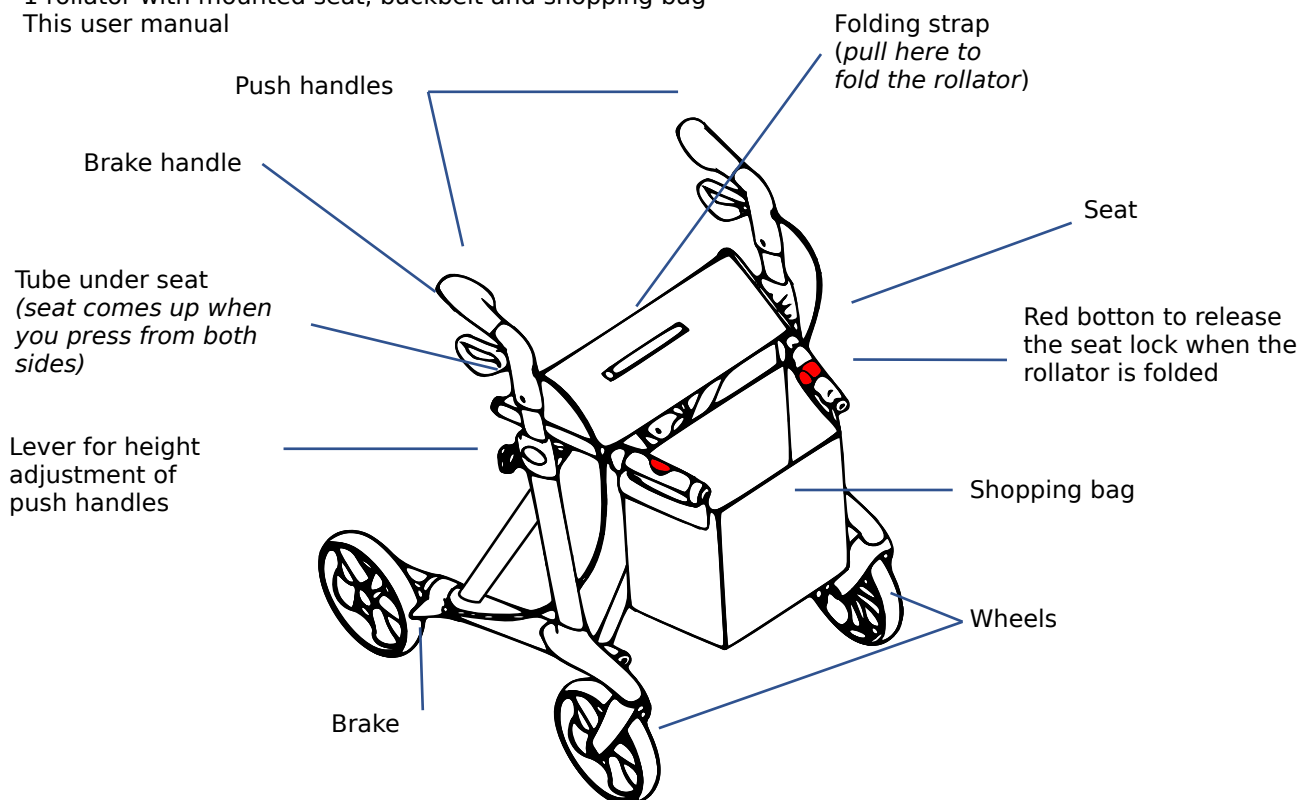
Product description

The Mobilex rollator "Lion" has a powder coated reinforced frame with 4 big wheels (see the technical data). The light-weight frame makes it easy to fold the rollator together and offers easy storage and transport. The rollator runs easily and is sufficiently stable. The height of the underarm supports can be easily adjusted. The rollator is delivered with a mounted seat, a backbelt and a shopping bag.

Parts supplied and mounting instructions

The Mobilex rollator "Lion" is delivered in a carton with the following content:

- ✓ 1 rollator with mounted seat, backbelt and shopping bag
- ✓ This user manual

Control upon delivery

After unpacking the rollator please check with the list above that everything is there. If you discover that some parts are missing or damaged, do not continue the assembly process. Contact your dealer right away.

Unfolding the rollator

- Press on the red button on the seat tube lock (see picture on the right)
- Press down both tubes under the seat until the frame unfolds and you can hear a "click". Now the locking device is in a fixed position.
- To be quite sure that the frame has been fully unfolded try to press the rollator together
- Check that the levers for height adjustment have been firmly tightened
- Check that both handles are at the same height and stable

Use of a defective rollator is strictly forbidden

Always check the function of all parts of the rollator before use. If the above recommendations are not followed, injuries may occur.

Folding the rollator

- First detach the strap which is mounted between both push handles
- Pull the strap at the seat until the frame folds
- Now the rollator is folded together

Height adjustment of underarm supports

- First unlock the strap which is mounted between both push handles
- Loosen the lever by turning it to the left
- Adjust the height
- Turn the lever to the right
- Attach the strap again

Both handles should always be at the same height

The correct height of the handles should always be the elbow height of the user when he/she stands with his/her arms hanging loose.

Length adjustment of push handles

- First detach the strap which is mounted between both push handles
- Loosen the lever by turning it to the left
- Adjust the length
- Turn the lever to the right
- Attach the strap again

Brake function

- To brake you must pull both brake handles towards yourself

- The more you pull the higher the braking capacity
- To ensure that the rollator remains STOPPED you must keep the brake handles pulled
- To make the rollator MOVE you must let go of the brake handles whereby the braking will stop

Never release the brakes when seated on the rollator

Always activate the brakes when not using the rollator. When placing the rollator on a non-plane surface you must always activate the brakes.

Activating and deactivating the brakes

- To stop the rollator pull the brake handles upwards
- To park the rollator push the brake handles downwards until a click is heard
- Do not sit on the rollator unless the brakes are in a locked position
- Before setting the rollator into motion you must stand up in the walking direction and release the brake handles by pulling them upwards

The brakes must always be in a locked position when the rollator is parked.

Never use the seat if the brakes are not locked. When seated, never lean backwards. It is recommended to sit only at the front part of the seat. Do not sit down on the bag.

Function description

As the frame can be folded together the rollator can easily fold and unfold, and as a result it requires very little space. Push handles can be freely adjusted to fit the size of your body. The seat allows you to rest.

Accessories

For the Mobilex rollator “Lion” various accessories are available with the following article no. 312340 312317 312323 312322 312331 31234A 312366 312367 312368 31236A 31236B 312321. Please contact your dealer.

Transport recommendations

It is recommended to transport the rollator in the folded position. Its construction allows easy unfolding and folding. You should check that the rollator is in a stable position during transport to avoid any deformation.

Use

Stand in the middle behind the back wheels. Grab the push handles and smoothly start walking. Due to the special design of the brakes, only a minor effort is required to activate the brakes.

4. Technical data

Mobilex rollator Lion	Standard	Small
	Item nr. 312372 (antracite grey) Item nr. 312374 (metallic brown)	Item nr. 312371 (antracite grey) Item nr. 312373 (metallic brown)
Height of the push handles (adjustable)	90-112 cm	77-90 cm
Recommended user heights	170 - 220 cm	165 - 175 cm
Seat height	62 cm	53 cm
Total width opened / folded	68 / 26 cm	68 / 26 cm
Inner width between push handles	56 cm	56 cm
Length (depth)	76 cm	76 cm
Weight (+ bag 380 g)	9,3 kg	9,0 kg
Max. load (rollator / bag)	200 kg / 5 kg	200 kg / 5 kg
Brakes	2 lockable with parking function	
Wheel sizes (soft wheels)	Front 285 x 44 mm - back 235 x 44 mm	

Product label - placed on the lower frame

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Example of product label Lion 312371

Overview icon meaning

5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

Recommended service life: Normally 10 years depending on use.

Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

Warning

If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

Warning

When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

Storage

We recommend storage in a dry place.

Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
The rollator is difficult to guide or push	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
The brakes do not work or work only in one side	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
The seat does not fit in when the rollator is unfolded	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Lion Outdoor rollator– item no. 312371+72+73+74

ACCESSORIES LIST

Item nr.	Accessories item nr.
312371+72 +73+74	312340 312317 312323 312322 312331 31234A 312366 312367 312368 31236A 31236B 312321

Harmonized norms used during conformity estimation:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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